



HEELCAP, ROBUST FABRIC AND FITTED WITH RIDGE ON SOLE CONTOUR TO HOLD THE BRACE IN PLACE **AVOID MINIMALIST SHOES WITH VERY LIGHT FABRIC AND THIN SOLE.**

ROBUST FABRIC BULKY SOLE+RIDGES FOR HEEL FRICTION GRIP







ASK PATIENT TO DON THE SHOE

INSTALL TIE-WRAPS IN EYELETS. & SELECT WIDTH FOR HOLES. NEVER ATTACH AROUND THE LACES!



LOCK TIE-WRAPS IN REAR CORNERS OR BEST FIT POSITION.

p.4



TIGHTEN TIE-WRAPS AT MAXIMUM VERY STRONG FIXATION IS IMPORTANT! PULL FIRMLY ON LACE CLIP AND VALIDATE IT'S SOLIDLY ANCHORED AND BONDED TO THE SHOE.

HOOK THE BRACE ON THE SHOE AND YOU'RE READY FOR ACTION! SEE LACE CLIP FITTING LEAFLET (VIDEOS TUTORIALS AVAILABLE

FOR MORE FOOTWEAR OPTIONS. ON YOUTUBE & OUR WEBSITE)




ASSEMBLY INSTRUCTIONS

XTERN SEMI-ASSEMBLED ORTHOTIC KIT SIZE SMALL (100100-SA-S)





2018-04-10 - 100100-SA-S XTERN

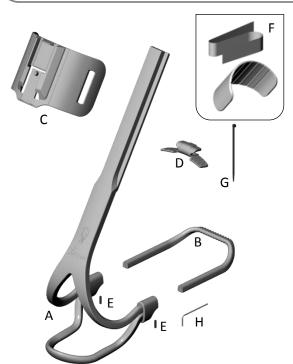



SUGGESTED TOOLS

- -POWERED DRILL (VARIABLE SPEED)
- -DRILL BIT 7/64" (2.8 MM)
- -HEX ALLEN KEY 1/16" (INCLUDED)
- -METAL HAND SAW
- -MEASURING CALIPER 300MM OR MEASURING TAPE
- -BENCH VISE WITH SOFT PAD JAW (PLASTIC / LEATHER)
- -TRAUTMAN SANDER AND CARVER
- -CUTTER PLIER
- -PROPANE BLOW TORCH (IF THERMOFORMING REQUIRED)

-PROTECTIVE SAFETY GEAR:


SAFETY GOGGLES, GLOVES, DUST MASK & ALL OTHER PROTECTION REQUIRED BY YOUR HEALTH & SAFETY WORK POLICY

	PARTS LIST - XTERN ORTHOTIC KIT, SIZE SMALL (100100-SA-S)											
Item	Qty	PRODUCT CODE	DESCRIPTION									
Α	1	200000-BA0-S	MAIN BRACE BODY ASSEMBLY, SIZE SMALL									
В	1	200000-U00-S	FRONT SHOE SECTION «U» SHAPE, SIZE SMALL									
С	1	200000-C00-S	PLASTIC CALF SECTION, SIZE SMALL									
D	5	200000-L00-S	LACE CLIP ATTACHMENT, SIZE SMALL									
E	3	700000-SET-002	SET SCREW 5-40 X 1/2"LONG. 18-8 STAINLESS STEEL									
F	1	500301-K00-SML	COMPLETE STRAP & CUSHIONS KIT									
G	20	700000-ZZZ-001	TIE-WRAP BLACK 18 LBS/4"LONG									
Н	1	700000-HEX-001	HEX-L-KEY 1/16" (FOR 5-40 SET SCREW)									


T: 1-888-778-8726 www.turbomedorthotics.com T: 1-888-778-8726 www.turbomedorthotics.com


XTERN SMALL KIT - LENGTH ADJUSTEMENT CUTTING CHART													
SHOE SIZE	US-MEN	2.5	3	3.5	4	4.5	5	5.5	6	6.5	7	7.5	8
SHOE SIZE	US-WOMEN	3.5	4	4.5	5	5.5	6	6.5	7	7.5	8	8.5	9
CUTTING REF.	X	14	18	22	25	30	34	38	42	46	49	52	55

SEE GRADUATIONS ENCARVED UNDER FRONT "U" SECTION

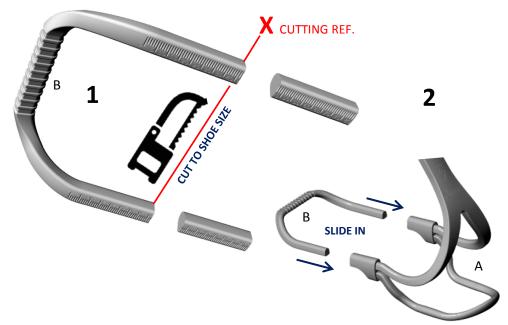

	XTERN SMALL KIT - LENGTH MEASUREMENT CHART													
SHOE SIZE	US-MEN	2.5	3	3.5	4	4.5	5	5.5	6	6.5	7	7.5	8	
SHOE SIZE	US-WOMEN	3.5	4	4.5	5	5.5	6	6.5	7	7.5	8	8.5	9	
FINAL LENGTH	L (mm)	160	164	168	171	175	180	183	187	190	193	196	199	

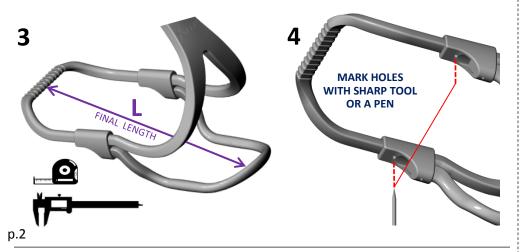

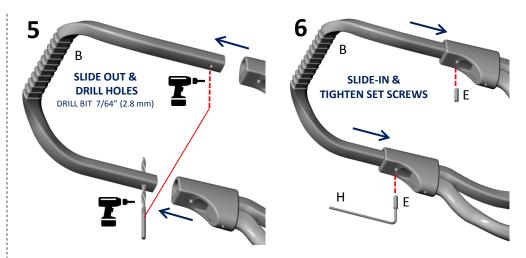

XTERN SMALL KIT - CALF SUPPORT ADJUSTEMENT CHART													
SHOE SIZE	US-MEN	2.5	3	3.5	4	4.5	5	5.5	6	6.5	7	7.5	8
SHOE SIZE	US-WOMEN	3.5	4	4.5	5	5.5	6	6.5	7	7.5	8	8.5	9
CUTTING REF.	Н	3	3	3	3	3	2	2	2	2	2	2	2

SEE NUMBERS MARKS ON PART

ALIGN MARKS

ALIGN MARKS

ALIGN MARKS

SEE ENGRAVED

SEE ENGRAVED

MARKS ON PART

MARKS ON PART

SUGGESTED CUTTING HEIGHT, WE RECOMMEND

TO MEASURE CALF HEIGHT ON PATIENT

