GENU-TEX® SPORLASTIC

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CONTRA-INDICATIONS

Higher-grade axis deviation > approx. 15°

Peripheral arterial occlusive disease with popliteal arterial occlusion and bypass revascularisation or marginal peripheral circulation reserve, vascular compression risk

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

The product is to be used by one patient only.

The very first time the GENU-TEX® is adjusted this must be done by trained technical staff, otherwise no product liability is warranted.

Improper modifications to the product referred to above exempt the manufacturer from product liability.

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QUALITY MANAGEMENT SYSTEM

All SPORLASTIC GmbH products undergo product testing as part of our Quality Management System.

However, should you have any complaints about our product, please get in touch with your stockist or directly with us.

CARE

Wash GENU-TEX® in lukewarm water with a cold-water detergent and air dry. Do not place on a radiator. The enclosed heel pad can be cleaned with a moist cloth.

When washing the GENU-TEX® please remove the two splints from the brace. Close the Velcro fasteners before washing. The Velcro fasteners will then last longer.

SIDE-EFFECTS

With proper use and proper fitting, so far there have been no reports of serious general side-effects.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restricting consistently shaped body fitting.

So far no consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

With proper use and proper fitting, so far there have been no reports of serious general side-effects.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restricting consistently shaped body fitting.

So far no consequential incompatibilities or allergic reactions have been reported.

FITTING

To fit the GENU-TEX®, it is better for you to sit on a chair so that your knee is slightly flexed.

After opening all the Velcro fasteners, grasp the GENU-TEX® with both hands holding the integrated pull straps with the popliteal stitching showing. Then pull the GENU-TEX® over your knee until it is below the popliteal cavity (Fig. 1).

Then turn the GENU-TEX® 180° at the lower leg. The popliteal stitching shows at the calf (Fig. 2).

Next, pull the GENU-TEX® over your knee until the silicone ring encloses the kneecap well (Fig. 3).

Now close the Velcro fasteners (upwards) as numbered in Fig. 4 with slight tension.

Stand up and check again whether the silicone ring still encloses the kneecap well (Fig. 5). If necessary, re-adjust the GENU-TEX® to your leg and then pull all the Velcro fasteners in such a way that all 8 fastening straps fit the leg well with even traction (Fig. 5).

To remove the GENU-TEX®, generally all you have to do is just open the front Velcro fasteners and proceed as described in 2-3 above in the reverse order.

INSTRUCTIONS FOR THE ORTHOPAEDICS TECH

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