

Madglove AssistTM Sizing Guide

Madglove AssistTM has two parts to size

Wrist Splint



1



Wrist circumference - Wrap the tape around your wrist, just above the bony bump.

Record in cm: _____ cm

2



Hand width - Place the tape across the back of your hand.

Record in cm: _____ cm

Finger Sleeves



3



Middle finger length - Measure from the tip of your middle finger to MCP knuckle.

Record in cm: _____ cm

4



Middle finger circumference - Measure around the tip.

Record in cm: _____ cm



Combine Your Sizes

1. Choose Left or Right side

2. Write your final fit _____

Example: Right Hand, Medium Splint + Small Sleeves

Splint Size Chart

Splint size	Wrist circumf.	Hand width
Small (S)	<15 cm	<8 cm
Medium (M)	15-18 cm	8-9 cm
Large (L)	18-21 cm	9-10.5 cm

Finger Sleeve Size Chart

Sleeve size	Finger Length	Finger circumf
Extra Small (XS)	<7 cm	<4.5 cm
Small (S)	7-8 cm	4.5-5.5 cm
Medium (M)	8-9 cm	5.5-6.5 cm
Large (L)	9-10 cm	6.5-7.5 cm

Tips

1. Measure twice to confirm.
2. If you're between sizes:
Splint → go up a size
Sleeves → go down a size
3. Measure at the time of day you'll most often use the Madglove (hands can swell).