madglove  
Amsterdam

Product Manual

MADGLOVE ASSIST

## English

### Madglove Assist Product Manual

#### 1. Introduction

Welcome to the Madglove Assist manual! This manual will guide you on how to safely and effectively use your Madglove Assist. *The Madglove Assist is designed to help you with hand function (extension) and promote independence in daily activities. It gives adjustable support for different needs, so it can assist you at actively participating in your life.*

**Important:** Before using the Madglove Assist, please read this entire manual carefully. Each section has clear instructions and pictures to help you use it safely and correctly. In any case of overextension with fingers we advise to combine the Madglove Assist with products that help mitigate this risk. If such additional product is deemed necessary is best assessed by a medical professional.

**If you have any questions or need help, contact Madglove (see Section 8 for contact information) or talk to your healthcare professional.**

#### 2. Indications, Contraindications, Precautions, and Adverse Effects

This section explains when you can use the Madglove Assist, when you should NOT use it, what you need to watch out for, and possible problems.

##### 2.1. Indications (When You Can Use It)

The Madglove Assist is for adults (18 years and older) who have upper limb and hand disabilities that affect their daily life. These disabilities may include:

- Spasticity (levels 1-3 on the Modified Ashworth Scale)
- Muscle Weakness
- Limited Range of Motion

The Madglove Assist helps by:

- Supporting the wrist
- Helping with finger and thumb extension
- Correcting hand and finger position

**2.2. Contraindications (When You Should NOT Use It)** Do NOT use the Madglove Assist if any of these apply to you:

**Severe Skin Conditions:** Open wounds, severe rashes, untreated infections, or active skin lesions on the areas where the glove touches your skin.

**Severe Peripheral Neuropathy:** Significant loss of feeling in your hands or reduced ability to feel pain, pressure, or temperature changes.

**Acute Joint Inflammation:** Active arthritis flare-ups or other acute joint inflammation.

**Uncontrolled Medical Conditions:** Uncontrolled medical conditions that could be negatively impacted by pressure or specific hand positions, including deep vein thrombosis, compromised circulation, or acute inflammatory conditions.

**Allergies:** Known allergies to any of the materials in the Madglove Assist. (See Section 6 for materials. *If you are unsure, consult your healthcare professional.*)

**Fractures or Unhealed Injuries:** Untreated fractures, open wounds, or unhealed injuries in the hand or wrist area.

**Fixed tendon shortening or contracture:** In these cases, wrist stabilization may increase finger flexor tone, reducing hand opening and causing discomfort. Functional electrical stimulation may help mitigate this effect. Consult with your healthcare professional.

**If any of these apply to you, DO NOT use the Madglove Assist and consult with your healthcare professional.**

#### 2.3. Precautions (What to Watch Out For)

**Proper Fit:** Make sure the Madglove Assist fits your hand correctly. A poor fit can cause discomfort, reduced effectiveness, or injury. Adjust the straps and tensioning system. *Refer to Section 5 for fitting instructions.*

**Skin Monitoring:** Check your skin regularly for irritation, redness, swelling, or allergic reactions. Pay attention to pressure points under the glove and straps. If any skin reactions occur, stop using the Madglove Assist and consult your healthcare professional.

**Pressure Monitoring:** Check for too much pressure from the glove or splint (at least 1 time per hour). Adjust the tightness and tension for comfort. Do not overtighten.

**Sensation:** If you have reduced sensation in your hands, be extra careful. Watch for signs of discomfort, numbness, tingling, or skin color changes (paleness or blueness) during and after use. Using the device with a caregiver present is advised.

**Limited Mobility:** The Madglove Assist is a supportive device and not a replacement for professional physical therapy or other medical interventions. It is a supplementary aid to support daily activities.

**Cleaning:** Always clean the Madglove Assist and its components properly. *Refer to Section 6 for cleaning instructions.*

**Storage:** Store the device in a clean, cool, dry place to prevent damage. Avoid extreme temperatures and direct sunlight.

**Regular Inspection:** Regularly check for wear and tear or malfunctions. If damage is detected, stop using the device and contact Madglove.

**If you experience any discomfort or unusual symptoms while using the Madglove Assist, stop using it immediately and consult your healthcare professional.**

#### 2.4. Adverse Effects (Possible Problems)

While the Madglove Assist is designed to be safe, problems can happen. These are usually related to improper use, sensitivities, or pre-existing conditions. Stop use and consult your healthcare professional if any of the following occur:

**Skin Irritation or Allergic Reactions:** Redness, itching, rash, swelling, or blistering where the device touches your skin.

**Compromised Circulation:** Numbness, tingling, coldness, or discoloration (paleness or blueness) in your fingers or hand.

**Increased Pain or Discomfort:** Significant increase in pain or discomfort.

**Worsening of Existing Conditions:** If you have arthritis or peripheral neuropathy, the Madglove Assist may worsen your symptoms.

**Pressure Sores:** Prolonged or excessive pressure can lead to pressure sores. Check skin regularly (at least 1 time per hour of wear).

**Damage to the Device:** Damaged components can increase the risk of injury.

### 3. Getting Started

**3.1. What's Inside the Kit?** Your Madglove Assist kit includes everything you need:

**The Madglove Assist:** The glove itself, already fitted with its mechanism cover.

**Reusable Washing Bag:** Use this mesh bag to protect the glove during cleaning.

**Two Sets of Finger Sleeves:** One set is thicker and more rigid, and the other is thinner and more elastic. Choose the set that best fits your needs.

**Proper use:** Properly decide on which finger set to use refer to Section 3.2 to set up the Madglove Assist.

**Pusher and Plug:** For the tension system.

**Instruction Manual:** This manual

**The magnetic closure box:** It is used for closure for transport but also can be used for storage after use.

#### 3.2. Choosing the Right Finger Sleeves

The Madglove Assist comes with two sets of finger sleeves to meet your needs.

**Thinner, More Elastic Sleeves:**

**Enhanced Dexterity:** Gives more freedom of movement for tasks that need precision.

**Superior Comfort:** Flexible and less restrictive.

**Improved Tactile Feedback:** Allows better sensitivity and touch.

**Thicker, More Rigid Sleeves:**


**Increased Support and Stability:** Prevents unwanted movements, good for more spastic fingers.

**Enhanced Control:** Aids in better control of finger movements, beneficial for individuals with spasticity or reduced fine motor control.

You can also mix and match sleeves, using thinner sleeves on some fingers and thicker ones on others. Experiment to find what works best for you. Omit sleeves on fingers that don't require support.

**Always follow your healthcare professional's advice.**

*The icons below can be found on the fingertips and indicate for which finger the sleeve is made.*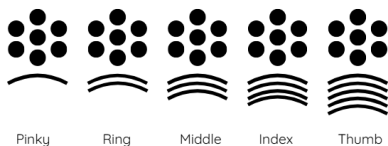


### 4. Operating the Madglove Assist

**For a more detailed explanation please go to our website for video instructions on how to set up the Madglove Assist.**

#### 4.1. How to Set Up

**Preparation:** Lay the glove flat on a clean, dry surface with all Velcro straps detached.

**Wrist Support:** Align the back of the hand on the open groove and place your hand's knuckles with the glove's top edge, placing the pinky-side of your hand onto the side with three elastic bands. Secure the wrist support by attaching the Velcro straps; start with the grey tab that is touching the skin first, then the parallel tab. Adjust the tightness to your preference using the remaining Velcro straps, repeating for both straps.

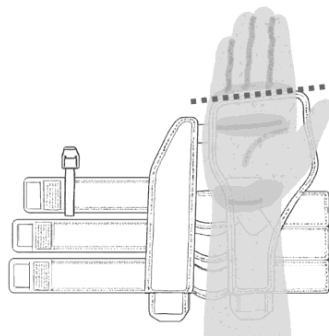**Finger Sleeves:** Ensure the tension wheel is fully released (no tension). Gently pull the finger sleeves over your fingertips, making sure the inside of your fingers is exposed, and the top of each finger is covered by the sleeve.

**Setting Finger Lengths:** For initial use, assistance is recommended. Locate the ropes at the bottom of each finger sleeve. Loop these ropes through the sliders at the top of the back of the glove and loop them through the tiny holes of the center point component. To secure these ropes push them into the matching groove at the top of the Center Point component. Make sure to loop the finger sleeve for the pinky finger on the side closest to the 3 bands on the side of the glove. The index finger furthest from these 3 connecting bands. To adjust the length of the sleeves, ensure that the Center Point component is aligned with the white indication line on the back of the hand before tightening the ropes back into place. For correct tensioning make sure you feel or can see slight pulling on the fingers when in this position. Push in the plug in the center point to lock the ropes in place. Use the pusher to remove the plug when needed. See the below pictures for guidance.

**Applying the Cover:** Position the cover over the glove, ensuring the circular cutout at the bottom aligns correctly over the tensioning wheel, and the cutouts at the top of the cover align with the sliders. Click the push buttons on the right and left side of the cover in place. Note the letter on the cover that aligns with the top of the center

point to record the current tension setting for future reference. If your medical professional has advised a specific tension setting, adjust accordingly until the indicated letter aligns.

**Applying Tension:** Rotate the tension wheel to apply tension. Adjust the tension to your desired level, but ensure it does not cause pain, discomfort, or overextension of your fingers. Regular adjustment of the tension may be necessary throughout use. To release the tension immediately remove the full glove. When too much tension is applied, rotate to the other side to adjust tension.

**Removal:** To remove the glove, first loosen the tensioning mechanism. Then, remove the Velcro wrist straps (first the tightening Velcro tab, after the closing Velcro tab. followed by the Velcro straps with grey tabs. The glove can now be removed.

#### 4.2 Troubleshooting

**Problem:** The glove feels too tight

**Solution:** Loosen the velcro straps around the wrist and reduce tension with the tensioning wheel

**Problem:** The glove is slipping

**Solution:** Tighten the velcro straps around the wrist and play with the amount of tension with the tensioning wheel If you encounter any difficulties during setup or have any questions, please refer to the illustrations in this manual or contact your healthcare provider for assistance.

#### 5. Care and Cleaning

**Handwashing:** The Madglove Assist and its components (finger sleeves, cover, etc.) can be spot cleaned using mild soap and lukewarm water. Gently rub the materials to remove dirt and grime. Avoid (harsh) chemicals, abrasive cleaners, and bleach, as these may damage the materials.

**Drying:** Allow the Madglove Assist to air dry completely before storing. Do not use a dryer or other heating methods, as these may cause damage to the materials.

#### 6. Product Specifications

**Sizes:** Small, Medium, Large (for both left and right hands)

**Glove Materials:**

60% Polyester 20% Neoprene (In direct contact with the skin)

16% Polyethylene

2% Nylon

2% Latex (In direct contact with the skin)

**Finger Sleeves Materials**

100% Polyester (thin finger sleeves)

100% Neoprene (Thick finger sleeves)

#### 7. Warranty Information

Madglove warrants that the Madglove Assist device will be free from defects in materials and workmanship for a period of one (1) year

from the date of original purchase. This warranty applies only to the original purchaser and is not transferable.

#### What is Covered:

This warranty covers defects in materials or workmanship that cause the Madglove Assist to not function according to the specifications outlined in this manual.

#### What is Not Covered:

This warranty does not cover:

- Damage caused by misuse, abuse, accident, or improper handling.
- Normal wear and tear.
- Damage caused by unauthorized repairs or modifications.
- Damage caused by improper cleaning or storage.
- Cosmetic damage that does not affect the functionality of the device.

#### How to Make a Claim:

To make a warranty claim, please follow these steps:

1. Contact Madglove Customer Support through our website complaint form, this will guide you through all information needed in order for us to solve the issue.
2. Securely package the Madglove Assist device, including a copy of your original proof of purchase (receipt or invoice) and a detailed description of the defect.
3. Ship the package to the address provided by Customer Support.

Madglove will, at its option, repair or replace the defective device with a new or refurbished device of equal or greater functionality. The repaired or replacement device will be warranted for the remainder of the original warranty period.

#### Limitation of Liability:

In no event shall Madglove be liable for any indirect, incidental, special, or consequential damages arising out of or in connection with the use or inability to use the Madglove Assist device, even if Madglove has been advised of the possibility of such damages.

#### Your Rights:

This warranty gives you specific legal rights, and you may also have other rights which vary from country to country.

#### 8. Contact Information

##### Madglove

Unit 1A-08, De Boelelaan 1095a, 1081 HV Amsterdam

Website: [www.madglove.com](http://www.madglove.com)

Email: [info@madglove.com](mailto:info@madglove.com)

**Note:** The information provided in this manual is for informational purposes only and does not constitute medical advice. Always consult with your healthcare professional for any health concerns or before making any decisions related to your treatment or care.

## Madglove Assist Product Handleiding

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