 embreis

Comfit AFO[®]

STANDARD / XTRA STRONG

A.

B.

C.

D.

E.

F.

Art / REF no	Trade name/ model/ variant	Side	Size	Shoe Size
E806113	Comfit AFO Standard	Left	Small	< 42
E806117	Comfit AFO Standard	Left	Large	42 - 47
E806123	Comfit AFO Standard	Right	Small	< 42
E806127	Comfit AFO Standard	Right	Large	42 - 47
E806157	Comfit AFO Standard wide	Left	Large	42 - 47
E806167	Comfit AFO Standard wide	Right	Large	42 - 47
E807113	Comfit AFO Xtra Strong	Left	Small	< 42
E807117	Comfit AFO Xtra Strong	Left	Large	42 - 47
E807123	Comfit AFO Xtra Strong	Right	Small	< 42
E807127	Comfit AFO Xtra Strong	Right	Large	42 - 47
E807157	Comfit AFO Xtra Strong wide	Left	Large	42 - 47
E807167	Comfit AFO Xtra Strong wide	Right	Large	42 - 47
E8061PS03	Comfit AFO classic pad kit	-	Small	< 42
E8061PS05	Comfit AFO classic pad kit	-	Large	42 - 47

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Intended purpose

Comfit AFO is an ankle-foot orthosis intended for adults with drop foot or reduced dorsiflexion due to neurological conditions or injury. The device supports dorsiflexion during gait and assists in restoring a more natural walking pattern. It is intended to be fitted by orthotists or other healthcare professionals in a clinical environment and subsequently used by the patient in daily life.

Indications

Comfit AFO is intended for patients with drop foot due to neurological conditions or injury. It is also intended for patients with weakness in the dorsiflexor muscles.

Contraindications

The device should not be used in patients with open wounds or skin lesions at the application site, severe foot deformities preventing proper fitting, moderate to severe spasticity of the lower limb, or known hypersensitivity to any device materials. In cases of bilateral foot drop, use is contraindicated unless a model suitable for bilateral use (e.g., Comfit AFO Xtra Strong) is selected and fitted by a qualified healthcare professional.

Accessories

The device shall be used with the following accessory: Comfit AFO Classic Pad Kit REF: E8061PS03, E8061PS05. No other accessories are required. Use of non-approved accessories may affect performance and safety.

Warnings

-  CAUTION: The device must only be fitted by a qualified healthcare professional.
-  CAUTION: The device must not be modified by heating, drilling, or bending, as this may compromise its structural integrity.
-  CAUTION: Do not use the device if cracks, fractures, or deformations are visible. Use of a damaged device may increase the risk of falls or further injury.
-  CAUTION: Discontinue use immediately if pain, numbness, swelling, or skin breakdown occurs.
-  CAUTION: The device is not designed for running, jumping, or high-impact activities; such use may cause device failure and patient injury.
-  CAUTION: Driving or operating machinery while wearing the device should only be done after consulting a healthcare professional, as the device may alter ankle mobility and reaction time.

Precautions

- Inspect the skin regularly, especially during the first weeks of use, to detect redness or irritation.
- Always wear the device with suitable footwear providing firm heel support; sandals or clogs are not appropriate. Unsuitable shoes may compromise stability and lead to falls.
- Ensure socks or similar protection are worn under the orthosis to reduce risk of skin irritation.
- Check straps, pads and fastenings regularly; replace worn components promptly. Loose or worn components may reduce support and cause instability.
- Ensure proper fit and secure fastening before walking to avoid instability.
- Patients with reduced sensitivity in the lower limb should be monitored closely for skin damage. Pressure marks may not be noticed until skin breakdown occurs.

Fitting & Adjustment (for professionals)

Comfit AFO is available in two sizes: Small, Large.

For the best fit in the shoe, the footplate may need to be adjusted in length and width.

 CAUTION: Adjustment of the device must only be performed by qualified healthcare professionals (e.g. orthotists).

 CAUTION: Patients must not attempt to adjust or modify the device themselves.

 CAUTION: Incorrect adjustment can compromise the safety and function of the orthosis and may lead to fracture, delamination, or breakage.

Adjusting the footplate

Place an insole from the shoe on top of the footplate and position it so that the lateral strut is:

- Behind the base of the fifth metatarsal.
- In front of the lateral malleolus. **See image A.**

When the foot is positioned correctly, use the insole as a template. **See image B.**

Cut or grind the footplate using tin snips or a suitable grinding machine. Always grind in the longitudinal direction of the footplate – never across the blade. To facilitate insertion of the orthosis into the shoe, the footplate may be shortened by approximately 10 mm at the front. **See image C and D.**

After adjustment, smooth the edges with fine sandpaper or wet sanding so that no sharp edges remain. This reduces the risk of damage to the skin and/or the shoe. **See image E.**

Ensure that the orthosis is pulled all the way back against the heel cap of the shoe. **See image F.**

Check rotation in the shoe

Depending on the shape of the shoe, the orthosis may rotate inward or outward.

Therefore, adjust the footplate carefully so that the strut remains stable and does not twist inside the shoe.

Important!

Never grind or adjust the lateral strut or the area where it is attached to the footplate.

This may impair the function and weaken the orthosis, increasing the risk of cracking, delamination, or breakage.

Patient Guidance (to be provided by fitter)

The fitter is responsible for ensuring that the patient both receives and understands the following instructions before using the orthosis.

Putting on the orthosis

1. Place the orthosis in a suitable shoe. Position the footplate of the orthosis underneath the provided insole.
2. Ensure that the orthosis is pushed all the way back against the heel cup of the shoe.
3. Open the shoe properly and insert the foot. The calf section of the orthosis should rest against the calf.
4. Place the strap with padding against the skin and fasten it using the hook-and-loop closure attached on both sides of the calf section.
5. Tie or fasten the shoe securely.

Taking off the orthosis

Detach the calf strap from the orthosis. *Tip: you may leave one side of the strap attached to the orthosis when removing it to make application easier next time.*

Open the shoe properly and remove the foot from the shoe while leaving the orthosis inside the shoe.

Protection against chafing

Always wear a sock higher than the orthosis to reduce the risk of chafing and skin irritation.

Check the skin regularly for any signs of chafing, irritation, or pressure marks.

Be attentive

Inspect the orthosis daily for cracks or abnormal wear.

If you experience pain, swelling, numbness, or chafing, stop using the orthosis immediately and contact your orthotist or other healthcare professional before resuming use.

Cleaning and Maintenance

Clean the strut, padding and straps with mild soap and water. Do not machine wash, dry clean, or use solvents. Allow the device to air dry away from direct heat or sunlight.

Inspect the device daily for cracks, wear or loose parts. Replace worn padding and Velcro straps promptly. If any defects are observed, discontinue use and contact the prescribing clinic.

Functional Life

The expected functional life of the Comfit AFO is 12–24 months under normal use. Actual functional life will depend on frequency of use, patient activity level, and proper maintenance. The device must be replaced immediately if cracks, deformation, or loss of function occur.

Storage & Transport

The device should be stored in a clean, dry place, protected from direct sunlight, heat sources, humidity and chemicals. After cleaning, allow the device to air dry at room temperature. Do not expose the device to mechanical loads, such as heavy objects placed on top of it, which may cause deformation. During transport, ensure the device is packaged and handled to prevent mechanical damage.

Disposal

The device contains no hazardous substances. At the end of its service life, it must be disposed of in accordance with local and national regulations in the country of use. This applies to patients, clinics and distributors.

Warranty

Embreis has a 12 months warranty covers manufacturing defects only. The warranty is valid provided the device has been fitted and used according to the instructions for use. The warranty does not cover damage resulting from misuse, modification, or overloading. For warranty claims, contact Embreis Customer Support and provide the batch number together with a completed return form.

Serious Incidents

Any serious incident that has occurred in relation to this device shall be reported to Embreis AB and to the competent authority of the Member State in which the user and/or patient is established.

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Symbols

Symbol	Meaning
	Catalogue number – Identifies the manufacturer's catalogue number.
	UDI carrier – Contains the Unique Device Identifier.
	Batch code – Indicates the manufacturer's batch code.
	Medical device – Identifies the product as a medical device.
	Manufacturer – Identifies the manufacturer of the product, accompanied by name and address.
	Date of manufacture – Indicates the date when the device was manufactured.
	Consult instructions for use – Indicates the need to consult the instructions for use.
	Single patient, multiple use.
	Warning – Indicates that caution is necessary.
	CE marking – Indicates conformity with the Medical Device Regulation (EU) 2017/745.

Document Control

IFU ID: P01-4.2

Revision: 2

Manufacturer

Embreis AB
Tumstocksvägen 11B
S-187 66 Täby
Sweden

Contact

Tel. + 46 8 410 621 00
info@embreis.com
www.embreis.com

embreis

Embreis AB | Tumstocksvägen 11B | S-187 66 Täby Sweden

Tfn + 46 8 410 621 00 | info@embreis.com

www.embreis.com