## Patella Tracking Orthosis (PTO) Knee Brace




PTO-KNIESCHENE  
SUPPORTO PER GINOCCHIO PER L'ORTOSI PATELLARE (PTO)  
ORTHESE DE GENOU POUR ALIGNEMENT ROTULIEN (PTO)  
RODILLERA PARA ORTOSIS DE RÓTULA (PTO)

© (U.S. Patented) Breg, Inc.  
Breg, Inc. (800) 321-0607  
Vista, CA 92081 U.S.A.  
www.breg.com  
Telephone (800) 321-0607  
Breg Healthcare Inc. (760) 599-3000  
Breg Healthcare Inc. (760) 329-7234  
Tel/Fax (760) 329-7234  
Germany  
KNU GEBRUEK 011

**WARNING**  
FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.  
CAUTION: FOR SINGLE PATIENT USE ONLY.  
CAUTION: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.  
WARNING: CAREFULLY READ USE INSTRUCTIONS AND WARNINGS PRIOR TO OPERATION.  
WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER USE AND MAINTENANCE OF THIS DEVICE ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM.

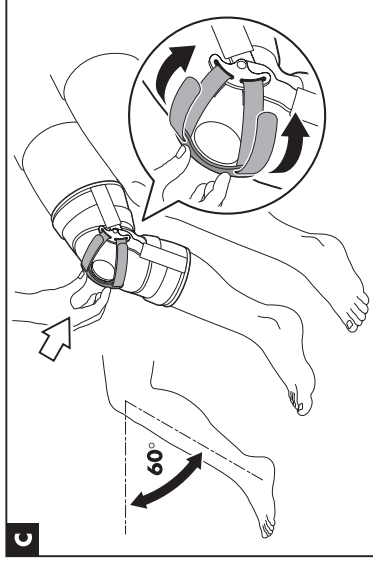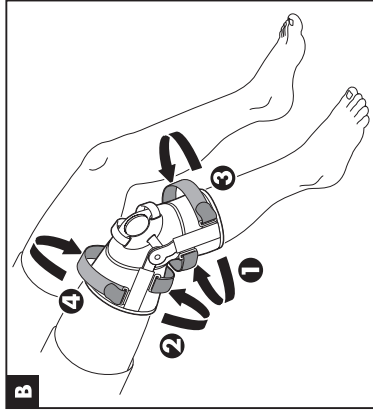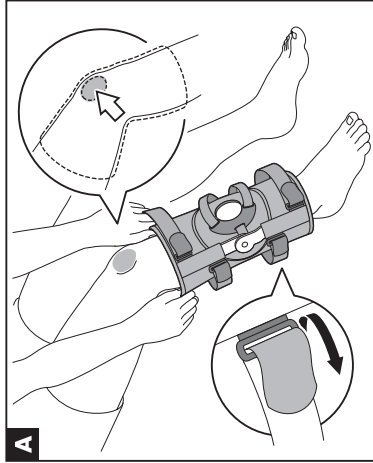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**BRACE APPLICATION**  
Unfasten and loosen all straps and fasten hook/loop tabs back against the strap as shown. This will facilitate pulling the brace on. While seated, pull on brace. The front hole must be positioned directly over the kneecap (patella), as shown. The plastic pressure plate should be positioned on the outside of the knee.

**TIGHTEN FOUR MAIN STRAPS**  
1st: lower rear; 2nd: upper rear;  
3rd: lower front; 4th: upper front.

**TIGHTENING PATELLA BUTTRESS STRAPS**  
Bend the knee to 60°. While pushing down on the pressure plate, tighten the buttress straps around the kneecap. These straps should be snug.

NOTE: over-tightening will cause the medial (inside) hinge to pull forward.

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