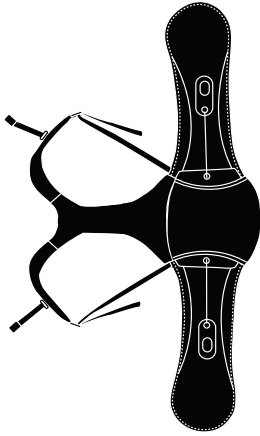
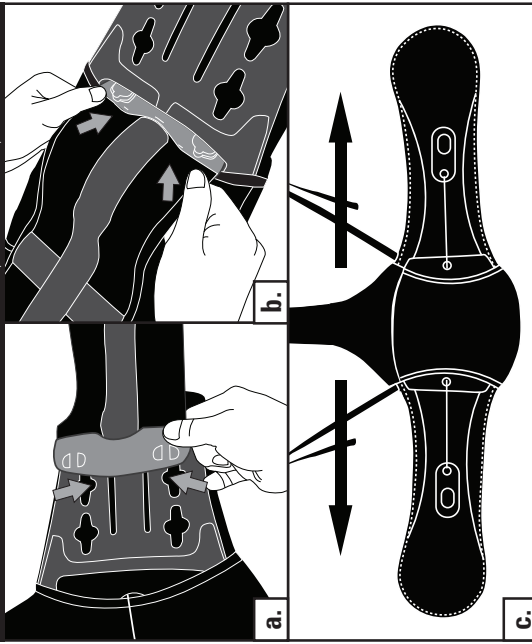
# SUMMIT™ 456

## SUMMIT™ ADJUSTABLE 456 TLSO

BELT SIZING FOR SUMMIT™ 456 TLSO (ADJUSTABLE ONLY)

### ENGLISH

#### BELT SIZING

- Fit male tabs of belt through proper sizing adjustment holes in sides of back panel.
- For smaller sizes, slide side panel through sizing adjustment slot and then fit tabs through proper adjustment holes from the under side.
- Once proper size is achieved, pull taught to lock tabs in place.

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\* The SUMMIT™ sizing follows closely to standard pants sizing. Size ranges are only guidelines. Your experience and personal assessment could make the difference in choosing the correct size. For the SUMMIT™ 456 add 2 inches for a better fit.

This product may be substantially modified, by an individual with expertise.

For more information, please visit  
[www.aspenmp.com/index.php/about-us/product-downloads/](http://www.aspenmp.com/index.php/about-us/product-downloads/).

**ENGLISH**  
**BACK PANEL**  
Custom form the malleable aluminum support in the back panel to properly fit patient anatomy.

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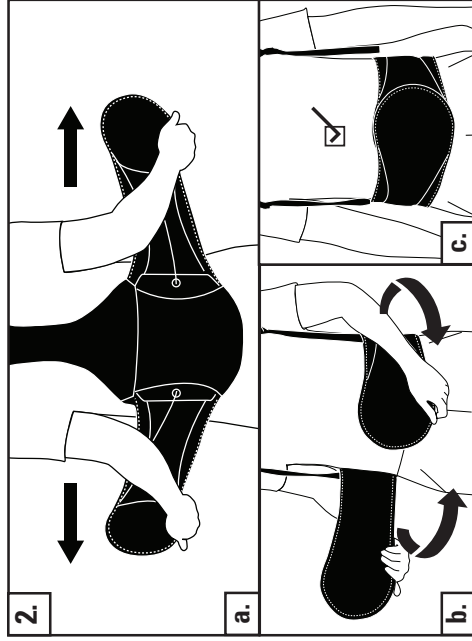
**ENGLISH**  
**PLACEMENT**  
a. Apply the fully stretched brace making sure the back panel is centered and low on the back.  
b. Overlap the ends of the brace so the hook material grips the opposite side loop as tightly as possible.  
c. Ends may be angled to accommodate for different anatomies.

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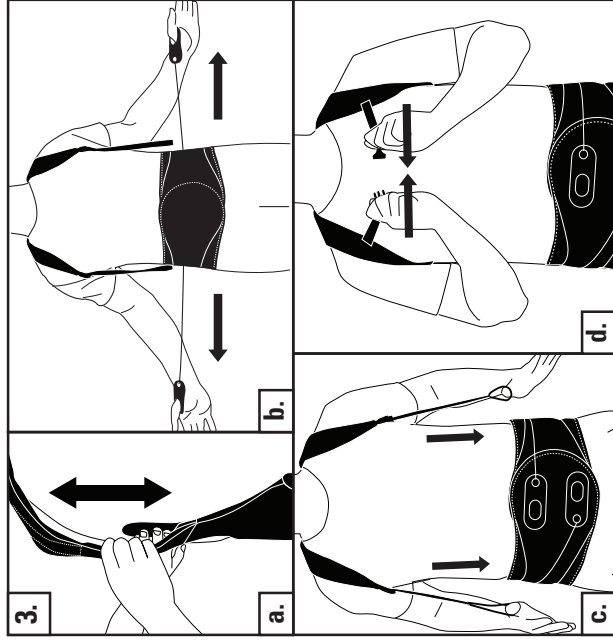
**ENGLISH**  
**TIGHTEN AND ADJUST**  
a. Adjust the vertical height of the posterior adjustment strap so that the yoke rests at C7, or 2" below the shirt collar.  
b. Slip each thumb into the pull tabs. Extend the tabs out and away from the brace. When desired compression is achieved, place the hook side of the pull tabs down onto the loop material to secure.  
c. Pull and tighten the shoulder straps so they fit wide on the shoulders.  
d. Fasten and tighten the chest buckle to secure the shoulder straps.

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**LIMITED WARRANTY**  
Aspen Medical Products, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products in the event of breach of warranty shall be to repair or replace the defective product or parts.  
Aspen Medical Products shall have no obligation under this limited warranty in the event:  
(a) The product is altered, repaired, modified, or used for purposes not intended by Aspen Medical Products or through its authorized channels of distribution;  
(b) The product is altered, repaired, modified, or used for purposes not intended by Aspen Medical Products; or  
(c) Any parts not supplied by Aspen Medical Products are inserted into the product; or  
(d) The product is not used in accordance with the Aspen Medical Products, Instructions for Use.

**READ INSTRUCTIONS BEFORE USE**  
Proper training in the use of this device should take place before use. These instructions are guidelines only and are not offered as medical recommendations.  
THIS PRODUCT IS INTENDED FOR APPLICATION BY LICENSED HEALTH CARE PRACTITIONERS AS DIRECTED BY A PHYSICIAN OR OTHER QUALIFIED MEDICAL AUTHORITY. ASPEN MEDICAL PRODUCTS SPECIFICALLY DISCLAIMS ANY LIABILITY FOR INJURY OR DAMAGE BY REASON OF ANY APPLICATION OF THIS PRODUCT. PLEASE SEE THE ACCOMPANYING LIMITED WARRANTY FOR FURTHER INFORMATION IN THIS REGARD.  
SEE THE ACCOMPANYING LIMITED WARRANTY FOR FURTHER INFORMATION IN THIS REGARD.

**READ INSTRUCTIONS BEFORE USE**  
Proper training in the use of this device should take place before use. These instructions are guidelines only and are not offered as medical recommendations.  
THIS PRODUCT IS INTENDED FOR APPLICATION BY LICENSED HEALTH CARE PRACTITIONERS AS DIRECTED BY A PHYSICIAN OR OTHER QUALIFIED MEDICAL AUTHORITY. ASPEN MEDICAL PRODUCTS SPECIFICALLY DISCLAIMS ANY LIABILITY FOR INJURY OR DAMAGE BY REASON OF ANY APPLICATION OF THIS PRODUCT. PLEASE SEE THE ACCOMPANYING LIMITED WARRANTY FOR FURTHER INFORMATION IN THIS REGARD.  
SEE THE ACCOMPANYING LIMITED WARRANTY FOR FURTHER INFORMATION IN THIS REGARD.

**MADE IN CHINA**  
**PATENT PENDING**