
1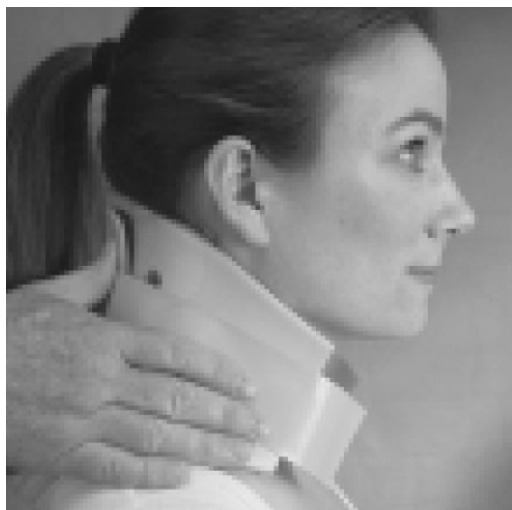

Please select the correct size as the collar comes in different heights and circumferences. Apply collar piece "Back" to the back of patient's neck.

Height – measure bottom of chin to top of sternum.

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2

Apply front half of collar with chin secured in recess. Center collar to ensure neutral alignment. Front half overlaps back half.

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3

With the hook and loop fasteners tighten collar with bi-lateral adjustments. Center collar with arrow pointing up.

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## Cervical Collar with Open Trachea Instructions for Use

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## WARNINGS

**WARNING:** Carefully read fitting instructions and warnings prior to use. To ensure proper performance of the brace, follow all instructions.

**WARNING:** If you experience, or have an increase in pain, swelling, skin irritation, or any adverse reactions while using this product, immediately consult your medical professional.

**WARNING:** This device will not prevent or reduce all injuries. Proper rehabilitation and activity modification are also an essential part of a safe treatment program. Consult your licensed health care professional regarding safe and appropriate activity level while wearing this device.

**CAUTION:** Follow application and care process described in instructions.

**CAUTION:** For single patient use only.

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Breg, Inc.