# Recover Knee Brace

## FITTING INSTRUCTIONS

### ANLEGEANWEISUNG

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### MISE EN PLACE

### INSTRUCCIONES DE AJUSTE

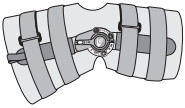
Breg, Inc.  
10000 Sycamore Ave East  
Carlsbad, CA 92010 U.S.A.  
P: 800-231-9607  
F: 760-439-8800  
www.breg.com  
©2013 Breg, Inc.  
AVL1.0093248V.A 0313


EG REP  
Eurasian Conformity  
AVS: 0911  
Sollmanstr. 4  
63429 Hanau  
Germany

### Recover Knee Brace


Airmesh • Luftdurchlässiges Netzmaterialel •  
Rete traspirante • Mallas aéreas • Tejido de malla


Long  
Lungo  
Longue  
Larga

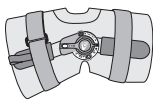Short  
Kurz  
Corta  
Corta  
Corta

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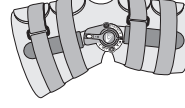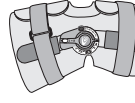Long  
Lungo  
Longue  
Larga

## WARNINGS

**WARNING:** GENERAL READ THE FITTING INSTRUCTIONS AND WARNINGS PRIOR TO USE TO ENSURE PROPER PERFORMANCE OF THE BRACE, FOLLOW ALL INSTRUCTIONS.

**WARNING:** DO NOT REMOVE THE RECOVER KNEE BRACE UNLESS INSTRUCTED BY YOUR MEDICAL TREATMENT PROFESSIONAL. DO NOT CHANGE RANGE OF MOTION HINGE SETTINGS WITHOUT SUPERVISION BY A MEDICAL PROFESSIONAL.

**WARNING:** IF YOU EXPERIENCE INCREASED PAIN, SWELLING, SKIN IRRITATION, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.

**WARNING:** THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM. CONSULT YOUR LICENSED HEALTH CARE PROFESSIONAL REGARDING SAFE AND APPROPRIATE ACTIVITY LEVEL WHILE WEARING THIS DEVICE.

**CAUTION:** FOLLOW REMOVAL AND CLEANING PROCESS DESCRIBED IN INSTRUCTIONS.

**CAUTION:** FOR SINGLE PATIENT USE ONLY

**CAUTION:** FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.

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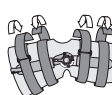
### Application Instructions for Knee Braces:

The wrap-on brace is applied with the closures in the front of the leg and the kneecap cutout centered over the knee. Secure the closures firmly.

The straps should be tightened closely, starting with the two straps closest to the knee.

### Tips on Use and Care of Your Knee Brace:

After exposure to salt water or dirt, thoroughly rinse the brace with fresh water and allow to air dry. A dry lubricant, such as teflon spray, may be used on the hinge occasionally. Do not use wet lubricants. The hinge assemblies may be removed to allow easier washing of the sleeve and straps.

Medical Professional - The side bars may be bent to accommodate individual legs. Bend in equal amounts above and below the hinge. Periodically inspect the sleeve, straps and hinges for excessive wear. Be sure all screws are tight and straps are in place.

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