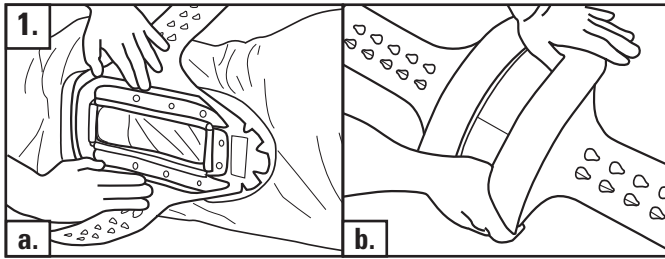
Aspen Medical Products, Inc.
 6481 Oak Canyon, Irvine, CA 92618
 800-295-2776 • 949-681-0200
 www.aspenmp.com

ENGLISH
SIZE SELECTION

a. Match the patient's height and girth to the correct size LSO. Hold the back panel up to the patient and note the location and amount of lordotic curve.
 b. Use the Aspen bending tool or the edge of a table to bend the aluminum rods in the back panel as needed. Test and adjust as needed.

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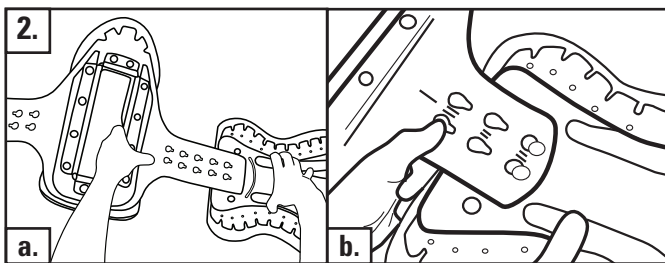
ENGLISH
ASSEMBLY

a. Average the patient's waist and lower rib circumference. Slide the ends of the back panel into the curved slots so the end is "hidden" under the side panels.
 b. Using the average circumference, seat the rivets in the appropriate SureSlot openings and pull firmly to snap in place.

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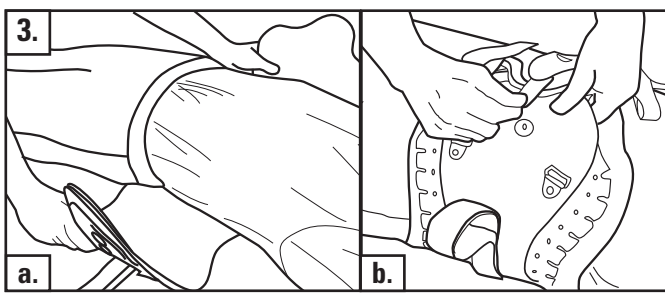
ENGLISH
POSITIONING & CLOSURE

a. Position the patient so the back panel is centered and properly located on the back. Ambulatory patients may hold the brace while it is being tightened.
 b. Attach the black Velcro start tab to the Velcro dot on the opposite side panel. If the Velcro start tab is too long or too short, the brace should be re-adjusted.

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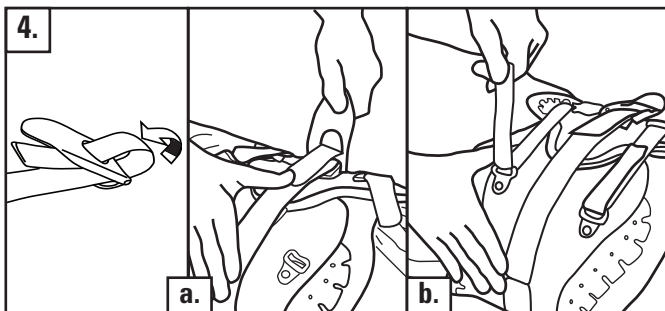
ENGLISH
TIGHTENING

a. Lace the 1.5" wide center strap through the LeverLatch. Securely attach the loop Velcro and push down as you move the LeverLatch to the patient's right side to tighten the brace.
 b. Attach and tighten the upper and lower straps to take up the slack. Repeat Lever tightening and upper/lower strap tightening as needed.

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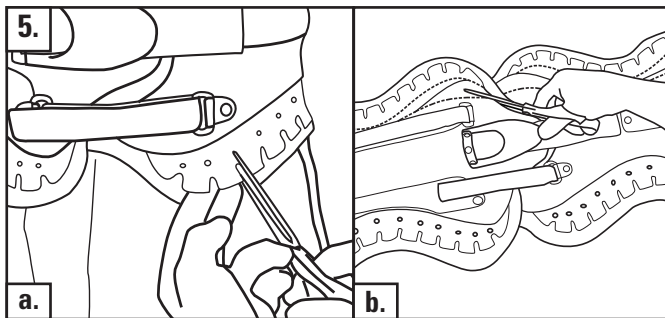
ENGLISH
CUSTOMIZATION

a. If the plastic edges are uncomfortable, cut from the bottom of the flex slot to the adjacent hole to reduce pressure.
b. The outer support layer may also be trimmed to accommodate anatomies. Drill out the aluminum rivets that hold the strap and chafe tab, and re-rivet them in the pre-drilled holes. Then trim as needed.

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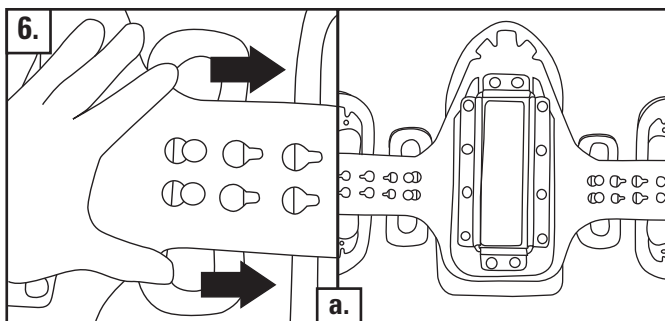
ENGLISH
FILLER PAD

Filler pads are sold separately.
a. If there is a space in the belt, snap the filler pads in place and slide to the right to lock.

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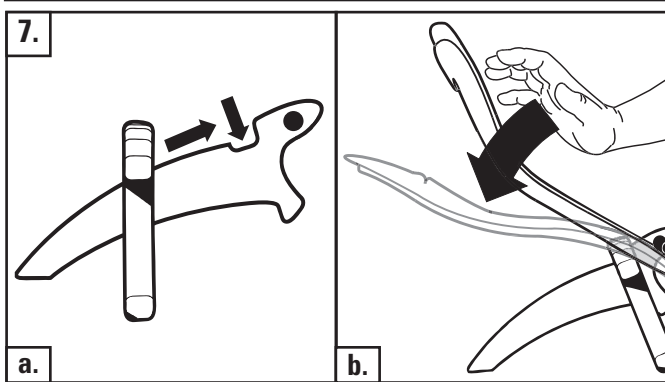
ENGLISH
BENDING TOOL

Bending tools are sold separately.
a. Slide the foot into the hole in the fulcrum component and lock securely. Position the retaining bar through the opening in the back panel.
b. Press on the brace over the foot until proper curvature is achieved.

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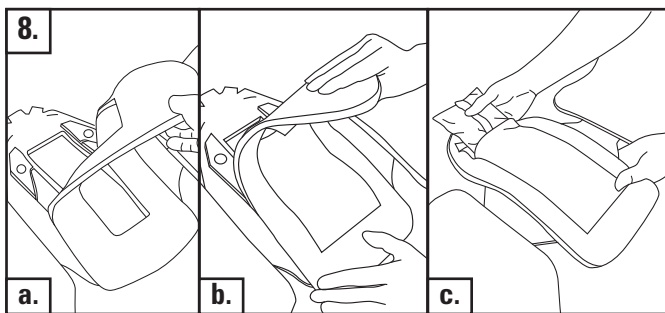
ENGLISH
COLD THERAPY

Cold Therapy packs are sold separately.
a. Remove the back panel foam.
b. Attach the new foam pad onto the back panel.
c. Insert the frozen cold therapy pack into the pouch on the new foam pad.

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LIMITED WARRANTY
Aspen Medical Products, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products in the event of breach of warranty shall be to repair or replace the defective product or part(s).
Aspen Medical Products shall have no obligation under this limited warranty in the event:
(a) The product was not purchased from Aspen Medical Products or through its authorized channels of distribution;
(b) The product is altered;
(c) Any parts not supplied by Aspen Medical Products are inserted into the product; or
(d) The product is not used in accordance with the Aspen Medical Products, Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS.
This warranty gives you specific legal rights and you may have other rights which vary from state to state.
Warning: To ensure proper use, please review all material.
A listing of authorized distributors in your area is available upon request.

READ INSTRUCTIONS BEFORE USE. Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations.

THIS PRODUCT IS INTENDED FOR APPLICATION BY LICENSED HEALTH CARE PRACTITIONERS AS DIRECTED BY A PHYSICIAN OR OTHER QUALIFIED MEDICAL AUTHORITY. PERMANENT INJURY MAY RESULT FROM USE OTHER THAN AS DIRECTED BY A PHYSICIAN OR OTHER MEDICAL AUTHORITY. ASPEN MEDICAL PRODUCTS SPECIFICALLY DISCLAIMS ANY OBLIGATION OR LIABILITY FOR INJURY OR DAMAGE BY REASON OF ANY APPLICATION OF THIS PRODUCT. PLEASE SEE THE ACCOMPANYING LIMITED WARRANTY FOR FURTHER INFORMATION IN THIS REGARD.