

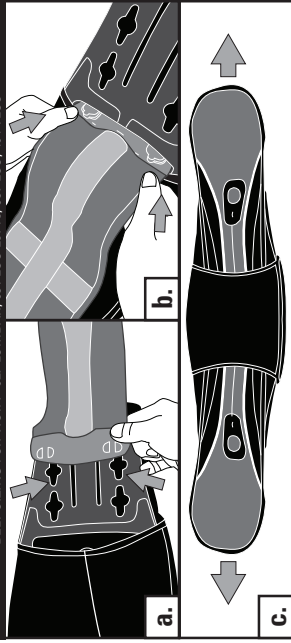
# Vista® LOWER SPINE

BELT SIZING FOR VISTA® 627 LUMBAR, 631 LSO LoPro, 637 LSO, 464 TLSO


	EXTRA SMALL	SMALL	MEDIUM	LARGE	EXTRA LARGE
<b>Vista® 627 LUMBAR ORTHOSIS</b>	26 - 31 inches 66 - 78 cm	30 - 37 inches 76 - 94 cm	36 - 43 inches 91 - 109 cm	42 - 49 inches 107 - 124 cm	48 - 55 inches 122 - 140 cm
<b>Vista® 631 LSO LoPro</b>					54 - 60 inches 137 - 152 cm
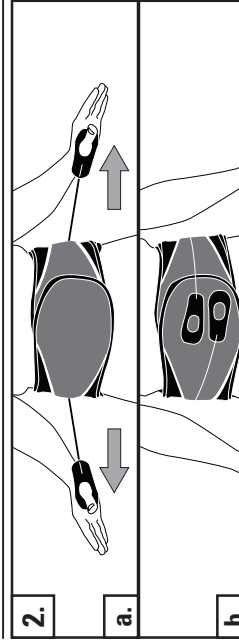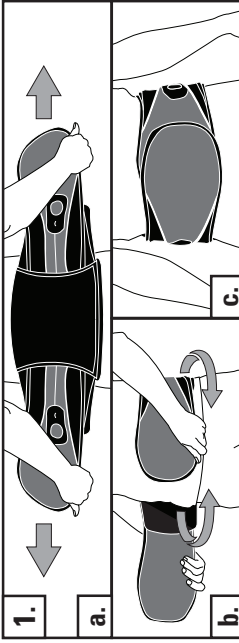
## Vista® 627 VISTA® 631 VISTA® 637 LUMBAR ORTHOSIS LUMBSACRAL ORTHOSIS LUMBSACRAL ORTHOSIS

### VISTA® Lower Spine

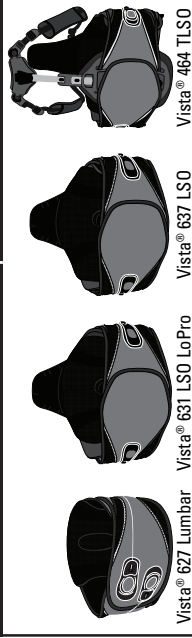Vista® 627 Lumbar Vista® 631 LSO LoPro Vista® 637 LSO Vista® 464 TLSO

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\* The Vista® 627 sizing follows closely to standard pants sizing. Size ranges are only guidelines. Your experiences and personal assessment could make the difference in choosing the correct size. For the Vista® 631, 637 and 464 add 2 inches for a better fit.

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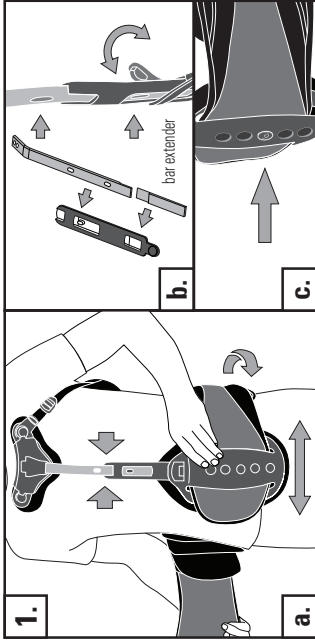
**ENGLISH**

**PLACEMENT & ADJUST**

a. Center back panel as low as possible. Slide abdominal panel and sternal pad sideways as needed to ensure brace and panel are at patient's midline.  
 b. For shorter patients, remove bar extender from the abdominal panel stretch. Pry button back and slide aluminum strut so that the button locks into the appropriate hole.  
 c. To further adjust the sternal pad on the abdominal panel, press the button with tear logo inward and raise or lower the sternal pad as needed.

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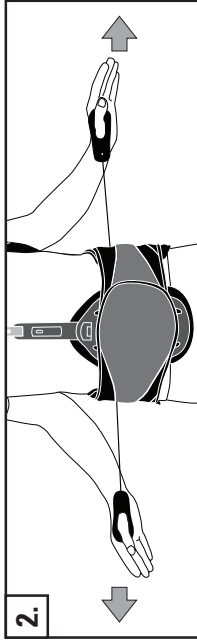
**ENGLISH**

**TIGHTEN**

Pull strongly away from body. Tighten more than you need and then loosen to achieve a proper fit. Pull left tab to tighten lower, pull right to tighten upper as needed.

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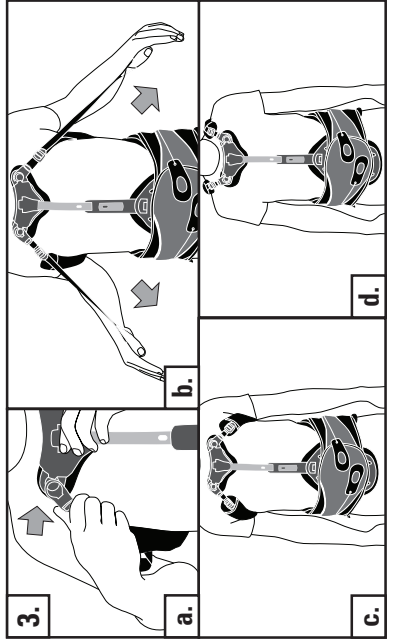
**ENGLISH**

**STRAPS**

a. Attach underarm control straps to sternal pad.  
 b. Pull underarm control straps to tighten as needed.  
 c. Underarm control straps and sternal pad should fit securely to patient's anatomy.  
 d. Under arm control straps can be positioned to fit over the shoulder. Note: If needed, the support strut can be bent to ensure a proper fit.

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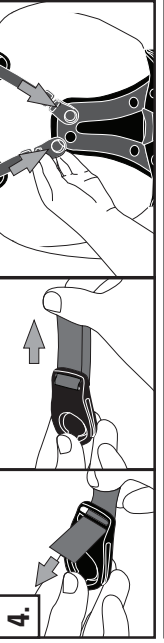
**ENGLISH**

**OVER SHOULDER OPTION**

If buckles are needed, add a buckle to straps ends and secure buckle. Attach buckles to the top of the TLSO back panel.

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**ENGLISH**

**ANTERIOR PANEL EXTENSION**

Position anterior panel extension pads on brace for desired coverage. Panels must overlap to encircle the trunk.

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**LIMITED WARRANTY**  
 Aspen Medical Products, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products in the event of breach of warranty shall be to repair or replace the defective product or parts.  
 Aspen Medical Products shall have no obligation under this limited warranty in the event:  
 (a) The product is altered, repaired, modified, or otherwise tampered with by anyone other than an authorized distributor;  
 (b) The product is altered, repaired, modified, or otherwise tampered with by anyone other than an authorized distributor;  
 (c) Any parts not supplied by Aspen Medical Products are inserted into the product; or  
 (d) The product is not used in accordance with the Aspen Medical Products, Instructions for Use.

**READ INSTRUCTIONS BEFORE USE** Proper fitting in the use of this device should take place before it is applied. These directions are guidelines only and are not intended as medical recommendations.  
 THIS PRODUCT IS INTENDED FOR APPLICATION BY LICENSED HEALTH CARE PRACTITIONERS AS DIRECTED BY A PHYSICIAN OR OTHER QUALIFIED AUTHORITY. PERMANENT INJURY MAY RESULT FROM USE OTHER THAN AS DIRECTED. THE USER ASSUMES ALL LIABILITY FOR INJURY OR DAMAGE BY REASON OF ANY APPLICATION OF THIS PRODUCT. PLEASE SEE THE ACCOMPANYING LIMITED WARRANTY FOR FURTHER INFORMATION IN THIS REGARD.  
 PATENT PENDING  
 MADE IN CHINA