NEURO-LUX® II SPORLASTIC

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vest. The arm element should be worn directly against the skin.

IMPORTANT

The product is to be used by one patient only. Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

QUALITY MANAGEMENT SYSTEM

All SPORLASTIC GmbH products undergo product testing as part of our Quality Management System.

However, should you have any complaints about our product, please get in touch with your stockist or directly with us.

CARE

We recommend that you wash NEURO-LUX® II gently in lukewarm water using liquid detergent and air-dry it. Do not place it on radiators.

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CONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

– Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.

Utilisation requires a prior check-up by your doctor, therapist or technician for:

– Impaired circulation or lymphatic soft tissue swelling.

– Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far no consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

To date, there have been no reports of serious general side effects with proper use and fitting.

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Wszystkie wyroby SPORLASTIC GmbH podlegają kontroli w ramach naszego systemu zarządzania jakością.