PostOp Knee Brace

POSTOPERATIVE KNIESCHIENE
SUPPORTO POST-OP DEL GINOCCHIO
ORTHESE DE GENOU POST-OPERATOIRE
RODILLERA POSTOPERATORIA

PostOp Lite Knee Brace

LEICHTE POSTOPERATIVE KNIESCHIENE
SUPPORTO LEGGERO POST-OP DEL GINOCCHIO
ORTHESE DE GENOU POST-OPERATOIRE LEGERE
RODILLERA LIGERA POSTOPERATORIA

PostOp Knee Brace with shells

POSTOPERATIVE KNIESCHIENE MIT SCHALEN
SUPPORTO CON INVOLUCRI POST-OP DEL GINOCCHIO
ORTHESE DE GENOU POST-OPERATOIRE A ENVELOPPES
RODILLERA POSTOPERATORIA CON ARMAZONES

PostOp QuickFit Knee Brace

POSTOPERATIVE QUICKFIT-KNIESCHIENE
SUPPORTO DEL GINOCCHIO POST-OP QUICKFIT
ORTHESE POST-OPERATOIRE QUICKFIT
RODILLERA POSTOPERATORIA QUICKFIT

MOSS GmbH
S-41071 Grieben, 41
52301, Germany

BREG an Orthofix Company
BREG Inc. 92081 U.S.A.
Telephone (800) 321-0607
(760) 599-3000
Telefax (800) 329-2734
AW-1-90032 REV A

W A R N I N G S

WARNING: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.
CAUTION: READ AND FOLLOW THE INSTRUCTIONS CAREFULLY.
CAUTION: IF YOU EXPERIENCE INCREASED PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR MEDICAL PROFESSIONAL.
WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM.

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1 Open the brace by releasing the straps and foam wrap and lying flat on the floor. Position the patient's leg (left leg illustrated) so that the knee is centered between the foam wrap pads.

2 Position and press into place the side bars. Reposition the straps slightly above the middle of the knee cap and toward the back of the knee on the sides. If necessary, the side bars may be removed from the foam wrap and repositioned.

3 Secure and fasten the 2 straps closest to the knee first. Note that the straps fasten in opposing directions.
4 Secure and fasten the remaining 2 straps also in opposing directions.

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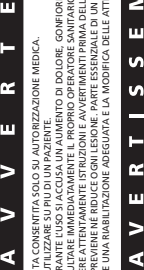
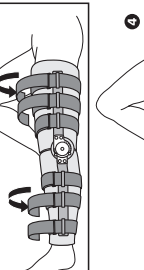
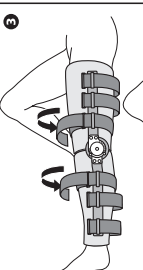
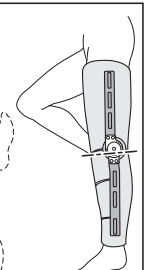
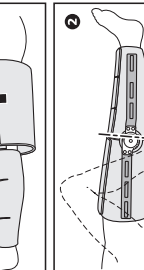
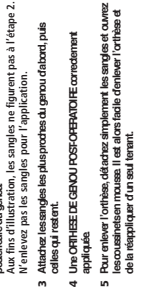
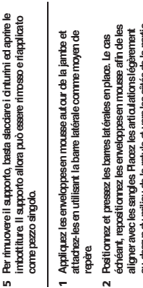
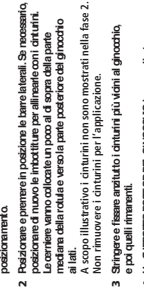
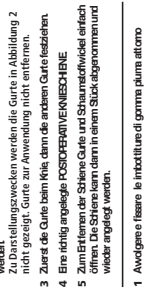
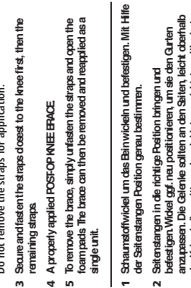
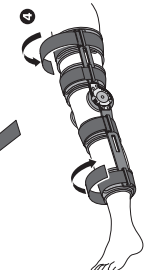
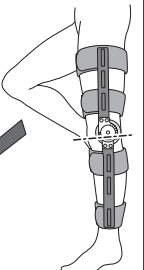
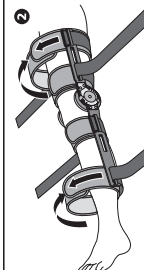
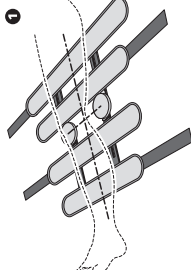
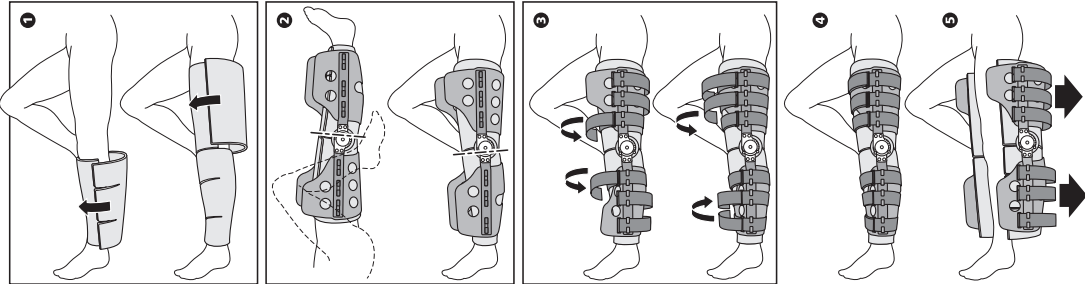
- 1 Wrap and fasten the foam wrap around the leg using the add-on as a guide for placement.
- 2 Position and press into place the side bars. Reposition the wrap as necessary. Then position the top and bottom bars on each side above the middle of the knee cap and toward the back of the knee on the sides.
- 3 For illustration purposes the straps are not shown in step 2. Do not remove the straps for application.
- 3 Secure and fasten the straps closest to the knee first, then the remaining straps.
- 4 A properly applied **FOCUS KNEE BRACE**.
- 5 To remove the brace, simply unfasten the straps and open the foam pads. The brace can then be removed and reappplied as a single unit.

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
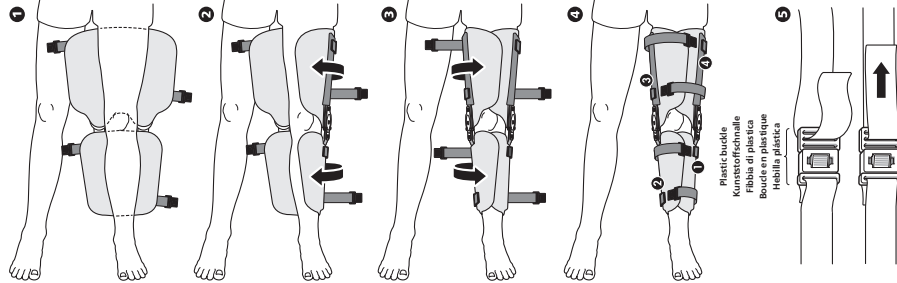
- 1 Open the topop-QuickFit brace and place it under the fully extended leg. The hinges should be at mid-patella level and mid-tibial level of the leg.
- 2 Fold foam wrap over from the medial or outside of the leg so that the hinges are on the inside of the knee. The hinges should be on the inside of the knee. The hinges should be on the inside of the knee. The hinges should be on the inside of the knee.
- 3 Tighten the straps by pulling the strap tight through the buckle and fastening back against itself by using the hook and loop closure.
- 4 Buckle the straps starting with the upper calf (1), the lower calf (2), the lower thigh (3), and last the upper thigh (4).
- 5 Tighten the straps by pulling the strap tight through the buckle and fastening back against itself by using the hook and loop closure.

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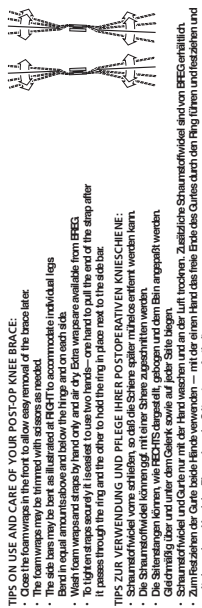- TIPS ON USE AND CARE OF YOUR POSTOP KNEE BRACE:**
- Open the brace in front of you to allow easy removal of the brace later.
 - The side bars may be bent as illustrated at RIGHT to accommodate individual legs.
 - Wash in equal warm/ambrose and below the hinge and on each side.
 - Do not remove the straps for application.
 - It is recommended to wear the brace for 24 hours during the first 48 hours after surgery.
 - It is recommended to wear the brace for 24 hours during the first 48 hours after surgery.
 - It is recommended to wear the brace for 24 hours during the first 48 hours after surgery.

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