

## Washing Instructions

- The Heal –Up Heel Relief Boot can be machine washed at 40°C WOOL CYCLE using a mild detergent. **DO NOT USE FABRIC CONDITIONER, DO NOT USE BLEACH**
- Ensure all hook and loop straps are closed and no hook is exposed. This avoids fabric particles damaging the hook closures
- After removing from machine, open all straps and air-dry on a Clothes Horse or Clothes Airing. Keep away from any heat sources
- When completely dry ensure that Heal- Up filling is evenly distributed throughout the boot
- Do not iron, Do not dry clean
- Check hook closures and clear any build up of fabric or lint as this can affect the ability to fasten straps securely and the overall performance of the Heal-Up



Treated with Agion® and Silvadur™ antimicrobial to inhibit the growth of bacteria.



Scan QR code for fitting instructions

[www.heal-up.co.uk](http://www.heal-up.co.uk)