# Comfit AFO<sup>®</sup>

INSTRUCTIONS FOR USE / BRUKSANVISNING / BRUGSANVISNING  
INSTRUKSJONER FOR BRUK / KÄYTTÖOHJEET / HANDBUCH

EN - Table of Contents.....	sid 4
SE - Innehållsförteckning.....	sid 10
DK - Inholdsfortegnelsen.....	sid 16
NO - Innholdsfortegnelse.....	sid 22
FI - Sisällys.....	sid 28
DE - Handbuch.....	sid 34

# Table of Contents

1. Introduction
2. General safety advice
3. Area of use
4. Working life
5. Adjustment/fitting
6. Good advice for walking and use
7. Cleaning
8. Maintenance
9. Storage
10. Disposal
11. Warranty
12. Key to symbols

# 1. Introduction

This aid offers the opportunity for increased functionality and quality of life for those suffering from foot drop. Comfit AFO has been developed over the course of many years to ensure function and quality. The lightweight, convenient design with the strut tight to the leg provides the user with maximum comfort. By continuously refining our manufacturing process, we strive to develop a product of the highest possible quality. Embreis AB can therefore guarantee that this a durable and top-class product.

Comfit AFO meets the requirements of the Medical Devices Directive for Class 1 products (MDD 93/42EEC).

Embreis AB

Manufacturer:

Embreis AB  
Tumstocksvägen 11A  
S-187 66 Täby  
Sweden

Tel. + 46 8 410 621 00  
[info@embreis.com](mailto:info@embreis.com)  
[www.embreis.com](http://www.embreis.com)

## 2. General safety advice

Please note the following:

- This product must not be exposed to extreme loads.
- Do not operate a motor vehicle while wearing this product without first discussing the matter with your doctor or an orthopaedic engineer.
- Make sure that you are supported by the complete foot blade when rising from a sitting position and when using stairs.
- It is preferable to avoid deep knee bends as this may subject the product to extreme stress and increases the risk of breaking.
- Use a shoe that sits firmly on the foot. Do not wear sandals or clogs.
- Jogging, running or jumping may subject the product to extreme stress and increases the risk of breaking.
- The product is manufactured in carbon fibre and cannot be reshaped using heat or the like. It should always be tested by a trained orthopaedic technician.
- Do not make any holes in the product as this may damage the carbon fibre and weaken the product.

## 3. Area of use

- This product supports the foot and helps patients suffering from foot drop.
- The product counters the consequences of foot drop and is designed to as far as possible recreate a normal gait.
- The product is designed to maintain the foot in a lifted position (dorsal flexed) during the swing phase while walking and to facilitate a more normal gait.
- The product must not be used by patients with open leg wounds.
- The product must not be used by patients with pronounced foot deformities.
- The product should not be used by patients with moderate or severe spasticity.

## 4. Working life

The product expected working life is affected by correct handling and requires proper use, cleaning, storage and fitting, and that padding and Velcro are replaced as and when required. It is the responsibility of the user to have the product refurbished if used beyond the calculated working life. Factors that may affect the product's working life are extreme loads such as those caused by jumping, running, crouching, rising from a sitting position, using stairs, long-term use and badly adapted shoes. When walking, be careful to ensure the gentlest possible load on the product and when using stairs place weight on the entire foot not just one part of it.

## 5. Adjustment/fitting

Adjusting/fitting the foot drop strut to the patient must be carried out by a qualified health-care professional or orthopaedic technician.

### *Adjustment to the lower leg*

Padding and straps must be adjusted to fit the lower leg so that the patient is comfortable. Replacement of padding and straps must be carried out by a qualified healthcare professional or orthopaedic technician. If the user feels any discomfort or chaffing, further padding can be fitted in order to achieve maximum comfort. The inside of the bracket can also be padded to relieve pressure on the lateral malleoli, or alternatively the Wide model can be used.

### *Adjustment to the shoe*

The footplate must be adapted to the size of shoe, so that the strut can be easily inserted into the shoe. Always adjust the size of the front part of the foot blade. Sandals or clogs should be avoided as these do not sit sufficiently firmly on the foot and therefore cannot secure the position of the strut in relation to the foot. The strut can be advantageously placed under any loose insole.

## 6. Good advice for walking and use

### *Guidelines for the patient/user*

- Always choose a shoe that sits well on the foot.
- If the shoe has a loose insole or if you have an additional insole, it is a good idea to place the strut under the insole.
- Use a shoe with a suitable heel (1-2.5 cm).
- Try to move your foot/leg back and forth without lifting your knee more than normal (as you may have done before you had your foot drop brace).
- Try wearing a longer sock that goes high up the lower leg.

## 7. Cleaning

The foot drop strut, padding and strap should only be cleaned using a damp cloth and mild detergent.

The padding and strap should be cleaned regularly to remove visible stains and dirt. These can be cleaned with either a brush or a damp cloth.

## 8. Maintenance

Inspect your foot drop brace daily prior to use. In the event of any visible alterations or suspected faults, contact a qualified orthopaedic technician and have them inspect the device.

## 9. Storage

Your foot drop brace should be stored in a safe place where it will not be subjected to external forces or impacts. There is no temperature requirement although if the device is stored in a cold environment it should be allowed to acclimatise prior to use in the interests of comfort.

## 10. Disposal

There are no specific requirements regarding disposal.

## 11. Comfit AFO – Warranty Terms and Conditions

Embreis' Warranty Terms and Conditions cover manufacturing faults affecting the product Comfit AFO. The validity of the warranty is conditional on the device having been fitted and adjusted in accordance with the instructions included with the product. Patient instructions shall be explained and handed over to the patient.

In the event of a successful warranty claim, Embreis AB will replace the product with a Comfit AFO model of equal value. Replacement is limited to two orthoses for any patient within a one-year period from the first fitting.

To make a warranty claim for Comfit AFO, please contact Embreis Customer Support stating the batch number and then return the defective product together with the completed returns form. Returns forms are available at [www.embreis.com](http://www.embreis.com) and [www.comfit-af0.com](http://www.comfit-af0.com) or from Embreis Customer Support.

This warranty does not cover changes or damage arising from the use of tools, the application of heat or exaggerated bending of the wings (forwards and backwards).

This warranty does not cover fitting costs, alterations or other testing costs.

## 12. Key to symbols

The following symbols are used on labels and/or in this manual.

Manufacturer

Date of manufacture

Serial number

Warning – see attached information

User information

CE labelled and approved in accordance with Medical Devices Directive MDD 93/42EEC

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Embreis AB Tumstocksvägen 11A S-187 66 Täby Sweden