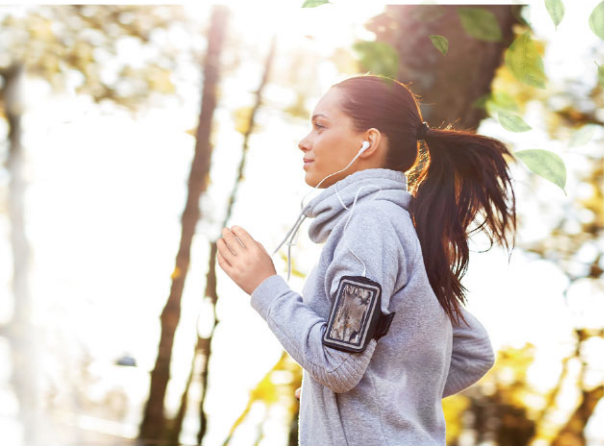
*Instructions for Use*

**Step Toward Tomorrow**

**REGAL PROSTHESIS LIMITED**

*An ISO 13485:2016 and ISO 9001:2015 certified company  
Your reliable partner*

## **Contents**

• EN	Instructions for Use	2
• DE	Bedienungsanleitung	8
• RU	Инструкция по применению	15
• TR	Kullanım Talimatları	22
• UK	Інструкції до використання	28
• ES	Instrucciones de uso	34
• PT	Instruções de utilização	40
• PL	Instrukcja użytkowania	46
• NL	Gebbruiksaanviesing	52
• ZH	使用说明	58
• ZH	使用說明	63
• KO	사용설명서	68
• HI	उपयोग के लिए नरिदेश	73
• HE	הוראות שימוש	79
• AR	تعليمات الإستخدام	84
• SV	Bruksanvisning	89
• HR	Upute za upotrebu	95
• DA	Brugsanvisning	101
• NO	Bruksanvisning	107
• FI	Käyttöohjeet	113
• FR	Mode d'emploi	119
• IT	Istruzioni per l'uso	126

## 1. Applications

- REGAL PROSTHESIS LIMITED Silicone Cosmetic Prostheses are used in providing aesthetic appearance to amputees so as to build up their confidence and to improve their social quality of life.

## 2. Indications and Patient Target Groups

- Model 101L, 102L, 102LS – For patients of above wrist amputation
- Model 101LL, 102LL, 102LSL – For patients of above wrist amputation (long sleeve).
- Model 103 – For patients of partial hand amputation
- Model 100 – For patients with either myo-electric or mechanical hands.
- Model 100L - For patients with either myo-electric or mechanical hands (long sleeve).
- Model 100PL – For pediatric patients with myo-electric hand.
- Model 200 series – For patients with finger amputation.
- Model HDSFHT, HDSFHT-ER, HDSF, HDSF-ER – For patients with partial foot amputation.
- Model TOE series – For patients with toe amputation
- Model BKSL – For patients who need covering the lose of partial calf muscle or covering lower limb prostheses respectively.
- Model BKSF – For patients who need covering the lose of partial calf muscle and partial foot amputation, or covering lower limbs prosthesis respectively.
- Model SL – For patients who need covering the lose of partial arm muscle.
- Model CH – For infant patients with hand amputation.
- Model PREM-FING-SP – Regal Finger Prosthesis, for patients who require premium silicone cosmetic prosthesis for finger.

- Model PREM-LL-SP – Regal Lower Limb Prosthesis, for patients who require premium silicone cosmetic prosthesis for Lower Limbs.
- Model PREM-TOE-SP – Regal Toe Prosthesis, for patients who require premium silicone cosmetic prosthesis for toe.
- Model PREM-UL-SP – Regal Upper Limb Prosthesis, for patients who require premium silicone cosmetic prosthesis for upper limbs.
- Semi-Custom Made (SCM) – Adaption to residual limb, applies to all above models.

### **3. Contraindications**

- Our products should not be used in the presence of open wound.

### **4. Precautions**

- Our Silicone Cosmetic Prostheses should be fitted by a certified prosthetist who is familiar with the purpose for which they are prescribed. The prosthetist is responsible for providing wearing instructions and precautions to the amputees.
- Contact your prosthetist if:
  - Unusual swelling, skin irritation occurs, limit blood circulation, stop using the product and consult with the prosthetist immediately;
  - The Silicone Cosmetic Prosthesis has split or any damage has occurred;
  - Any of the inner structure such as finger wires or wrist connector is broken which may cause injury or damage the prosthesis with continued use;
  - The prosthetic socket is broken which may cause injury or damage the Silicone Cosmetic Prosthesis with continued use
- Under any circumstances the silicone prosthesis shows potential safety precaution to the patients should discontinue

to use and to consult your certified prosthetists and orthotists immediately.

## **5. Instructions for Use**

- Model 101L, 102L, 102LS, 101LL, 102LL, 102LSL, CH:
  - Apply water-based lotion onto the prosthetic socket and gently slide the glove onto it. (Consult with your prosthetist for arm socket).
  - For Model 102L and 102LL, epoxy glue can be used at the bottom end of the socket to attach onto the pre-filled foam.
- Model 100, 100L and 100PL:
  - Apply water-based lotion inside the glove surface before installing the glove on the myo-electric or mechanical hand.
- Model 103, 200 series, HDSFHT, HDSFHT-ER, HDSF, HDSF-ER, TOE series, BKSL, BKSF and SL:
  - Apply water-based lotion onto the residual limb before donning the prosthesis.

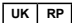
## **6. Care and Cleaning**

- Keep the silicone prosthesis away from fire, solvents and oil-based liquid; DO NOT soak prosthesis with any type of solvent;
- Avoid using sharp objects such as knives or scissors as these could cut and tear the prosthesis easily.
- Do NOT expose the prosthesis to sunlight for extended periods, this could cause the color to fade.
- If a small split happens at the edge of the prosthesis, please return the prosthesis to your prosthetists for removing the split. Otherwise, the small split will continue to grow and break the whole prosthesis.
- DO NOT use excessive force to pull residual fingers from the silicone prosthesis, this may tear the prosthesis. The prosthesis may create a vacuum between the residual limb and the prosthesis inner surface, in this case, simply pinch

the prosthesis and make an air passage to eliminate vacuum before removing the prosthesis.

- Remove stains immediately with mild soap and water. Stains may become permanent if not cleaned immediately, our warranty does not cover such incidents;
- For hygiene purposes, we recommend to cleaning your prosthesis daily with mild soap and water;
- Make use of small items such as bandages, rings, bracelets, sports wrist band or watches to camouflage the prosthesis edge;
- When using a silicone prosthesis with zipper, please ensure the zipper is FULLY opened before donning and doffing. If the zipper is not fully opened while donning or doffing the zipper can be damaged easily; Clean the zipper everyday with water and soft toothbrush, gently brush the zip teeth and remove the dirt;
- For models that integrated finger wires, hinges, or wrist connectors: DO NOT apply force such as using the prosthesis to support or to lift the body or carry any object. Do keep the silicone prosthesis away from fire, solvents and oil-based liquid; DO NOT soak prosthesis with any type of solvent; doing so will damage the finger wires or hinges and injury may result from broken finger wires or hinges caused by misuse. DO NOT use our finger prosthesis to type or carry object. Our silicone prostheses are designed for cosmetic purposes only;
- For all models, DO NOT take the inner filling out of the prosthesis as this will void the warranty.
- For upgrade options (Dual Coloring, Custom Coloring, X-series, Acrylic Nails, Smooth Coating, Custom Filling, Fingers Construction and Glue To Socket), please refer to our catalogue and our website ([www.regalprosthesis.com](http://www.regalprosthesis.com)).
- For more care and cleaning details, please refer to our catalogue or website ([www.regalprosthesis.com](http://www.regalprosthesis.com)), and our YouTube channel.

## 7. Normalized Symbols

	Follow operating instructions
	Complies with the European Medical Device regulation (MDR 2017/745)
	Manufacturer information
	Authorized representative in the European Community
	Authorized representative in United Kingdom

## 8. Storage

- Avoid moisture. Store below 65%RH;
- Avoid direct sunlight. Store in original package when not used.

## 9. Warranty and Return Policy

- The product is warranted against defective materials and workmanship for a period of 12 months from the date of installation, conditions applied. For more details, please refer to our catalogue or website ([www.regalprosthesis.com](http://www.regalprosthesis.com)).
- Warranty will be VOID:
  - if the product comes into contact with aggressive cleaning agents, salt water, chlorinated water or the like;
  - not following the instructions mentioned above;
  - modifications of the products not expressly approved by the manufacturer

## 10. Disposal

- Dispose of the device in accordance with the regulation applicable at the place of operation. If you have any queries, please refer to the local authorities responsible for waste disposal.

## **11. Liability**

- The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by the component combinations that were not authorized by the manufacturer.

## **12. CE conformity**

- This product meets the requirements of Medical Device Regulation (EU) 2017/745 guidelines for medical products. This product has been classified as a class I product according to the classification criteria outlined in appendix IX of the guidelines.
- If any serious incident occur in relation to the device, patient or partitioner should report to the manufacturer and the competent authority of the Member State in which the user and/ or patient is established.

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