medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43 011 24 25 15
 F +43 512 57 95 15
 vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium bvba
 Posthoornstraat 13/1
 3582 Koersel
 Belgium
 T +32 011 24 25 60
 F +32 011 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Brasil
 Rua Neuza 216
 Diadema-Sao Paulo
 Cep 09941-640
 Brazil
 T +55 11 3201 1188
 F +55 11 3201 1185
 sac@medibrasil.com
 www.medibrasil.com

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Important notes

This product is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability according to the Medical Devices Act will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the support, please consult your doctor or orthotist immediately. Do not wear the brace over open wounds, and use it only as instructed by your doctor or orthotist.

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Intended purpose

Genumedi plus is a knee support with patella silicone ring and straps. The product is exclusively to be used for the orthotic fitting of the knee and only on unbroken skin.

Areas of use

- Inflammation of the joint (Arthritis)
- Painful joint wear (arthrosis)
- Irritation of the joint (e.g. excessive stress, injuries, operations) linked to swelling and effusion
- Pain in the kneecap (femoropatellar pain syndrome, anterior knee pain)

Side-Effects / Contraindications

No general side effects are known. However, if supports (or similar aids) or the straps are too tight, local pressure effects or constriction of blood vessels or nerves may occur in individual cases. This can be overcome by opening the straps and closing less tightly. If you have any of the following complaints please consult your doctor before using the brace:

- Diseases or injuries of the skin in the area of use, especially with inflammatory symptoms (excessive warming, swelling or reddening).
- Sensitivity and perfusion disorders of the legs or feet (e.g. in the case of diabetes).
- Lymph outflow disorders, as well as undiagnosed swellings of soft tissues, apart from the area of use.

Any combination with other products (e.g. compression stockings) must be discussed in advance with the responsible doctor.

Mode of Action / Comment on Wear

The integrated silicone ring of the Genumedi reduces stress on the kneecap. Together with the compression of the knitted fabric, it causes a massage effect. This enhances perfusion of the joint and decreases swelling and effusion. Irritation and pain are alleviated. At the same time, muscular control and inherent stabilisation of the knee joint are improved.

Supports are most beneficial during physical activity. The support can in principle be worn throughout the day. This must depend on how comfortable the wearer is. Moreover, the knee may not be bent by more than 70° when sitting. The support should be removed for long breaks (e.g. long periods sitting, long drives or sleeping).

If you suffer excessive pain or discomfort during wear, please take off the product immediately and contact your doctor or specialised shop.

The support is suitable for use in water. After use, please rinse thoroughly

Putting on / Taking off

- Loosen the two straps.
- Grasp the support with the thumb on the internal upper end of the lateral knob adhesive bands (Fig. 1)
- Pulls the support over the knee in such a way that the silicone ring covers the kneecap centrally and free of pressure (Figs. 2 +3).
- Then close the straps. (Fig. 4)
- To take off the support, grasp the lower lateral rods and pull it downwards and away (Fig. 5).

Please take care that trained staff instruct you when putting on the support for the first time.

Care instructions

Velcro fasteners should be closed for washing. The silicone ring does not need to be removed before washing. Fabric softener, fats, oils and ointments can attack the material and the Clima Comfort Effect can be impaired. Soap residues can cause skin irritation and material wear.

- Wash the product by hand, preferably using a medi clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

**Note on Storage**

Must be stored dry and at room temperature. Must be protected from the sun and moisture.

Material composition

Polyamide, elasthan

Warranty / contractual guarantee

Consistent with legal guidelines.

Disposal

The product can be disposed of in the domestic waste.

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