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INSTRUCTIONS FOR USE GB

Dear Customer,

we would like to ask you to follow these Instructions for Use with care. If you should have any queries, ask the doctor treating you, at your nearest stockist or contact us directly.

FUNCTION

By compression and friction massage on the muscles of the forearm, the EPI-HIT® Brace stimulates the lymphatic system and reduces tension in the area of the muscles and tendons. This positive reduction in tension enhances the physiological healing process of the part of the tendon or tissue affected.

EPI-HIT® brace therapy can be supplemented in terms of extended functional therapy by the additional use of a Wrist Bandage.

Because the wrist bandage partially immobilises the wrist, this prevents the increase in mechanically caused tension of the extensor muscles. In this way the Wrist Bandage makes a substantial contribution to reducing the pulling forces and to the associated tissue regeneration. This reduces rotational/tensile movements or tensile movements that promote epicondylitis.

INDICATIONS

- Epicondylitis (rad.hum.)
- Lateral tendomyopathy

CONTRA-INDICATIONS

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.
- Impaired circulation or lymphatic soft tissue swelling.
- Neurogenically-specific organoleptic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far not consequential incompatibilities or allergic reactions have been reported.

SIDE-EFFECTS

With proper use and proper fitting, so far there have been no reports of serious general side-effects.

Local pressure symptoms and impaired circulation can be

prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restricting consistently shaped body fitting.

FITTING

The ideal position for the EPI-HIT® Brace is about 3–4 cm below the inflamed tendon tissue or directly on the pathologically hypertonic (twisted) muscle.

- Adjust the brace via the rear catches so that the ends lie on the muscle bellies (1); see the picture for correct fit. Adjust the pad appropriately.
- Close the loop-and-hook fastener.
- The closure of the brace is continuously variable and the brace should be adjusted so that it sits firmly without causing unpleasant pressure. The correct pressure has been adjusted if, after a few minutes, the clasp can no longer be felt when the arm is relaxed, yet when the hand is used the pressure of the pad is once again noticeable.
- If the clamp is too tight then loosen it slightly or move the clasp downwards by a few millimeters. The clasp should also be moved downwards if it pinches when the elbow is bent. Wear the clasp throughout the day, but please remove it for the night. In terms of functional therapy we recommend that you use the Therapy Supplement Set.
- The wrist bandage should surround the wrist fully. You now dose the degree of the supporting function as required with the stabilizing band.
- Note for the orthopaedic technician: If the Velcro closure straps are too long, they can be shortened at one of the seals. The smaller pad should lie under the larger pad where they overlap. The Velcro fastening point can be moved, if necessary. The pad can be shortened where it narrows at one of the thinner strips, as needed.

IMPORTANT

The product is to be used by one patient only.

Improper modifications to the product and/or improper use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

QUALITY MANAGEMENT SYSTEM

All SPORLASTIC GmbH products undergo product testing as part of our Quality Management System.

However, should you have any complaints about our product, please get in touch with your stockist or directly with us.

CARE

We recommend that you wash the EPI-HIT® Brace and Wrist Bandage gently in lukewarm water with cold detergent and air-dry (do not place on top of radiator). Removal of the pad is not necessary for laundering purposes. Close the Velcro fasteners when washing for them to last longer and stop them damaging other items of clothing.

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