medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium bvba
Posthoornstraat 13/1
3582 Koersel
Belgium
T +32 011 24 25 60
F +32 011 24 25 64
info@medi-belgium.be
www.medi-belgium.be

medi Brasil
Rua Neusa 216
Diadema-Sao Paulo
Cep 09941-640
Brazil
T +55 11 3201 1188
F +55 11 3201 1185
sac@medibrasil.com
www.medi-brasil.com

medi Danmark ApS
Vejlegardsvej 59
2665 Vallensbæk Strand
Denmark
T +45 46 55 75 69
F +45 70 25 56 20
kundeservice@sw.dk
www.meditandmark.dk

medi Bayreuth España SL
C/Canigó 2 – 6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34 93 260 04 00
F +34 93 260 23 14
medi@mediespana.com
www.mediespana.com

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33 1 48 61 76 10
F +33 1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
Bokor u. 21.
1037 Budapest
Hungary
T +36 1371 0090
F +36 1371 0091
info@medi-hungary.hu
www.medi-hungary.hu

medi Medical Support Sdn Bhd
medi representative office Asia
Unit No. B-2-19, Block B, No. 2,
Jalan PJU 1A/7A
Oasis Ara Damansara, PJU 1A,
47301 PETALING JAYA
Selangor Darul Ehsan
Malaysia
T: +6 03 7832 3591
F: +6 03 7832 3921
info@medi-asia.com
www.medi-asia.com

medi

medi Epibrace®

Gebrauchsanweisung. Instructions for use.
Mode d'emploi. Instrucciones de uso. Instruções
para aplicação. Istruzioni per l'uso.
Gebruiksaanwijzing. Brugsvejledning.
Bruksanvisning. Návod k použití. Uputa za uporabu.
Instrukcija zakladania. Οδηγία εφαρμογής.
Instrukcja z wykorzystania. Navodila za uporabo.

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Epicondylitis orthosis

Indications

Inflammation of the common tendon at the elbow, e.g. tennis elbow (lateral epicondylitis) or golfer's elbow (medial epicondylitis)

Contraindications

No side effects have been reported to date. In the following circumstances you should consult your doctor before use:

- Disorders or injuries of the skin in the area where the product is used, above all in case of signs of inflammation (excessive heat, swelling or reddening).
- Sensory or circulatory disturbances of the forearm or the hand.
- Disorders of lymphatic drainage and swelling of unclear origin of the soft tissues distal to the elbow.
- Impingement syndrome (pressure irritation of the nerves supplying the arm).

If you have a known history of contact allergy or tenderness to pressure, you should wear a thin cotton armsleeve under the medi Epibrace as a preventive measure. In principle, supports or braces that fit too tightly to or around the body can cause local pressure phenomena or, more rarely, to constriction of blood vessels or nerves.

Wearing directions

During longer periods of rest (e.g. when sleeping) you should take the medi Epibrace off.

Applying the strap

- Attach the pad within the marked frame depending on the indication. For lateral epicondylitis (tennis elbow) onto the outer area, for medial epicondylitis (golfer's elbow) onto the inner area. For the right arm ("R") the tip of the pad faces upwards, for the left arm ("L") the tip of the pad faces downwards. When wearing the support the tip of the pad should always point towards the hand.
- Adjust the circumference to be as large as possible and fix the strap to the support (fig. 2).

- Pull the support up over the affected forearm, always with the fastener facing outwards (fig. 3).
- In order to find the correct position for the pad on your forearm pull your clenched fist upwards if you have lateral epicondylitis, or downwards if you have medial epicondylitis. Place the pad on the tense muscles (radialis = outside of the arm [lateral]), ulnaris = inside of the arm [medial] so that the edge of the support is about 5 cm from your elbow joint (fig. 4).
- Lay your forearm on a firm surface and tighten the strap to the number that corresponds to the circumference of your arm, e.g., up to “6” for a forearm circumference of 29 cm (fig. 5). The support should now fit your arm firmly without slipping or causing constriction. You can adjust the pressure to maintain your own personal comfort.

The product must always be used on the instructions of a doctor and put on for the first time under the supervision of trained professional staff, e.g. an orthopaedic technician. Please ask your doctor if you are using other products at the same time.

Care instructions

Close the Velcro fastener before washing. Wash the product at 30° C by hand using a mild detergent such as woolwash. Rinse the product thoroughly and hang up to dry. Do not dry on the radiator or in direct sunlight. Do not remove stains using benzene or chemical detergents. Soap residues can cause skin irritations and wearing of the material.

Storage instructions

Store in a dry place at room temperature. Protect from heat, direct sunlight and moisture.

Your medi team

wishes you a quick return to full fitness.

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