medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at

medi Belgium bvba
Posthoornstraat 13/1
3582 Koersel
Belgium
T +32 011 24 25 60
F +32 011 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Brasil
Rua Neusa 216
Diadema-Sao Paulo
Cep 09941-640
Brazil
T +55 11 3201 1188
F +55 11 3201 1185
sac@medibrasil.com
www.medibrasil.com

medi Danmark ApS
Vejlegardsvej 59
2665 Vallensbæk Strand
Denmark
T +45 46 55 75 69
F +45 70 25 56 20
kundeservice@sw.dk
www.medit danmark.dk

medi Bayreuth España SL
C/Canigó 2 – 6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34 93 260 04 00
F +34 93 260 23 14
medi@mediespana.com
www.mediespana.com

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33 1 48 61 76 10
F +33 1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
Thán Károly utca 20
1119 Budapest
Hungary
T +36 1371 0090
F +36 1371 0091
info@medi-hungary.hu
www.medi-hungary.hu

medi Middle East
P. O. Box: 109307
Abu Dhabi
United Arab Emirates
T +971 2 6429201
F +971 2 6429070
info@mediuae.ae
www.mediuae.ae

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31 76 57 22 555
F +31 76 57 22 565
info@medi.nl
www.medi.nl

medi Polska Sp. z o. o.
Chełmońskiego 1a
44-100 Gliwice
Poland
T +48 32 230 60 21
F +48 32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
Rua Eugénio de Castro Rodrigues, nº2
1700-184 Lisboa
Portugal
T +351 21 843 71 60
F +351 21 847 08 33
medi.portugal@medibayreuth.pt
www.medi.pt

medi RUS LRC
Business Park "Rumyantsevo"
Rumyantsevo, Build. 1
Leninsky Township
142784 Moscow Region
Russia
T +7 495 229 04 58
F +7 495 229 04 58
info@medirus.ru
www.medirus.ru

medi UK Ltd.
Plough Lane
Hereford HR4 OEL
United Kingdom
T +44 1432 37 35 00
F +44 1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

medi Ukraine LLC
Tankova Str. 8, office 38
Business-center "Flora Park"
Kiev 04112
Ukraine
T +380 44 591 1163
F +380 44 455 6181
office@medi-ua.com
www.medi-UA.com

medi USA L.P.
6481 Franz Warner Parkway
Whitsett, N.C. 27377-3000
USA
T +1 336 4 49 44 40
F +1 888 5 70 45 54
orders@mediusa.com
www.mediusa.com

medi

Genumedi®

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Ekipa medi Vam želi hitro okrevanje!

Genumedi®

Knee support with patella silicone ring

Dear Patient

We are pleased that you have decided to use the high-quality product from medi and thank you for the trust you have shown in us. Please read these instructions before using the support for the first time. If you apply the support correctly, it will develop its complete medical efficacy and it will be comfortable to wear – so that you will feel better soon.

Areas of use

- Inflammation of the joint (Arthritis)
- Painful joint wear (arthrosis)
- Irritation of the joint (e.g. excessive stress, injuries, operations) linked to swelling and effusion
- Pain in the kneecap (femoropatellar pain syndrome, anterior knee pain)

Side-Effects / Contraindications

No general side effects are known. However, if supports (or similar aids) are too tight, local pressure effects or constriction of blood vessels or nerves may occur in individuals. If you have any of the following complaints please consult your doctor before using the brace:

- Diseases or injuries of the skin in the area of use, especially with inflammatory symptoms (excessive warming, swelling or reddening).
- Sensitivity and perfusion disorders of the legs or feet (e.g. in the case of diabetes).
- Lymph outflow disorders, as well as undiagnosed swellings of soft tissues, apart from the area of use.

Any combination with other products (e.g. compression stockings) must be discussed in advance with the responsible doctor.

Mode of Action / Comment on Wear

The integrated silicone ring of the Genumedi reduces stress on the kneecap. Together with the compression of the knitted fabric, it causes a massage effect. This enhances perfusion of the joint and decreases swelling and effusion. Irritation and pain are alleviated. At the same time, muscular control and inherent stabilisation of the knee joint are improved.

Supports are most beneficial during physical activity. The support can in principle be worn throughout the day. This must depend on how comfortable the wearer is. Moreover, the knee may not be bent by more than 70° when sitting. The support should be removed for long breaks (e.g. long periods sitting, long drives or sleeping).

If you suffer excessive pain or discomfort during wear, please take off the product immediately and contact your doctor or specialised shop.

The support is suitable for use in water. After use, please rinse thoroughly

Putting on / Taking off

- Grasp the support with the thumb on the internal upper end of the lateral knob adhesive bands (Fig. 1)
- Pull the support over the knee in such a way that the silicone ring covers the kneecap centrally and free of pressure (Figs. 2 +3).
- To take off the support, grasp the lower lateral rods and pull it downwards and away (Fig. 4).

Please take care that trained staff instruct you when putting on the support for the first time.

Instruction for Washing / Care Tips

The silicone ring does not need to be removed before washing. (See figs. 5 on how to re-insert). Wash the product separately with a woollen or mild detergent up to 30° in cradle wash action or by hand. Rinse out the product properly and hang it up to dry. Do not dry it in the dryer, on the radiator or in direct sun light Do not remove spots with benzine or chemical detergents. Fabric softener, fats, oils and ointments can attack the material and the Clima Comfort Effect can be impaired. Soap residues can cause skin irritation and material wear.

**Note on Storage**

Must be stored dry and at room temperature. Must be protected from the sun and moisture.

Composition of the Material

Polyamide, polyester, elasthan

Statutory Defects Liability/Contractual Guarantee

Consistent with legal guidelines.

Important Notes

It is recommended that the support should only be used for single patient. If the support is used for the treatment of more than one patient or is improperly modified, the product liability of the producer lapses according to the Medical Product Law. If you feel too much pain or an unpleasant feeling when wearing the support, take it off at once and contact your doctor or your specialist provider.

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