medi GmbH & Co. KG  
Medicusstraße 1  
D-95448 Bayreuth  
Germany  
T +49 921 912-0  
F +49 921 912-780  
ortho@medi.de  
www.medi.de

medi Austria GmbH  
Adamgasse 16/7  
A-6020 Innsbruck / Austria  
T +43 512 57 95 15  
F +43 512 57 95 15 45  
vertrieb@medi-austria.at

medi Belgium bvba  
Voortstraat 79  
3580 Beringen / Belgium  
T +32 11 24 25 60  
F +32 11 24 25 64  
info@medibelgium.be  
www.medibelgium.be

medi Danmark  
Vejlegardsvej 59  
2665 Vallensbæk Strand / Denmark  
T +45 46 55 75 69  
F +45 70 25 56 20  
kundeservice@sw.dk  
www.meditandmark.dk

medi Bayreuth España SL  
C/Canigó 2 – 6 bajos  
Hospitalet de Llobregat  
08901 Barcelona / Spain  
T +34 93 260 04 00  
F +34 93 260 23 14  
medi@mediespana.com  
www.mediespana.com

medi UK Ltd.  
Plough Lane  
Hereford HR4 OEL / United Kingdom  
T +44 1432 37 35 00  
F +44 1432 37 35 10  
enquiries@mediuk.co.uk  
www.mediuk.co.uk

medi Polska Sp. z o. o.  
Chełmońskiego 1a  
44-100 Gliwice / Poland  
T +48 32 230 60 21  
F +48 32 202 87 56  
info@medi-polska.pl  
www.medi-polska.pl

medi France  
Z.I. Charles de Gaulle 25,  
rue Henri Farman  
93297 Tremblay en France Cedex / France  
T +33 1 48 61 76 10  
F +33 1 49 63 33 05  
infos@medi-france.com  
www.medi-france.com

medi Italia S.r.l.  
Via Guido Rossa 28  
40033 Casalecchio di Reno-Bologna / Italy  
T +39 051 6 13 24 84  
F +39 051 6 13 29 56  
info@medi-italia.it  
www.medi-italia.it

medi Nederland BV  
Heusing 5  
NL-4817 ZB Breda / The Netherlands  
T +31 76 57 22 555  
F +31 76 57 22 565  
info@medi.nl  
www.medi.nl

medi Bayreuth Unipessoal, Lda  
Rua Eugénio de Castro Rodrigues, nº2  
1700-184 Lisboa / Portugal  
T +351 21 843 71 60  
F +351 21 847 08 33  
medi.portugal@medibayreuth.pt

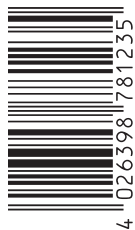
medi RUS LRC  
Business Park "Rumyantsevo"  
Rumyantsevo, Build. 1 Leninsky Township  
142784 Moscow Region / Russia  
T +7 495 229 04 58  
F +7 495 229 04 58  
info@medirus.ru  
www.medirus.ru

medi USA L.P.  
6481 Franz Warner Parkway  
Whitsett, N.C. 27377-3000 / USA  
T +1 336 4 49 44 40  
F +1 888 5 70 45 54  
orders@mediusa.com  
www.mediusa.com

medi Middle East  
P. O. Box: 109307  
Abu Dhabi / United Arab Emirates  
P +971 2 6429201  
F +971 2 6429070  
info@mediuae.ae  
www.mediuae.ae

medi Sweden AB  
Box 6034  
192 06 Sollentuna / Sweden  
P +46 8-96 97 98  
F +46 8-626 68 70  
info@medi.se  
www.medi.se

medi Hungary Kft.  
H - 1119 Budapest  
Than K. u. 20. / Hungary  
T +36 / 1371 0090  
F +36 / 1371 0091  
E-Mail: info@medi-hungary.hu  
www.medi-hungary.hu

E006456/04.10/1

# Lumbamed® stabil

## Lumbalstützorthese Lumbar support Ceinture de soutien lombaire Órtesis con soporte lumbar

Gebrauchsanweisung. Instructions for use. Mode d'emploi. Instrucciones de uso.  
Instruções para aplicação. Istruzioni per l'uso. Gebruiksaanwijzing. Brugsvejledning.  
Bruksanvisning. Návod k použití. Uputa za uporabu. Инструкция по использованию.  
Kullanma kılavuzu. Instrukcja zakładania. Οδηγία εφαρμογής.

medi

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### Important notes

This brace is recommended for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability according to the Medical Devices Act will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the support, please consult your doctor or orthotist immediately. Do not wear the brace over open wounds, and use it only as instructed by your doctor or orthotist.

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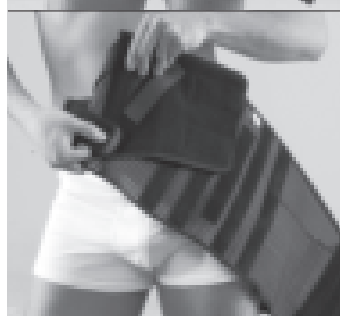
## Lumbar orthosis

### Indications

- For the treatment of painful conditions of the lumbar spine
- Lumbar sciatica (degenerative cause)
- Spondylosis without vertebral slipping
- Mild Spondylosis / Spondylolisthesis
- Disc protrusion
- Osteochondrosis, spondyloarthrosis
- Irritation of the ISG

### Contraindications

None known to date

### Mode of action

Controlled compression is applied through the material and integral stabilising rods; this supports the trunk muscles, relieving strain on the lumbar spine and easing pain.  
(The optional back pad, which can be ordered separately, reinforces this effect by exerting additional pressure on the paravertebral tissues and a massage effect on the back muscles).

### Wearing directions

Unless otherwise directed by your doctor, do not wear the support for more than 8 hours at a time. After a break of approx. 2 hours it may be refitted.

### Fitting instructions

- If a pad has been prescribed for you please first attach it to the inner fleece strip as your orthopaedic technician showed you. The wider surface always faces upwards. (fig. 1)
- First secure the ends of the two tension straps to the outer edge of the fasteners. (fig. 2)
- Pass your left hand though the loop of the lower fastener (no medi logo).
- Pass the support around your body and pass your right hand through the hand loop of the upper fastener (medi label). (fig. 3+4)
- Now position the support on your back so that the spine lies in the middle between the stabilisation rods, or the central recess in the back pad lies over the spine. (fig. 5)
- Hold the lower fastener tightly in front of your abdomen, pull the upper fastener over it and attach it provisionally to the narrow hook strip. (fig. 6)
- After pulling your left hand back through the loop you can use the hand to fix the upper fastener flat against the lower fastener. (fig. 7)
- Finally, pull the tension straps to achieve the desired force and then secure them, the left-hand strap first. (fig. 8)

---

**Care instructions**

Velcro fasteners should be closed for washing. Remove the back pelotte before washing. Wash the product at 30 °C on a delicates setting or by hand using a mild detergent such as woolwash. Rinse the product thoroughly and hang up to dry. Do not dry on the radiator or in direct sunlight. Do not remove stains using benzene or chemical detergents. Fabric softeners, grease, oil, lotions and ointments can affect the bandaging material. Soap residues can cause skin irritations and wearing of the material.

---

**Storage instructions**

Keep in a dry place at room temperature. Protect from heat, direct sunlight and moisture.

**Your medi team**

wishes you a quick return to full fitness.

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