medi

Manumed

Handgelenkorthese

Wrist support

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Órtesis de muñeca

Gebrauchsanweisung. Instructions for use. Mode d'emploi. Instrucciones de uso. Instruções para aplicação. Istruzioni per l'uso. Gebruiksaanwijzing. Brugsvejledning. Bruksanvisning. Návod k použití. Uputa za uporabu. Инструкција по использованию. Kullanma kılavuzu. Instrukcja zakładania. Οδηγία εφαρμογής. Uputstvo za upotrebu. Navodila za uporabo.

medi. I feel better.

1

2

3

4

5

Manumed

Wrist support

Indications

- Rheumatoid arthritis
- Immobilisation for distortions of the wrist
- Soft tissue inflammation (chronic, post-traumatic, postoperative)
- Secondary consequences of paralysis (radial paresis)
- Tendinitis, tenosynovitis

Contraindications

None known to date

Mode of action

Immobilisation and stabilisation of the wrist with volar aluminium support.

Wearing directions

Only wearing the Manumed consistently will support the therapy.

Applying the support

The initial fitting should be supervised by a healthcare professional.

- The pre-shaped aluminium splint can be removed and easily moulded to match different wrist contours (fig. 1).
- Open all the straps and slip your hand into the support (with the aluminium splint at the bottom – fig. 2).
- First close the front strap and then the back strap (fig. 3).
- Now close the broad central strap. This strap is attached to the underside of the support by Velcro and can be moved before fastening if necessary (fig. 4 + 5).

Care instructions

Hand wash in cool, soapy water. Rinse thoroughly. Do not wring. Leave to dry naturally. Do not iron. Soap residues can cause skin irritation and material wear. The product can be worn in both fresh and salt water. After wearing in water, rinse the product thoroughly with clean fresh water. Leave to dry naturally. Do not remove stains using benzene or chemical detergents.

Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

Material composition

Aluminum, polyamide.

Warranty / contractual guarantee

Consistent with legal guidelines.

Your medi team

wishes you a quick return to full fitness.

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