medi

Manumed active

Handgelenkbandage

Wrist support

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Muñequera

Gebrauchsanweisung. Instructions for use. Mode d'emploi.

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Manumed active

Dear Patient

We are pleased that you have decided to use a high-quality product from medi and thank you for the trust you have shown in us.

Please read these instructions before using the support for the first time. If you apply the support correctly, it will develop its complete medical efficacy and it will be comfortable to wear – so that you will feel better soon.

Areas of use

- Painful attrition of the wrist (arthrosis)
- Irritations and inflammations in the wrist (e.g. arthritis, rheumatic, through overstrain, after operations, etc.)
- Wrist injuries, e.g. sprains
- Tendon inflammations
- Feelings of instability in the wrist

Side-effects / Contraindications

There are no known side effects if the correctly sized product is fitted as recommended. However, if supports (or similar aids) are too tight, this may lead to local pressure effects or the constriction of blood vessels or nerves in individual cases. You should thus consult your doctor in the following circumstances and before applying the support,:;

- Diseases or injuries of the skin in the area of use, especially with inflammatory symptoms (excessive warming, swelling or reddening).
- Paralgesia and circulatory disorders of the legs or feet (e.g. in the case of diabetes).
- Lymph outflow disorders as well as unclear swellings of soft tissues outside the area of use.

Any combination with other products (e.g. arm stockings) must be discussed beforehand with the attending doctor.

Mode of action / Comment on wear

Manumed active has a stabilising effect that prevents unhealthy movements of the wrist through the interaction of compressive knitted fabric, belt strap and individually adjustable splints. This relieves the muscles of the lower arm as well as tendons and tendon insertions in the affected area and reduces irritations. The compressive effect of the knitted fabric also restricts swelling.

In principle, the support can be worn for the whole day but should be taken off during longer periods of rest (e.g. sleep). But you should make this dependent

on how you feel. In the event of unexpected, excessive changes, e.g. an intensification of the symptoms, please take the product off immediately and contact your doctor or supplying dealer.

The support is suitable for use in water. After use, please rinse out thoroughly.

Putting on / Taking off

Open the belt strap before putting the support on.

1. Fold back the lower end of the support. The splint is now inserted into the corresponding pocket on the inside of the support. Pay attention to the correct alignment of the splint to the pocket.
2. Pull the support over your hand so that you can see the „m“ on the back of your hand.
3. Close the belt strap so that the opening in the belt strap is positioned over the outer knuckle.

To remove, open the belt strap. Grasp inside the support from the palm.

Pull the support off your hand while pressing against the inside pocket and splint.

Please note that the first time you put the support on you should be instructed by qualified personnel.

Adjusting the splint

The splint can be individually adjusted at a temperature of around 200 °C.

Instruction for washing / Care tips

Wash the product separately with a woollen or mild detergent up to 30° in cradle wash action or by hand. Rinse out the product properly and hang it up to dry. Do not dry it in the dryer, on the radiator or in direct sun light. Do not remove spots with benzene or chemical detergents. Fabric softener, fats, oils and ointments can attack the material and the Clima Comfort Effect can be impaired. Soap residues can cause skin irritation and material wear.

Note on storage

Must be stored dry and at room temperature. Must be protected from the sun and moisture.

Composition of the material

Polyamide, polyester, elastan

Statutory Defects Liability/Contractual Guarantee

Consistent with legal guidelines.

Important notes

It is recommended that the support should only be used for a single patient. If the support is used for the treatment of more than one patient or is improperly modified, the product liability of the producer will be forfeited according to the Medical Product Law. If you feel too much pain or an unpleasant feeling when wearing the support, take it off at once and contact your doctor or your specialist provider.

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