medi GmbH & Co. KG  
Medicusstraße 1  
D-95448 Bayreuth  
Germany  
T +49 921 912-0  
F +49 921 912-57  
info@medi.de  
www.medi.de

medi Austria GmbH  
Adamgasse 16/7  
6020 Innsbruck  
Austria  
T +43 512 57 95 15  
F +43 512 57 95 15 45  
vertrieb@medi-austria.at  
www.medi-austria.at

medi Belgium bvba  
Posthoornstraat 13/1  
3582 Koersel  
Belgium  
T +32 011 24 25 60  
F +32 011 24 25 64  
info@medibelgium.be  
www.medibelgium.be

medi Brasil  
Rua Neusa 216  
Diadema-Sao Paulo  
Cep 09941-640  
Brazil  
T +55 11 3201 1188  
F +55 11 3201 1185  
sac@medibrasil.com  
www.medibrasil.com

medi Danmark ApS  
Vejlegardsvej 59  
2665 Vallensbæk Strand  
Denmark  
T +45 46 55 75 69  
F +45 70 25 56 20  
kundeservice@sw.dk  
www.meditandmark.dk

medi Bayreuth España SL  
C/Canigó 2 – 6 bajos  
Hospitalet de Llobregat  
08901 Barcelona  
Spain  
T +34 93 260 04 00  
F +34 93 260 23 14  
medi@mediespana.com  
www.mediespana.com

medi France  
Z.I. Charles de Gaulle  
25, rue Henri Farman  
93297 Tremblay en France Cedex  
France  
T +33 1 48 61 76 10  
F +33 1 49 63 33 05  
infos@medi-france.com  
www.medi-france.com

medi Hungary Kft.  
Bokor u. 21.  
1037 Budapest  
Hungary  
T +36 1371 0090  
F +36 1371 0091  
info@medi-hungary.hu  
www.medi-hungary.hu

medi Medical Support Sdn Bhd  
medi representative office Asia  
Unit No. B-2-19, Block B, No. 2,  
Jalan PJU 1A/7A  
Oasis Ara Damansara, PJU 1A,  
47301 PETALING JAYA  
Selangor Darul Ehsan  
Malaysia  
T: +6 03 7832 3591  
F: +6 03 78323921  
info@medi-asia.com  
www.medi-asia.com

medi Middle East  
P. O. Box: 109307  
Abu Dhabi  
United Arab Emirates  
T +971 2 6429201  
F +971 2 6429070  
info@mediuae.ae  
www.mediuae.ae


EO10050/07.2013

## protect. Bunion cushion

Medizinischer Ballenschutz aus Silikon  
Silicone bunion cushion  
Protection médicale en silicone  
Protector de silicona para juanetes

Gebrauchsanweisung. Instructions for use. Mode d'emploi.  
Instrucciones de uso. Instruções para aplicação. Istruzioni per  
l'uso. Návod k použití. Инструкция по использованию.

medi

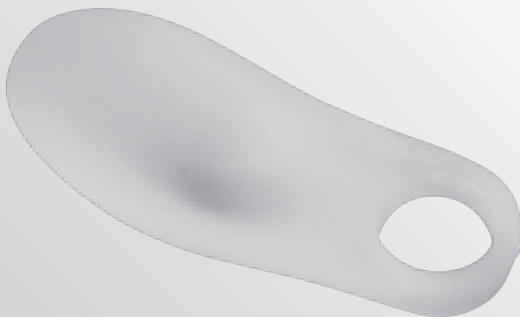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### Important notes

This product is recommended for one patient use only. If it is used for treating more than one patient, the manufacturer's product liability according to the Medical Devices Act will become invalid. If undue pain or an unpleasant sensation develops while you are wearing this product, please take it off immediately and consult your doctor or orthotist.

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---

### Indications

- Correction
- Pressure relief
- Prophylactic use for the diabetic foot

### Contraindications

Do not wear the product over open wounds.

### Mode of action

For the rapid, lasting relief of pain caused by bunions.

### Properties

Made from high-quality silicone, skin-friendly and very pleasant to wear. Adapts to the shape of every part of the foot without bunching up and returns to its original shape immediately after wearing. Made from a hygienic material that can be washed with soap and water and disinfected if necessary.

### Instructions for use

Apply to clean, dry skin. Slip the big toe into the loop intended for it so that the cushion lies on the part of the foot that is particularly affected.

---

### Care instructions

Can be washed in soap and water. Dry well. Do not dry on a radiator.

---

### Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

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