medi

protect.Forefoot cushion plus

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Silicone forefoot cushion

Indications

- Correction
- Pressure relief
- Prophylactic use for the diabetic foot

Contraindications

None known to date if used as instructed

Mode of action

For pressure relief in the forefoot area and for support of foot arch.

Properties

Made from high-quality silicone, skin-friendly and very pleasant to wear. Adapts to the shape of every part of the foot without bunching up and returns to its original shape immediately after wearing. Made from a hygienic material that can be washed with soap and water and disinfected if necessary.

Instructions for use

Apply to clean, dry skin. Place the product in the area of the forefoot and position the toe strap around the second or third toe as required.

Care instructions

Can be washed in soap and water. Dry well. Do not dry on a radiator.

Important notes

This product is recommended for one patient use only. If it is used for treating more than one patient, the manufacturer's product liability according to the Medical Devices Act will become invalid. If undue pain or an unpleasant sensation develops while you are wearing this product, please take it off immediately and consult your doctor or orthotist. Do not wear this product over open wounds, and use it only as instructed by your doctor or orthotist. It is important to fit this product correctly, to ensure that it will function effectively.

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