medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-57
info@medi.de
www.medi.de

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at

medi Belgium bvba
Posthoornstraat 13/1
3582 Koersel
Belgium
T +32 11 24 25 60
F +32 11 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Brasil
Rua Neusa 216
Diadema-Sao Paulo
Cep 09941-640
Brazil
T +55 11 3201 1188
F +55 11 3201 1185
sac@medibrasil.com

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Important notes

This product is recommended for one patient use only. If it is used for treating more than one patient, the manufacturer's product liability according to the Medical Devices Act will become invalid. If undue pain or an unpleasant sensation develops while you are wearing this product, please take it off immediately and consult your doctor or orthotist.

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4 046114 441786

protect.Heel soft

Medical silicone heel cushion with “soft spot“

Indications

- Pain of the Achilles tendon and bursa (achillodynia)
- Calcaneal spur
- Arthrosis
- Repetitive strain tendinopathy
- For leg length differences of up to 1 cm (compensation)

Contraindications

Do not wear the product over open wounds.

Mode of action

The soft cushion and the soothing massage effect relieve the symptoms caused by pressure on the heel. Relief of the calf musculature, the tendons and the ligaments. Absorption of the shock on the ankle, knee and hip joints as well as the spine.

Properties

Made of high quality silicone, skin-friendly and non-slip. Special soft cushioning of the sensitive heel area with the integrated “soft spot“

Instructions for use

- Place the insoles in the heels of your shoes.
(If necessary, take out any existing insoles first to achieve the optimum fit).
 - Make sure you put insoles in both shoes.
(Exception: if the insole is being used to compensate for different leg lengths.)
 - The heel cushions can be worn in street, sports and casual shoes, but also in working shoes.
-

Care instructions

Wash by hand in a lukewarm detergent for delicates. Do not place on a radiator to dry.

Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

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