medi

protect.Lumbostyle

Lumbalstützorthese

Lumbar orthosis

Ceinture lombaire

Ortesis con soporte lumbar

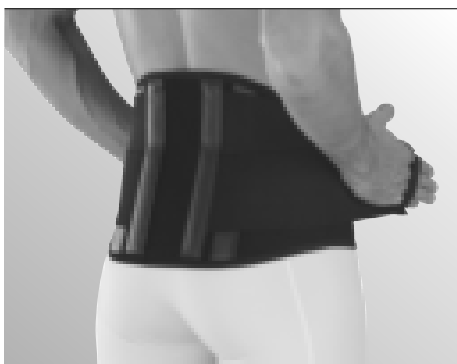
Gebrauchsanweisung. Instructions for use. Mode d'emploi. Instrucciones de uso.

Instruções de uso. Istruzioni per l'uso. Gebruiksaanwijzing. Brugsvejledning.

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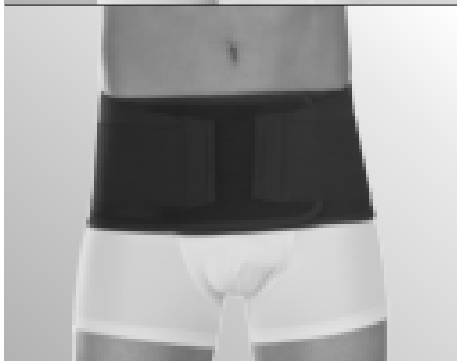
Lumbar orthosis

Indications

- For the treatment of painful conditions of the lumbar spine
- Lumbar sciatica (degenerative cause)
- Spondylosis without vertebral slipping
- Mild Spondylosis / Spondylolisthesis
- Disc protrusion
- Osteochondrosis, spondyloarthrosis
- Irritation of the ISG

Contraindications

None known to date

Mode of action

Controlled compression is applied through the material and integral stabilising rods; this supports the trunk muscles, relieving strain on the lumbar spine and easing pain. Wearing directions. Unless otherwise directed by your doctor, do not wear the support for more than 8 hours at a time. After a break of approx. 2 hours it may be refitted.

Fitting instructions

- First secure the ends of the two tension straps to the outer edge of the fasteners.
- Wrap the support around the body.
- Press the left side of the fastener on your abdomen. Put the right side over the left to and secure it.
- The support should be pulled down until the lower edge is just above the groin. Always wear the support under your clothes.
- Finally, pull the tension straps to achieve the desired force and then secure them, the left-hand strap first.

Care instructions

Velcro fasteners should be closed for washing. Wash the product at 30 °C on a delicates setting or by hand using a mild detergent such as woolwash. Rinse the product thoroughly and hang up to dry. Do not dry on the radiator or in direct sunlight. Do not remove stains using benzene or chemical detergents. Fabric softeners, grease, oil, lotions and ointments can affect the bandaging material. Soap residues can cause skin irritations and wearing of the material.

Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

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