medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-57
info@medi.de
www.medi.de

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at

medi Belgium bvba
Posthoornstraat 13/1
3582 Koersel
Belgium
T +32 011 24 25 60
F +32 011 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Brasil
Rua Neusa 216
Diadema-Sao Paulo
Cep 09941-640
Brazil
T +55 11 3201 1188
F +55 11 3201 1185
sac@medibrasil.com
www.medibrasil.com

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Important notes

It is recommended that the product should only be used for single patient. If the product is used for the treatment of more than one patient or is improperly modified, the product liability of the producer lapses according to the Medical Product Law. If you feel too much pain or an unpleasant feeling when wearing the product, take it off at once and contact your doctor or your specialist provider. Do not wear the product over open wounds, and use it only as instructed by your doctor or orthotist.

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4 046114 609438

protect.Metatarsal cushion

Medical silicone metatarsal cushion

Indications

- For flat foot and splay foot
- Pressure relief

Contraindications

None known to date if used as instructed

Mode of action

Relieves pressure and helps reduce pain in the case of flat foot and splay foot.

Properties

Made from high-quality silicone, skin-friendly and very pleasant to wear. Adapts to the shape of every part of the foot without bunching up and returns to its original shape immediately after wearing. Made from a hygienic material that can be washed with soap and water and disinfected if necessary.

Instructions for use

Apply to clean, dry skin. Note the designation R for right and L for left. Position the product, with the structured massage side upward, in the middle of the foot so that the cushion rests against the area that is particularly affected.

Care instructions

Can be washed in soap and water. Dry well.
Do not dry on a radiator.

Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

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