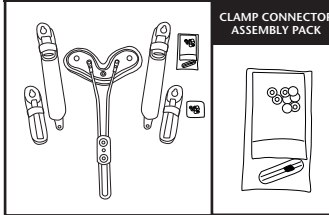
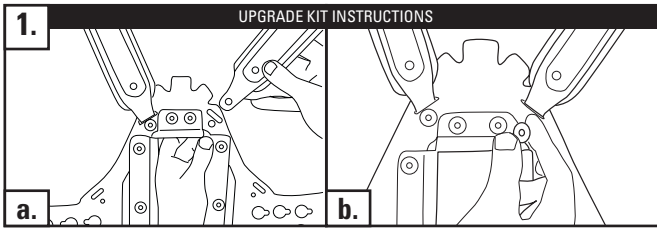
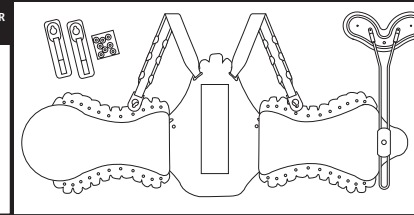
## TLSO COMPONENTS (UPGRADE KIT)

## ASPEN TLSO

### ENGLISH

#### SHOULDER STRAPS

- With the back panel lying on a table, insert the strap through the slot from above.
- Align the rivet (female end away from the patient) and press firmly until it snaps into place. Do this on both sides.

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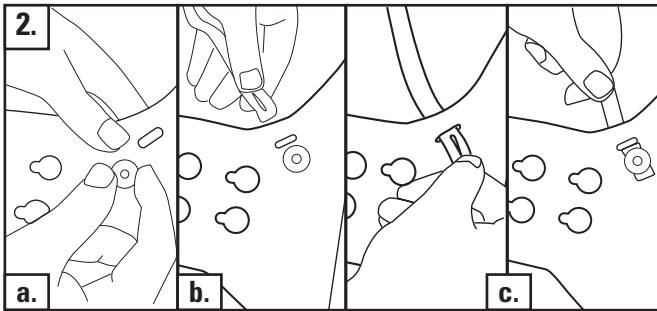
### ENGLISH

#### TORSO CONTROL STRAP

- Align the rivet (female end away from the patient) at the torso control strap attachment point. Press firmly until it snaps into place.
- Lace the strap through the slot from inside.
- Pull the buttonhole at the end of the strap over the plastic rivet and button it on. Pull tight to secure.

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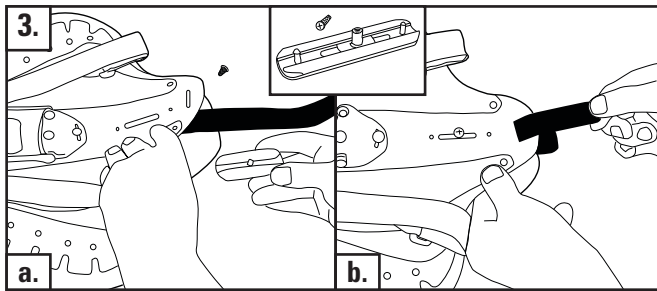
### ENGLISH

#### CLAMP CONNECTOR

- Remove the screw from the clamp connector. Unlace the black start tab and position the clamp connector so the threaded sleeve comes through the slot.
- Align the plastic rivets with the two small holes. Press until they snap into place then replace the start tab.

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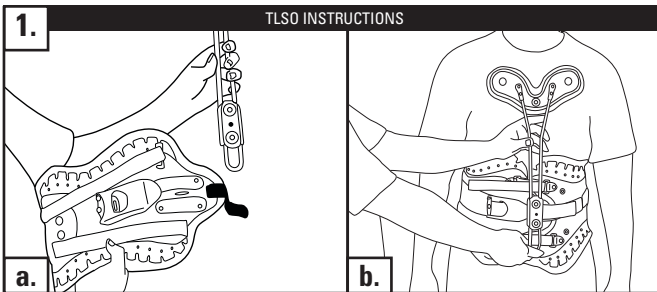
### ENGLISH

#### PREPARATION

- Remove the single screw in the middle of the T-bar mount. Remove the clamp and the posts. Apply and tighten the Aspen LSO. (Instructions at [www.aspenmp.com](http://www.aspenmp.com))
- Hold posts and chest panel in position on the patient. If needed, bend the support posts to fit patient.

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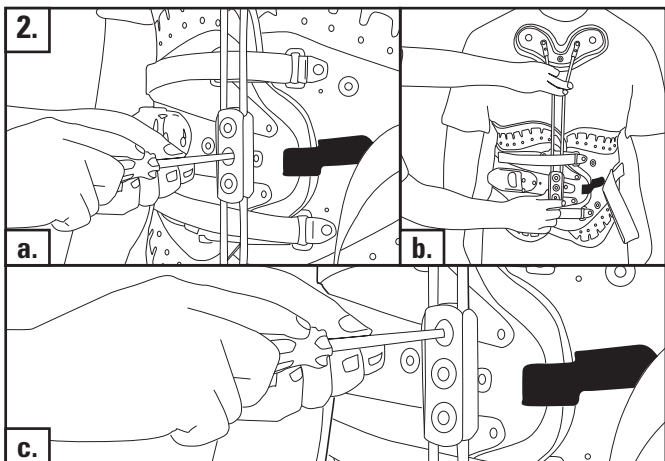
**ENGLISH**  
**CHEST PANEL ATTACHMENT**

a. Unlace the center strap and attach the post clamp and slide it laterally until it is positioned at the patients mid line. Replace and tighten the central screw.  
b. Adjust the height of the chest panel to 1" below the sternal notch. If the bottom of the post extends down too far, bend it outward for comfort.  
c. Tighten upper and lower screws to lock in place. Re-lace center strap through the LeverLatch and secure. Re-position the upper and lower straps so they lie over the support posts.

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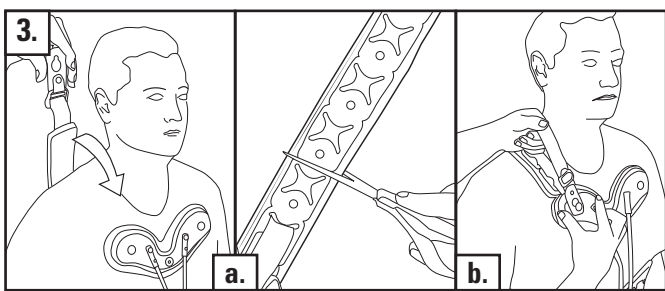
**ENGLISH**  
**SHOULDER STRAP ADJUSTMENT**

a. Hold each shoulder strap in place and determine the appropriate length. If needed, trim the plastic and foam pad with scissors.  
b. Attach the straps to the aluminum rivets using the connecting tabs and tighten until snug.

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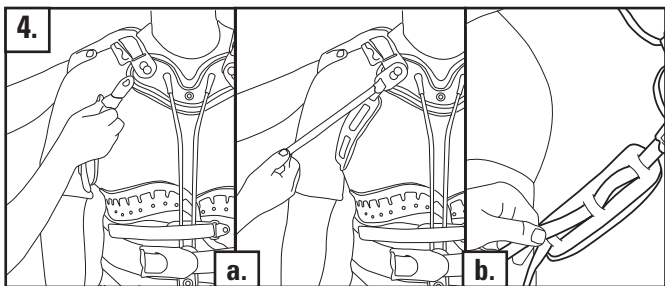
**ENGLISH**  
**TORSO CONTROL STRAP**

a. Refer to upgrade kit installation section 2. Unfold and connect torso control straps to the rivets on the chest panel. Adjust webbing to the desired tightness.  
b. Extra length in the strap can be laced through the foam pad. Trim access webbing and heat seal to prevent fraying.

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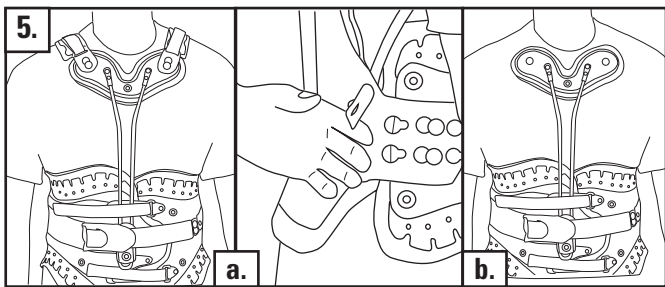
**ENGLISH**  
**STEP DOWN OPTION**

a. If torso straps are not needed, disconnect them where they attach to the back panel.  
b. If shoulder straps are no longer needed, disconnect them from the chest panel and cut off the plastic rivets to disconnect them from the back panel.

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**LIMITED WARRANTY**  
Aspen Medical Products, Irvine, CA 92618, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Aspen Medical Products in the event of breach of warranty shall be to repair or replace the defective product or part(s).  
Aspen Medical Products shall have no obligation under this limited warranty in the event:  
(a) The product was not purchased from Aspen Medical Products or through its authorized channels of distribution;  
(b) The product is altered;  
(c) Any parts not supplied by Aspen Medical Products are inserted into the product; or  
(d) The product is not used in accordance with the Aspen Medical Products, Instructions for Use.

THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF, OR DEFECT IN, THE PRODUCT. ASPEN MEDICAL PRODUCTS SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE, AND ALL OBLIGATIONS OR LIABILITIES ON THE PART OF ASPEN MEDICAL PRODUCTS FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY ASPEN MEDICAL PRODUCTS.  
This warranty gives you specific legal rights and you may have other rights which vary from state to state.  
Warning: To ensure proper use, please review all material.  
A listing of authorized distributors in your area is available upon request.

**READ INSTRUCTIONS BEFORE USE.** Proper training in the use of this device should take place before it is applied. These directions are guidelines only and are not offered as medical recommendations.

THIS PRODUCT IS INTENDED FOR APPLICATION BY LICENSED HEALTH CARE PRACTITIONERS AS DIRECTED BY A PHYSICIAN OR OTHER QUALIFIED MEDICAL AUTHORITY. PERMANENT INJURY MAY RESULT FROM USE OTHER THAN AS DIRECTED BY A PHYSICIAN OR OTHER MEDICAL AUTHORITY. ASPEN MEDICAL PRODUCTS SPECIFICALLY DISCLAIMS ANY OBLIGATION OR LIABILITY FOR INJURY OR DAMAGE BY REASON OF ANY APPLICATION OF THIS PRODUCT. PLEASE SEE THE ACCOMPANYING LIMITED WARRANTY FOR FURTHER INFORMATION IN THIS REGARD.

PATENT PENDING