**medi**

# M.4<sup>®</sup> AGR

Knieorthese


Knee Brace

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
Ortesis de rodilla

Gebrauchsanweisung. Instructions for use. Mode d'emploi. Instrucciones de uso.  
Instruções para aplicação. Istruzioni per l'uso. Gebruiksaanwijzing. Brugsvejledning.  
Bruksanvisning. Инструкция по использованию. Kullanma kılavuzu.


medi. I feel better.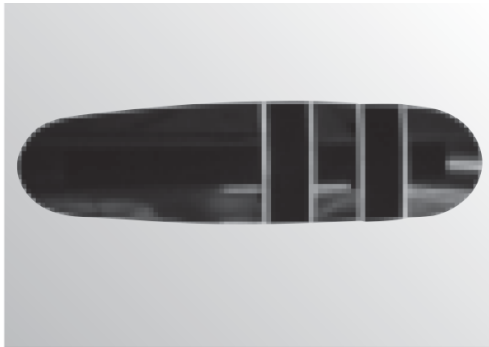

1

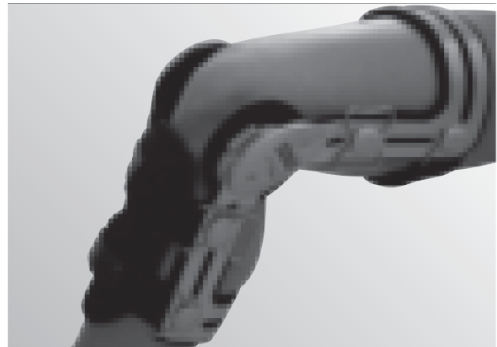2

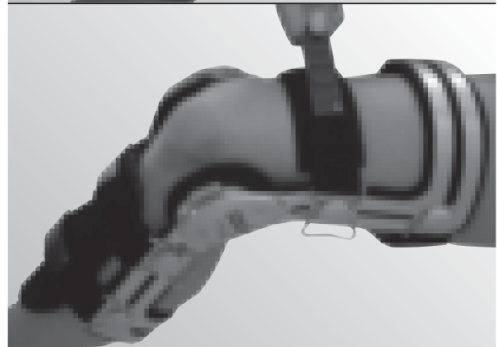3

4

## M.4<sup>®</sup> AGR

### Ändamål

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## M.4<sup>®</sup> AGR

### Intended purpose

The M.4 AGR is a functional knee orthosis. The product is exclusively to be used for the orthotic fitting of the knee and only on unbroken skin.

### Indications

“Genu recurvatum”, otherwise known as hypermobility

### Contraindications

None have been noted to date.

### Important Note

Only change or modify the extension limits on the instruction of your healthcare provider or orthopaedic technician. To avoid hyperextension, always insert extension inserts at 0°. Furthermore, ensure the correct positioning of the brace: the central parts of the hinge should be in line with the top of the kneecap; the hinge should be positioned behind the medial lateral line of the leg.

### Instructions for the orthopaedic technician:

### Adjusting the extension:

Extension limitation inserts (grey; extension limits):  
0°, 10°, 20°, 30°.

- 1 The medial adjustment (inner) and lateral adjustment (outer) of the degrees should always be equal. Extension inserts at 0° are already integrated into the brace.

- 2 To change the inserts, loosen the cover screws, remove the cover and unscrew the insert screws. To easily remove the inserts, gently bend the knee device.
- 3 The extension inserts are marked R-E (RIGHT) or L-E (LEFT) and indicate the respective degrees. Position the insert in such a way that the letter L or R is the same as the hinge plate. Push the new extension insert between the two plates and fasten it with the screws.
- 4 Bend and flex the hinge to check that the inserts have been correctly positioned. Lastly, reposition the hinge cover.

### Replacing the rubber washers:

The rubber washer inside the hinge can be replaced if it becomes worn or in order to increase or reduce the resistance by selecting the correct density of the mechanical device (increase or decrease, respectively, the hardness of the rubber washers). Before replacement, select the rubber washer with the most suitable hardness (shore - hardness of the rubber washer: 80 shore = min, 90 shore = medium, 95 shore = max), cross-referencing the weight and recurvatum degree as shown in Table 1.

**NB:** The rubber washer positioned in the medial part (inner) and the rubber washer located in the lateral part (outer) should always have the same hardness and degree of wear.


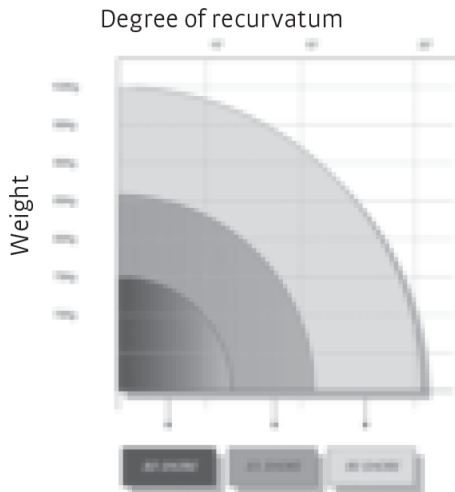
Table 1

**Instructions for use:**

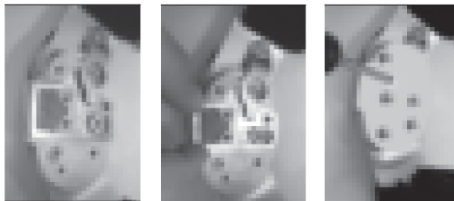- Remove the Velcro tab located on the internal part of the hinge.
- Unscrew the six screws and then remove the plate.
- Carefully extract the rubber washer located in the anterior part of the hinge and insert the new rubber washer as shown in the photograph.
- Reposition the hinge plate making sure to correctly fitted and fasten it using the four 4x6 screws (the shorter screws) for the upper part and the two 4x8 screws for the lower part.
- Lastly, affix the Velcro oval (provided in the packaging) to the cover.

**Applying The Brace**

(Abb. 1) The tibia support is positioned with the wider part towards the top and the concave part cushioning the inside. Remove the paper and affix the two

adhesive Velcro tabs to the outer part of the tibia support so that they correspond with the two tibial straps (#1 and #4).

(Abb. 2) Open all of the Velcro closures. Sit on a chair and position the brace around the affected leg; flex the leg so that it forms a 45° angle. Position the hinges in such a way that the centres of the hinge are positioned in line with the top part of the kneecap.

(Abb. 3+4) Keep the knee in a flexed position and support the two hinges at the back of the leg. Fasten the upper rear strap (#2) and then the upper front strap (#6).

(Abb. 5+6) Keep the knee in a flexed position and support the two hinges at the back of the leg. Fasten the upper rear strap (#2) and then the upper front strap (#6).

(Abb. 7+8) Fasten the lower rear strap (#5); and then the upper rear strap (#3).

For knee devices worn without a tibia support, follow the numbering of the straps: 1, 2, 3, 4, 5 and 6.

To prevent the brace from slipping, gently flex the knee and check that the Velcro straps have been correctly fastened.

**What to do when...**

**....the brace slips?**

- Check that the brace has been fastened and applied correctly.
- The centre of the hinge should be located in line with the top of the kneecap.
- The lower rear strap (#5) should be situated above the calf muscle.
- Remove the coverings from the pads on the knee and wear them directly against the skin. Be aware of any cutaneous reactions; cutaneous

irritations may occur in sensitive persons.

- Do not wear the brace over your clothes.

---

#### Care instructions

Velcro fasteners should be closed for washing. Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the paddings and covers by hand, preferably using medi clean washing agent.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

Check regularly that the joint screws are secure, tighten them if necessary.

---

#### Storage instructions

Keep the product in a cool, dry place and do not expose to direct sunlight.

#### Material composition

Avional, velours, polyester, PU-foam, Elastomer

#### Warranty / contractual guarantee

Consistent with legal guidelines.

#### Disposal

The product can be disposed of in the domestic waste.

#### Your medi Team

Wishes you a quick return to full fitness.