medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at

medi Belgium bvba
Posthoornstraat 13/1
3582 Koersel
Belgium
T +32 011 24 25 60
F +32 011 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Brasil
Rua Neusa 216
Diadema-Sao Paulo
Cep 09941-640
Brazil
T +55 11 3201 1188
F +55 11 3201 1185
sac@medibrasil.com
www.medibrasil.com

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Important notes

This brace is recommended for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability according to the Medical Devices Act will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the support, please consult your doctor or orthotist immediately. Do not wear the brace over open wounds, and use it only as instructed by your doctor or orthotist.

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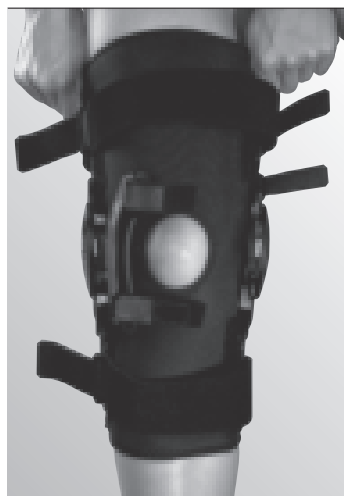
4

medi PT control

Dear Patient,
Your doctor has prescribed the patella correction brace medi PT control for you. Please read through these instructions carefully to help ensure that you fit your brace correctly.

Indications

- Conservative or operative treatment of patella luxation in connection with a slight to medium knee instability
- Lateral misalignment in connection with a slight to medium knee instability
- Following lateral release operations
- Retropatellar cartilage surgery

Contraindications

None known to date.

Setting the flexion / extension limits. Only to be carried out by an orthopedic technician.

Extension limit wedge:

0°, 10°, 20°, 30°, 45°

Flexion limit wedge :

0°, 10°, 20°, 30°, 45°, 60°, 75°, 90°, 120°

Can be immobilised at:

0°, 10°, 20°, 30°, 45°

The medial (inside) and the lateral (outside) wedge settings must always be the same.

1. 0° extension wedges are pre-installed in your knee brace as standard. They are found in the front aspect of the brace's hinge.
2. To replace the wedges first loosen the screws in the hinge cover and take it off. Open the extension or flexion screws. Insert the new wedge so that the letter shown on the hinge plate points upwards (R or L).

3. To replace the extension wedge first open the screw and remove the wedge by pulling it gently out in a forwards direction. Replacement extension wedges can be found in the wedge set marked "E". Slide the new wedge into the hinge from the front and screw it tight.
4. The flexion wedges are installed from the back. Choose the suitable wedge marked with „F“. Insert the wedge so that the letter shown on the hinge plate points upwards. Now insert the screw in the thread and screw it tight.
5. Bend and flex the brace a couple of times and check that the wedges are positioned correctly. Finally, secure the hinge covers.

Important notes

Only change the extension and flexion limits on instruction from your doctor or orthotist. In order to prevent hyperextension, always insert extension wedges or 0° wedges. The wedges define the exact excursion of the orthosis. In difficult soft-tissue conditions, e.g. where the soft-tissues are under compression, it may be necessary to insert the larger wedge to limit the joint movement to the desired excursion. In addition, make sure that the orthosis hinges are correctly positioned: the middles of the hinges should be level with the upper border of the patella. The hinge must lie behind the midline of the leg.

Fitting instructions

- Open all the straps.
- Pull the brace up the affected leg and flex your knee to 30°. Position the hinges so that the centres of the hinges (extension wedges) are on a level with the top edge of the kneecap and ensure that the kneecap is exactly in the centre of the patella recess.

- Now close the straps as follows:
 - Back calf strap
 - Lower back thigh strap
 - Front calf strap
 - Front thigh strap
 - Patella straps so that the patella is taken from the outside to the inside – please make sure that the patella air pad only presses from the outside on the patella – not from the front
 - Upper back thigh strap

Care instructions

Velcro fasteners should be closed for washing. Wash by hand in mild, cold soapy water. Rinse thoroughly. Do not wring out. Leave in the air to dry. Do not iron. Please remove the joints before washing. Soap residues may cause skin irritation or damage to the material. When used normally the brace is maintenance-free. However, after using the brace in water, the hinges can be lubricated using Teflon™ spray.

Storage instructions

Keep in a dry place at room temperature. Protect from heat, direct sunlight and moisture.

Your medi team

wishes you a quick return to full fitness.

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