medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-780
ortho@medi.de
www.medi.de

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at

medi Belgium bvba
Posthoornstraat 13/1
3582 Koersel
Belgium
T +32 011 24 25 60
F +32 011 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Brasil
Rua Neusa 216
Diadema-Sao Paulo
Cep 09941-640
Brazil
T +55 11 3201 1188
F +55 11 3201 1185
sac@medibrasil.com
www.medibrasil.com

medi Danmark ApS
Vejlegardsvej 59
2665 Vallensbæk Strand
Denmark
T +45 46 55 75 69
F +45 70 25 56 20
kundeservice@sw.dk
www.medit danmark.dk

medi Bayreuth España SL
C/Canigó 2 – 6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34 93 260 04 00
F +34 93 260 23 14
medi@mediespana.com
www.mediespana.com

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33 1 48 61 76 10
F +33 1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
Thán Károly utca 20
1119 Budapest
Hungary
T +36 1371 0090
F +36 1371 0091
info@medi-hungary.hu
www.medi-hungary.hu

medi Medical Support Sdn Bhd
medi representative office Asia
Unit No. B-2-19, Block B, No.2,
Jalan PJU 1A/7A
Oasis Ara Damansara, PJU 1A,
47301 PETALING JAYA
Selangor Darul Ehsan
Malaysia
T: +6 03 7832 3591
F: +6 03 78323921
info@medi-asia.com
www.medi-asia.com

medi Middle East
P. O. Box: 109307
Abu Dhabi
United Arab Emirates
T +971 2 6429201
F +971 2 6429070
info@mediuae.ae
www.mediuae.ae

medi Nederland BV
Heusing 5
4817 ZB Breda
The Netherlands
T +31 76 57 22 555
F +31 76 57 22 565
info@medi.nl
www.medi.nl

medi Polska Sp. z o. o.
Chełmońskiego 1a
44-100 Gliwice
Poland
T +48 32 230 60 21
F +48 32 202 87 56
info@medi-polska.pl
www.medi-polska.pl

medi Bayreuth Unipessoal, Lda
Rua Eugénio de Castro Rodrigues, nº2
1700-184 Lisboa
Portugal
T +351 21 843 71 60
F +351 21 847 08 33
medi.portugal@medibayreuth.pt
www.medi.pt

medi RUS LRC
Business Park “Rumyantsevo”
Rumyantsevo, Build. 1
Leninsky Township
142784 Moscow Region
Russia
T +7 495 229 04 58
F +7 495 229 04 58
info@medirus.ru
www.medirus.ru

medi UK Ltd.
Plough Lane
Hereford HR4 OEL
United Kingdom
T +44 1432 37 35 00
F +44 1432 37 35 10
enquiries@mediuk.co.uk
www.mediuk.co.uk

medi CTS

Handgelenkorthese mit Fingerauflage
Wrist orthosis with finger support
Orthèse de poignet main avec repose-doigts
Órtesis de muñeca con soporte para dedos

Gebrauchsanweisung. Instructions for use. Mode d'emploi. Istrucciones de uso. Instruções para aplicação. Istruzioni per l'uso. Gebruiksaanwijzing. Brugsvejledning. Bruksanvisning. Návod k použití. Uputa za uporabu. Инструкция по использованию. Kullanma kılavuzu. Instrukcja zakładania. Οδηγία εφαρμογής. Інструкція з використання.

medi

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Important notes

This brace is recommended for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability according to the Medical Devices Act will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the support, please consult your doctor or orthotist immediately. Do not wear the brace over open wounds, and use it only as instructed by your doctor or orthotist.

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Wrist support

Indications

- Conservative therapy and post-operative aftercare of the carpal tunnel
- Support of the finger and the wrist after injuries
- Post-operative aftercare
- Tendovaginitis
- Arthrosis of the wrist
- Post-traumatic aftercare of distortions of the wrist, the metacarpus and finger

Contraindications

Unstable fractures in the area of the hand.

Applying the support

- The volar splint is supplied anatomically preformed. If necessary, you can adapt the support to the shape of the patient's hand. If possible, make sure that there are no gaps between the support and the arm or hand. The fingers should be supported in a relaxed neutral position (fig. 1)
- Now bend up the side flaps along the forearm, the wrist and the fingertips. (fig. 2)
- Finally fasten the four straps (fig. 3)

Wearing directions

Wearing the medi CTS consistently will support the therapy.

Care instructions

Close the Velcro fastener before washing. Wash the product at 30 °C by hand using a mild detergent such as woolwash. Rinse the product thoroughly. Leave to dry naturally. Do not dry on the radiator or in direct sunlight. Do not remove stains using benzene or chemical detergents. Soap residues can cause skin irritations and wearing of the material.

Storage instructions

Keep in a dry place at room temperature. Protect from heat, direct sunlight and moisture.

Material composition

The splint is made of aluminum. The carrier material is made of polyamide.

Statutory Defects Liability / Contractual Guarantee

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