# Levamed

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# Levamed

## Intended purpose

Levamed is an ankle support with silicone pads. The product is exclusively to be used for the orthotic fitting of the ankle joint and only on unbroken skin.

## Areas of Use

- Feeling of instability in the ankle joint (acute or chronic, after injuries to the ligament)
- Irritations in the joint (e.g. excessive stress, injuries, operations) in connection with swelling and bruising
- Inflammations in the joint (arthritis)
- Painful wear of the joint (arthrosis)

## Side-Effects/Contraindications

There are no known side effects if the correctly sized product is fitted as recommended. If unexpected pain, swelling or numbness occur whilst wearing the support remove it and seek medical advice. Particular care is required where the following conditions are present:

- Diseases or injuries of the skin in the area of use, especially with inflammatory symptoms (excessive warming, swelling or reddening).
- Sensitivity and perfusion disorders of the legs or feet (e.g. in the case of diabetes).
- Lymph outflow disorders, as well as unclear swellings of soft tissues, apart from the area of use.

Any combination with other products (e.g. compression stockings) must be discussed in advance with the responsible doctor.

## Mode of Action/Comment on Wear

Levamed causes a massage effect, which supports perfusion, with interaction between the compressive knitted fabric and the two integrated silicon inlays. This effect reduces swelling and bruising. Irritation and pain are relieved. Muscular control and the stabilisation of the ankle joint improve.

Supports are particularly beneficial during physical exercise. The supports is designed to fit comfortably and may be worn all day. It should be removed during prolonged periods of inactivity (e.g. sitting for a long time or a long car drive, sleeping).

If you suffer excessive pain or discomfort during wear, please take off the product immediately and seek medical advice.

The support is suitable for use in water. After use, please rinse thoroughly.

## Putting on/Taking off

- Fold the top of the support over as far down as the silicone inlay (figure 1).
- Take hold off the support in the middle of the silicone inlays and pull it over the foot until the heel part fits tightly (figure 2).
- Unfold upper part of the support and pull it upwards, if necessary (figure 3). Ensure that the silicone inlays are correctly located either side of the ankle.

Initial application must be made under medical supervision.

---

### Care instructions

Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the product by hand, preferably using a medic clean detergent, or in delicate cycle at 30°C using a mild detergent without fabric conditioners.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

### Note on Storage

Must be stored dry and at room temperature. Must be protected from the sun and moisture.

### Composition of the Material

Polyamide, polyester, elasthan

### Warranty / contractual guarantee

Consistent with legal guidelines.

### Disposal

The product can be disposed of in the domestic waste.

### Your medi Team

Wishes you a rapid recovery!