M.4 X-lock®

Kníeorthese

Knee Brace

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Ortesis funcional de rodilla

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M.4 X-lock®

Intended purpose

The M.4 X-lock is a functional knee orthosis. The product is exclusively to be used for the orthotic fitting of the knee and only on unbroken skin.

Indications

- Meniscus injuries, especially after meniscus resections
- Ruptures of the anterior or both cruciate ligaments, with accompanying injuries
- Ruptures and/or instability of the collateral ligaments
- After reconstruction of the anterior cruciate ligament
- For conservative treatment of knee joint instabilities
- After operative treatment of ruptures to the knee extensor tendons (patellar tendon, quadriceps tendon).
- After cartilage surgery in the area of the kneecap and the femoral groove.
- After restraintment operation of the kneecap (e.g. MPFL reconstruction).

Contraindications

None known at present.

Note regarding use of the knee brace

Please heed the specifications of your physician when using the knee brace, especially with respect to stressing the leg and to knee flexibility.

In general, stress the operated leg only in the extended position - the knee brace joints are locked in the extended position (red area in the joint window). This applies until your physician instructs otherwise.

When the brace is not stressed (e.g. when you are sitting or lying down), you can unlock the joints (green area in the joint window) and bend your leg to the

extent permitted by your physician. BEFORE GETTING UP, however, make sure that the joints are locked in the extended setting and then stand up with your leg stretched (Fig. A-D).

Setting the flexion / extension limits

Extension limit wedge, silver:

0°, 10°, 20°, 30°, 45°

Flexion limit wedge, black:

0°, 10°, 20°, 30°, 45°, 60°, 75°, 90°, 120°

Can be immobilised at:

0°, 10°, 20°, 30°, 45°

For immobilising the orthosis first insert the flexion wedge and then the corresponding extension wedge. The medial (inside) and the lateral (outside) wedge settings must always be the same.

1. 10° extension and 90° flexion wedges are pre-installed in your knee brace as standard (In the UK only 10° extension wedges fitted, remainder included as spare parts).
2. To change the wedges, simply remove the joint cover.
3. A set of wedges is included with the knee brace.
4. The silver extension wedges are marked with "R-E" or "L-E" and the corresponding angle setting. Slide the appropriate extension wedge from the front into the joint. Insert the wedge so that the letter – which is also displayed on the joint plate – faces upwards (R or L). Then insert the bolt into the threaded hole and tighten it.
5. Bend and flex the brace 'joint' a couple of times and check that the wedges are positioned correctly. Finally, secure the joint cover.

Important notes

Only change the extension and flexion limits on instruction from your doctor or orthotist. In order to prevent hyperextension, always insert extension wedges or 0° wedges. The wedges define

the exact excursion of the orthosis. In difficult soft-tissue conditions, e.g. where the soft-tissues are under compression, it may be necessary to insert the larger wedge to limit the joint movement to the desired excursion. In addition, make sure that the orthosis joints are correctly positioned: the middles of the joints should be level with the upper border of the patella. The joint must lie behind the midline of the leg (Fig.12).

Shaping the orthosis

When shaping the orthosis, please ensure that the joints stay parallel. This guarantees that the orthosis will function well and run easily and that there is no excessive wear.

Locking mechanism

The locking slide can be pulled back after removing the joint cover. This can facilitate changing the wedges. When re-attaching the slide, make sure to re-insert it from the rear with the green side first.

Fastening the belt eyes

The belt eyes have been adequately fastened and are, to some extent, deliberately mobile. Please do not tighten the screws; they are tight enough. If you apply too much force, you can break off the screws.

Fitting instructions

Whenever possible, put on the knee brace in the extended position. To do so, set the knee brace to the extended position and lock both joints by moving the slider forward (you can see a red area in the windows on the joints); see Figures 1+2.

Open all the straps. Sit on the edge of a chair and put the brace on. Position the brace so that the centre of the hinges (silver wedges) are on a level with the top edge of the kneecap (Fig. 9). Make sure that the brace is not twisted on your leg.

First fasten the lower front strap #1 (Fig. 4), then the upper rear thigh strap #2 (Fig. 5). Position the two joints close to your joint towards the front of the leg (Fig. 6).

Now fasten the lower rear thigh strap #3 (Fig. 7). Pull the joints slightly forward again. Make sure that you do not pull the joints in front of the midline of the leg.

Now fasten the upper front calf strap #4 (Fig. 8), then the upper back calf strap #5 (Fig. 9 + 12), and finally the front thigh strap #6 (Fig. 10).

Care instructions

The pads are water-repellent; nevertheless, they should be wiped off and dried briefly after you take a shower – the same applies to the straps. Please let any residual moisture air-dry. When used normally the brace is maintenance-free. However, the knee brace should be shaken briefly after you take a shower to remove residual water. The hinges can be lubricated using Teflon™ spray. Check regularly that the joint screws are secure, tighten them if necessary.

Storage instructions

Keep the brace in a cool, dry place and do not expose to direct sunlight.

What can I do if the brace slips?

- Check that the brace is fitted properly. The centre of the brace joints should be level with the upper border of the kneecap.
- Make sure that the back thigh strap (#5) is above the belly of the calf muscle
- Take the covers off the knee pads and wear the pads directly on your skin. Watch for skin reactions; skin irritation may occur in people with very sensitive skin.
- Do not wear the brace over clothes.

Important notes

This product is made for single patient use only. If it is used to treat more than a single patient, the manufacturer's product liability according to the Medical Devices Act becomes invalid. If undue pain or an unpleasant sensation develops while you are wearing the support, please consult your doctor or orthotist immediately.

We always check that our orthoses are semi-sterile. When testing the size, please remember to wear a stocking under.

Material composition

Avional, Velours, Polyester, PU foam

Warranty / contractual guarantee

Consistent with legal guidelines.

Disposal

The product can be disposed of in the domestic waste.

Your medi Team

Wishes you a quick return to full fitness.