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Instructions for Use GB

Dear Customer,

We would like to ask you to follow these Instructions for Use with care. If you should have any queries, ask the doctor treating you, at your nearest stockist or contact us directly.

Function:

The NEURODYN foot levator orthotic device enables free plantar flexion of the foot for a good return of the foot during the swinging phase while walking, thereby ensuring an almost completely natural movement.

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TIP: Ensure that bridle 2 is secured so that it matches over the fastener of bridle 1 to avoid damaging the stockings.

8-shape bridle lacing supports forehead lifting and stabilizes the upper ankle.

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Finally conduct the non-elastic bridle 3 with an upward pull around the outer side of the ankle. Lift the outer edge of the foot slightly with the bridle. Then secure bridle 3 to the large fastener of bridle 2.

TIP: Bridle 3 must be laced vertically upwards.

When applying bridle 2 ensure that the Velcro® fastener for bridle 3 is conducted around the lower leg and positioned in such a manner that bridle 3 can be vertically laced.

The recess in the bridle prevents pressure points on the outside of the ankle.

Das The correctly applied bridle 3 stabilises the ankle and prevents ankle sprains when walking.

Indications:

■ Peroneal paralysis.

All degrees of intensity, especially light to moderate foot and toe lift paralysis (degree of strength 2 - 4).

Contra-indications:

Generally speaking you should see the doctor treating you about the indication and method of wearing an orthopaedic aid and if you have the following conditions:

- Long-lasting, rigid immobilisation, especially in the elderly.
- Allergic, inflammatory or lesion-specific skin alterations (e.g. swelling, reddening) of the body areas being treated.
- Impaired circulation or lymphatic soft tissue swelling.

■ Neurogenically-specific organolectic and skin trophic disorders in the body area being treated (sensory disorders with and without skin damage).

So far not consequential incompatibilities or allergic reactions have been reported.

Side-effects:

With proper use and proper fitting, so far there have been no reports of serious general side-effects.

Local pressure symptoms and impaired circulation can be prevented with sufficient certainty on an individual basis if allowance is made for any contraindications and with non-restricting consistently shaped body fitting.

Important:

The product is to be used by one patient only.

Improper modifications to the product and / or improper use of the product referred to above exempt the manufacturer from product liability.

You ought to discuss potential reciprocal health risks or other disadvantages with specific treatments that may arise in connection with the use of the product with the doctor treating you.

For the product to work effectively over a long period, it must not be worn in conjunction with fatty or acidic remedies, ointments or lotions.

SPORLASTIC GmbH is affiliated to Duales System Deutschland GmbH return and recycling system, 'The Green Spot', and thus meets the requirements of the Packaging Order applicable in the Federal Republic of Germany.

Please dispose of your used packaging at the local recycling depot.

Quality Management System

All SPORLASTIC GmbH products undergo product testing as part of our Quality Management System.

However, should you have any complaints about our product, please get in touch with your stockist or directly with us.

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