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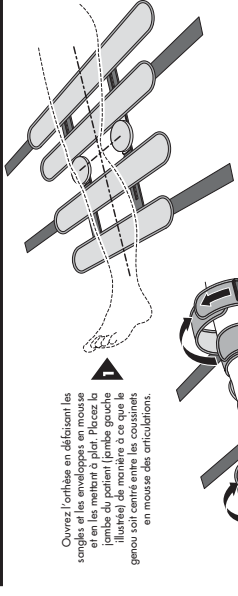
CE
 Conforme alla norma
 93/42/EEC, Annex I, Essential Requirements
 MDS
 Homologation 39
 P.30774 Mannheim, Germany

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
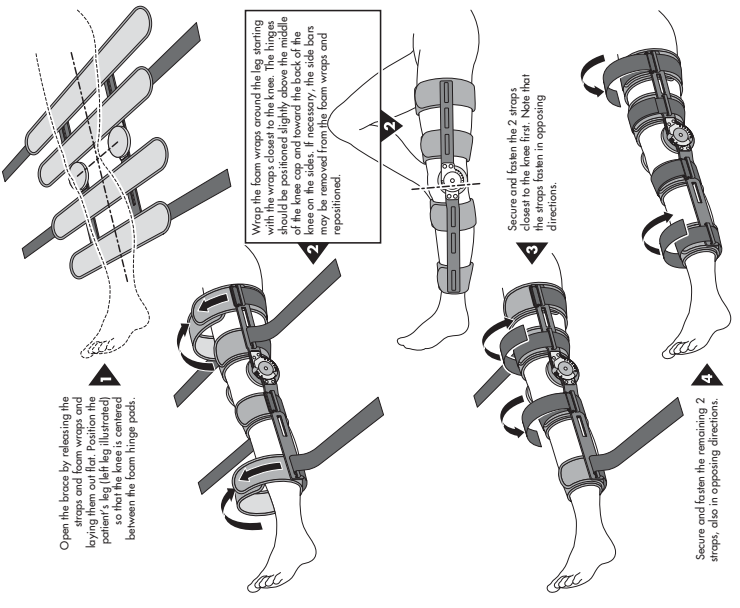
1 CAUTION: FEDERAL LAW RESTRICTS THIS DEVICE TO SALE BY OR ON THE ORDER OF A LICENSED HEALTH CARE PRACTITIONER.
2 CAUTION: FOR SINGLE PATIENT USE ONLY.
3 CAUTION: IF YOU EXPERIENCE PAIN, SWELLING, OR ANY ADVERSE REACTIONS WHILE USING THIS PRODUCT, IMMEDIATELY CONSULT YOUR HEALTH CARE PROFESSIONAL.
4 WARNING: CAREFULLY READ THE INSTRUCTIONS AND WARNINGS PRIOR TO OPERATION.
5 WARNING: THIS DEVICE WILL NOT PREVENT OR REDUCE ALL INJURIES. PROPER REHABILITATION AND ACTIVITY MODIFICATION ARE ALSO AN ESSENTIAL PART OF A SAFE TREATMENT PROGRAM.

POSTOP LITE KNEE BRACE

TIPS ON USE AND CARE OF YOUR POST-OP LITE KNEE BRACE:

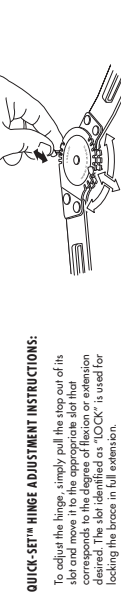- Close the foam wraps in the front to allow easy removal of the brace later.
- The foam wraps may be trimmed with scissors as needed.
- The side bars may be bent as illustrated in Figure 1 to accommodate individual legs. Bend in equal amounts above and below the hinge and on each side.
- Wash foam wraps and straps by hand only and air dry. Extra wraps are available from BREG. Wash the BREG straps after it passes through the ring and the other to push the strap on the leg toward the ring.

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