protect.ROM telescopic

Universalknieschiene
Universal knee brace
Atelle de genou articulée
Órtesis de rodilla

Gebrauchsanweisung. Instructions for use. Mode d'emploi. Instrucciones de uso.
Instruções de uso. Istruzioni per l'uso. Gebruiksaanwijzing. Brugsvejledning.
Bruksanvisning. Návod k použití. Uputa za uporabu. Инструкция по использованию.
Kullanma kılavuzu. Instrukcja zakładania. Οδηγία εφαρμογής. Uputstvo za upotrebu.

protect.ROM telescopic

Universal knäortos

Indikationer

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Universal knee brace

Indications

- Following surgery of the knee joint
- For immobilisation after knee joint injuries and subsequent mobilisation
- For reliable stabilisation of the collateral ligaments after injury or surgery

Contraindications

None known at present.

Adjusting the flexion and extension stops

To adjust the range of movement first turn the opening in the plastic to expose one of the pins. Now press and hold the red button, at the same time turn to set the desired degree figure. Repeat this for the second pin.

Extension limitation: 0°, 10°, 20°, 30°

Flexion limitation: 0°, 10°, 20°, 30°, 45°, 60°, 75°, 90°, 105°, 120°

The medial (inside) and the lateral (outside) wedge settings must always be the same.

Setting the length

Bring the yellow slider into the 'open position'. Pull the orthesis to the desired length and close the lock mechanism again until you hear it snap into place. The medial and lateral lengths must always be equal.

Fitting instructions

- Open all the straps and take the joint braces out of the pads.
- Wrap the calf and thigh pads tightly around the leg and fasten at the front.
- First attach the lateral (outside) brace to the pad. Make sure that the brace joint is level with the patella and the midline of the knee.
- Now attach the medial (inside) brace in the

same way. Make sure that the two joints are at the same level.

- First fasten the upper thigh strap, then the lower thigh strap. Now fasten the other straps, starting with the bottom one and working upwards.
- To take the orthosis off just open the straps and pads and pull the orthosis of backwards.

Important notes

Only change the extension and flexion limits on instruction from your doctor or orthotist to do so.

Care instructions

Please close the Velcro fasteners before washing and remove the orthoses. Wash the product by hand, preferably using medi clean washing agent. Rinse thoroughly. Do not wring out. Leave to dry naturally. Do not iron. Soap residues, lotions and ointments can cause skin irritation and material wear. The product can be worn in both fresh and salt water. After wearing in water, rinse the product thoroughly with clean fresh water. With normal use, the hinge mechanism should not require any maintenance, but it can be lubricated with Teflon spray.

Storage instructions

Keep the brace in a cool, dry place and do not expose to direct sunlight.

Material composition

Aluminum, PU-foam, cotton

Statutory Defects Liability / Contractual Guarantee

Consistent with legal guidelines.