medi GmbH & Co. KG  
Medicusstraße 1  
D-95448 Bayreuth  
Germany  
T +49 921 912-0  
F +49 921 912-780  
ortho@medi.de  
www.medi.de

medi Austria GmbH  
Adamgasse 16/7  
6020 Innsbruck  
Austria  
T +43 512 57 95 15  
F +43 512 57 95 15 45  
vertrieb@medi-austria.at  
www.medi-austria.at

medi Belgium bvba  
Posthoornstraat 13/1  
3582 Koersel  
Belgium  
T +32 011 24 25 60  
F +32 011 24 25 64  
info@medibelgium.be  
www.medibelgium.be

medi Brasil  
Rua Neusa 216  
Diadema-Sao Paulo  
Cep 09941-640  
Brazil  
T +55 11 3201 1188  
F +55 11 3201 1185  
sac@medibrasil.com  
www.medibrasil.com

medi Danmark ApS  
Vejlegardsvej 59  
2665 Vallensbæk Strand  
Denmark  
T +45 46 55 75 69  
F +45 70 25 56 20  
kundservice@sw.dk  
www.meditandmark.dk

medi Bayreuth España SL  
C/Canigó 2 – 6 bajos  
Hospitalet de Llobregat  
08901 Barcelona  
Spain  
T +34 93 260 04 00  
F +34 93 260 23 14  
medi@mediespana.com  
www.mediespana.com

medi France  
Z.I. Charles de Gaulle  
25, rue Henri Farman  
93297 Tremblay en France Cedex  
France  
T +33 1 48 61 76 10  
F +33 1 49 63 33 05  
infos@medi-france.com  
www.medi-france.com

medi Hungary Kft.  
Bokor u. 21.  
1037 Budapest  
Hungary  
T +36 1371 0090  
F +36 1371 0091  
info@medi-hungary.hu  
www.medi-hungary.hu

medi Medical Support Sdn Bhd  
medi representative office Asia  
Unit No. B-2-19, Block B, No.2,  
jalan PJU 1A/7A  
Oasis Ara Damansara, PJU 1A,  
47301 PETALING JAYA  
Selangor Darul Ehsan  
Malaysia  
T: +6 03 7832 3591  
F: +6 03 78323921  
info@medi-asia.com  
www.medi-asia.com

medi Middle East  
P.O. Box: 109307  
Abu Dhabi  
United Arab Emirates  
T +971 2 6429201  
F +971 2 6429070  
info@mediuae.ae  
www.mediuae.ae

medi Nederland BV  
Heusing 5  
4817 ZB Breda  
The Netherlands  
T +31 76 57 22 555  
F +31 76 57 22 565  
info@medi.nl  
www.medi.nl

medi Polska Sp. z o. o.  
Zygmunta Starego 26  
44-100 Gliwice  
Poland  
T +48 32 230 60 21  
F +48 32 202 87 56  
info@medi-polska.pl  
www.medi-polska.pl

medi Bayreuth Unipessoal, Lda  
Rua Eugénio de Castro Rodrigues, n.º2  
1700-184 Lisboa  
Portugal  
T +351 21 843 71 60  
F +351 21 847 08 33  
medi.portugal@medibayreuth.pt  
www.medi.pt

medi RUS LRC  
Business Park "Rumyantsevo"  
Rumyantsevo, Build. 1  
Leninsky Township  
142784 Moscow Region  
Russia  
T +7 495 229 04 58  
F +7 495 229 04 58  
info@medirus.ru  
www.medirus.ru

medi UK Ltd.  
Plough Lane  
Hereford HR4 0EL  
United Kingdom  
T +44 1432 37 35 00  
F +44 1432 37 35 10  
enquiries@mediuk.co.uk  
www.mediuk.co.uk

medi

## medi SAS<sup>®</sup> 15

### Schulterabduktionsschiene Shoulder abduction splint Orthèse d'abduction de l'épaule Férula para abducción de hombro

Gebrauchsanweisung. Instructions for use. Mode d'emploi. Instrucciones de uso.  
Instruções para aplicação. Istruzioni per l'uso. Gebruiksaanwijzing. Brugsvejledning.  
Bruksanvisning. Návod k použití. Uputa za uporabu. Инструкция по использованию.  
Kullanma kılavuzu. Instrukcja zakładania. Οδηγία εφαρμογής. Інструкція з використання.

E006506/06.2013

medi. I feel better.

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#### Important notes

This brace is recommended for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability according to the Medical Devices Act will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the support, please consult your doctor or orthotist immediately. Do not wear the brace over open wounds, and use it only as instructed by your doctor or orthotist.

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# medi SAS<sup>®</sup> 15

**Dear patient,**  
**Your doctor has prescribed the shoulder abduction splint medi SAS 15 for you. Please read these instructions carefully to help ensure that you fit the splint correctly.**

## Indications

- Conservative/postoperative in acute/chronic bursitis
- Conservative immobilisation of a first-time shoulder dislocation
- After removal of calcium deposits
- Impingement syndrome
- Frozen shoulder following mobilisation under narcosis
- Shoulder prosthesis insertion
- After rotator cuff surgery.

## Contraindications

None known at present.

## Important notes

Only change the angle of abduction on instruction to do so from your doctor or orthotist.

## Fitting instructions

The medi SAS 15 can be used on both the right and the left sides.

- Take the sling off the cushion and open the fastener on the hip belt.
- Place the cushion on the hip of the affected side. Wrap the belt around your waist and secure it. Hang the sling around your neck and lay your arm in it.

- Make sure that your elbow is lying comfortably in the sling. Place your hand on the ball. The ball is designed as a rehabilitation tool, by squeezing it regularly; your muscles should regain their strength much sooner.
- Adjust the length of the shoulder strap. If the strap is too long you can undo the Y Velcro strip, trim the strap to the desired length with scissors, and re-attach the fastener. Attach the sling to the cushion with the Velcro strip.
- Place the neck/shoulder padding in the correct position.

## Care instructions

Velcro fasteners should be closed for washing. Wash the product at 30 °C on a delicates setting or by hand using a mild detergent such as woolwash. Rinse the product thoroughly and hang up to dry. Do not dry on the radiator or in direct sunlight. Do not remove stains using benzene or chemical detergents. Soap residues can cause skin irritations and wearing of the material.

## Storage instructions

Please keep the support in a cool, dry place and do not expose it to direct sunlight.

## Your medi team

wishes you a quick return to full fitness.

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medi SAS 15 kan användas på både vänster och höger sida.