eSpine

LUMBAR-X

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EN - eSpine Lumbar-X is a back brace that provides support and compression for lower back problems. The compression of the elastic and the thermoplastic struts make for a stable, convenient and functional brace, which for best effect is usually worn close to the body, over a tank top or similar garment. The stabilising straps make it possible to easily adjust the stability and pressure according to your personal needs. The material is breathable to ensure the brace is cool and comfortable to wear for an extended period of time.

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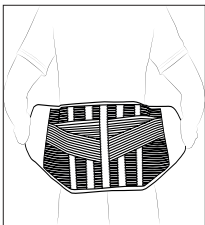
1. Open up the velcro fasteners and position the back brace as shown in the image. The back panel should be centrally placed over the lumbar region. If necessary it can be placed somewhat higher up over the midriff, or lower down towards the hip and sacrum.

2. Close the velcro fasteners around the front of the stomach. The brace should now be sitting in a stable and comfortable position.

3. Pull the straps on each side and fasten them to the velcro on the front of the back brace. The level of support is easily regulated via the straps.

4. A correctly applied eSpine Lumbar-X should sit in a stable and comfortable manner around the lumbar, hip and midriff. When putting the brace on and taking it off, the straps can be placed neutrally on the side panels.

If you have any questions with regard to this product, please contact the party having prescribed or dispensed the product to you, or get in touch with Embreis AB.

Material/Materialer/ Materialer/ Materiaalit/Materials
Polyester/ Polyester/ Polyester/ Polyesteri/ Polyester
Bomull/ Bomull/ Bomuld/ Puuvilla/ Cotton
Polyamid/ Polyamid/ Polyamid/ Polyamidi/ Polyamide
Elastan/ Elastan/ Elastan/ Elastaania/ Elastane

Art.no. 20 cm	Art.no. 24 cm	Art.no. 28 cm	Size	Hip
E506102	E504102	E507102	X-Small	75 - 85 cm
E506103	E504103	E507103	Small	85 - 95 cm
E506105	E504105	E507105	Medium	95 - 105 cm
E506107	E504107	E507107	Large	105 - 115 cm
E506108	E504108	E507108	X-Large	115 - 125 cm
E506109	E504109	E507109	XX-Large	125 - 135 cm

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Om frågor kring produkten uppstår kontakta ansvarig ordinarator, utprovare eller Embreis AB.
Ved spørsmål om produktet, ta kontakt med den som foreskrev støtten, forhandleren eller Embreis AB.
Jos sinulla on kysyttävää tuotteesta, ota yhteyttä tuotteen luovuttajaan, jälleenmyyjään tai Embreis AB:hen.