eWrap Soft

WRAP-AROUND KNEEBRACE

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EN - eWrap Soft is a knee bandage that provides support and compression for tender and overworked knees. The wrap-around design of the knee bandage means that the parts that wrap around the calf and thigh can easily be adjusted for perfect individual fit and comfort.

If you have any questions with regard to this product, please contact the party having prescribed or dispensed the product to you, or get in touch with Embreis AB.

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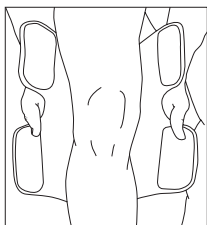
1. Open up the velcro fasteners for the thigh and calf parts. Place the bandage against the back of the knee, with the open wings facing forward. The inner silicon linings should be against the skin, to prevent the bandage from slipping out of position. Centre the bandage so that the opening in the middle is at the same level as the kneecap.

2. Wrap the calf part around the calf and close the velcro fastener so that the bandage is sitting comfortably. Repeat the process for the thigh part. Stability and compression are the determining factors for how tightly the calf and thigh parts should be fastened.

3. A correctly applied eWrap Soft should sit in a stable, comfortable and centred position over the knee.

Material/Materialer/ Materiales/ Materiaalit/Materials
Foam/Skum/Skum/ Vaahtomuovi/Foam
Bomull/Bomull/ Bomuld/Puuvilla/Cotton
Nylon/Nylon/Nylon/Nailon/Nylon

Art.no.	Size	Thigh circ. (A)	Calf circ. (B)
E670102	X-Small	32 - 36 cm	27 - 30 cm
E670103	Small	34 - 40 cm	28 - 33 cm
E670105	Medium	39 - 44 cm	32 - 37 cm
E670107	Large	43 - 49 cm	36 - 41 cm
E670108	X-Large	48 - 53 cm	40 - 45 cm
E670109	XX-Large	52 - 58 cm	44 - 49 cm
E670110	XXX-Large	57 - 63 cm	48 - 53 cm

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Jos sinulla on kysyttävää tuotteesta, ota yhteyttä tuotteen luovuttajaan, jälleenmyyjään tai Embreis AB:hen.
Embreis AB, Tumstocksvägen 11A, S-187 66 Täby