medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-783
ortho@medi.de
www.medi.de

medi Australia Pty Ltd
 83 Fennell Street
 North Parramatta NSW 2151
 Australia
 T +61-2 9890 8696
 F +61-2 9890 8439
 sales@mediaustralia.com.au
 www.mediaustralia.com.au

medi Austria GmbH
 Adamgasse 16/7
 6020 Innsbruck
 Austria
 T +43 512 57 95 15
 F +43 512 57 95 15 45
 vertrieb@medi-austria.at
 www.medi-austria.at

medi Belgium bvba
 Posthoornstraat 13/1
 3582 Koersel
 Belgium
 T: + 32-11 24 25 60
 F: +32-11 24 25 64
 info@medibelgium.be
 www.medibelgium.be

medi Brasil
 Rua Cristovan de Vita, n.º 260, Un. 30,
 Centro Logístico Raposo Tavares,
 Vila Camargo
 Vargem Grande Paulista - SP
 CEP 06730-000
 Brasil
 T: +55-11-3500 8005
 sac@medibrasil.com
 www.medibrasil.com

MAXIS a.s.,
 medi group company
 Sležská 2127/13
 120 00 Prague 2
 Czech Republic
 T: +420 571 633 510
 F: +420 571 616 271
 info@maxis-medica.com
 www.maxis-medica.com

medi Danmark ApS
 Vejlegardsvej 59
 2665 Vallensbaek Strand
 Denmark
 T +45-70 25 56 10
 F +45-70 25 56 20
 kundeservice@sw.dk
 www.medit danmark.dk

medi Bayreuth Espana SL
 C/Canigo 2-6 bajos
 Hospitalet de Llobregat
 08901 Barcelona
 Spain
 T +34-932 60 04 00
 F +34-932 60 23 14
 medi@mediespana.com
 www.mediespana.com

medi France
 Z.I. Charles de Gaulle
 25, rue Henri Farman
 93297 Tremblay en France Cedex
 France
 T +33-1 48 61 76 10
 F +33-1 49 63 33 05
 infos@medi-france.com
 www.medi-france.com

medi Hungary Kft.
 Bokor u. 21.
 1037 Budapest
 Hungary
 T +36 1 371-0090
 F +36 1 371-0091
 info@medi.hu
 www.medi.hu

medi

medi SLK 90

Schulterluxations-Kissen · Shoulder abduction pillow ·
 Orthèse d'abduction de l'épaule · Cojín para abducción del
 hombro

Gebrauchsanweisung. Instructions for use. Mode d'emploi. Instrucciones de uso.
 Instruções para aplicação. Istruzioni per l'uso. Gebruiksaanwijzing.
 Brugsvejledning. Bruksanvisning. Návod k použití. Uputa za uporabu.
 Инструкция по использованию. Kullanna klavuzu. Instrukcja zakładowania.
 Οδηγία εφαρμογής. Інструкція з використання. 使用说明。

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Important notes

This medical device is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability according to the Medical Devices Act will become invalid. If undue pain or an unpleasant sensation develops while you are wearing the support, please consult your doctor or orthotist immediately. Do not wear the brace over open wounds, and use it only as instructed by your doctor or orthotist.

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medi SLK 90

Intended purpose

medi SLK 90 is a shoulder luxation pillow. The product is exclusively to be used for the orthotic fitting of the shoulder and only on unbroken skin or if any wounds have been properly covered.

Indications

- Postoperative immobilisation after posterior stabilisation of the shoulder
- Postoperative immobilisation in multidirectional instability
- Conservative treatment of traumatic posterior dislocation of the shoulder with reverse Bankart lesion.
- Immobilisation in bidirectional anterior and posterior instability of the shoulder.
- Immobilisation after anterior capsule release
- Conservative treatment of non-dislocated fractures of the greater tuberosity
- Latissimus dorsi transfer

Contraindications

- Irreducible posterior dislocation of the shoulder.
- Instability of the shoulder in Parkinson's disease

Fitting instructions

- Open the hip belt's red click-in fastener on the cushion. Place the shaped side of the cushion above the hip bone of the affected side.
- Wrap the belt around your waist and secure it. Adjust the length of the belt to achieve a perfect fit.
- Open the black click-in fastener at the front of the cushion and allow the belt to fall down backwards. Open the straps on the sling and lay your arm in

the sling, making sure that your elbow is right in the corner of the sling. Place your hand on the ball. Close the sling with the Velcro strips.

- Pass the shoulder strap over the healthy shoulder from the back. Now pass your arm through the loop when putting on a jacket.
- Fasten the shoulder strap to the front of the cushion. Adjust the length of the shoulder strap. Make sure that the strap is under tension but does not cut into the skin under the arm. If the strap is too long, you can undo the Y-Velcro strip, trim the strap to the desired length with scissors, and re-attach the Y fastener.

Care instructions

The product should not be worn in water. Velcro fasteners should be closed for washing. Soap residues, lotions and ointments can cause skin irritation and material wear.

- Wash the product by hand, preferably using medi clean washing agent.
- Do not bleach.
- Leave to dry naturally.
- Do not iron.
- Do not dry clean.

Storage instructions

Store in a dry place at room temperature. Protect from heat, direct sunlight and moisture.

Material composition

PU foam, polyamide

Liability

The manufacturer's liability will become void if the product is not used as intended. Please also refer to the corresponding safety information and instructions in this manual.

Disposal

The product can be disposed of in the domestic waste.

Your medi team

wishes you a quick return to full fitness.

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