

Using your AFO




Attaching the strap

Pull the strap across your leg and into the window on the calf cuff. You should feel the plastic button on the back side of the strap lock into place.



Tightening the strap

Use the Boa® Fit System to dial in the perfect fit. Pull up on the dial to release the strap. Push in and twist to tighten.

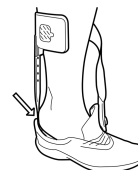

Keep your AFO in your shoe

When not using your AFO, it's best to keep it installed in your shoe under the insole. Simply insert your foot and attach the strap next time you wear it.

Other information

Cleaning your AFO

Carbon Fiber / Plastic parts:

- Wipe with a clean, damp cloth

Liner and Strap:

- Hand wash with mild detergent.
- Pat surface dry with towel and leave to air dry completely.
- Do not use bleach, fabric softener, chlorinated water, or tumble dry in a machine dryer.

Videos and accessories

Our website is a great resource to learn more about your Elevate product.

Find videos on how to properly wear your AFO and more. You can also buy extra liners, straps, and Elevate gear!

Learn more:
elevatemovement.com

If you have any issues with your product, please contact the clinician or clinic who performed your initial fitting to start a warranty claim with the Elevate team.

077-1027-B • Rev 01/23

More questions? We're here to help.
Reach out us today or find more info online.

(888) 504-0230

Monday - Friday | 9am-8pm EST

www.elevatemovement.com

