step

Instructions for use
(IFU)

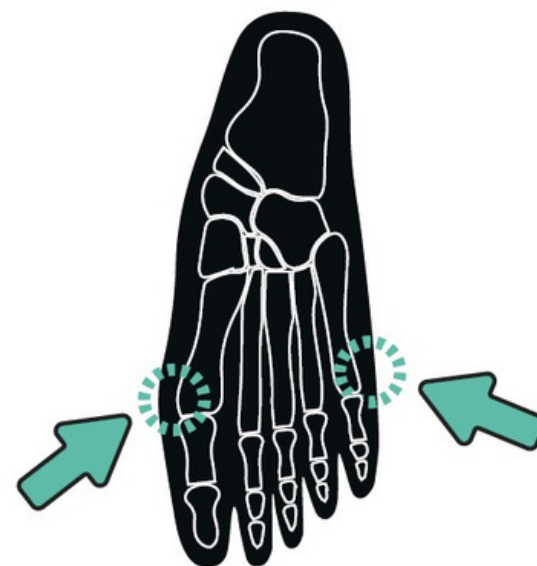
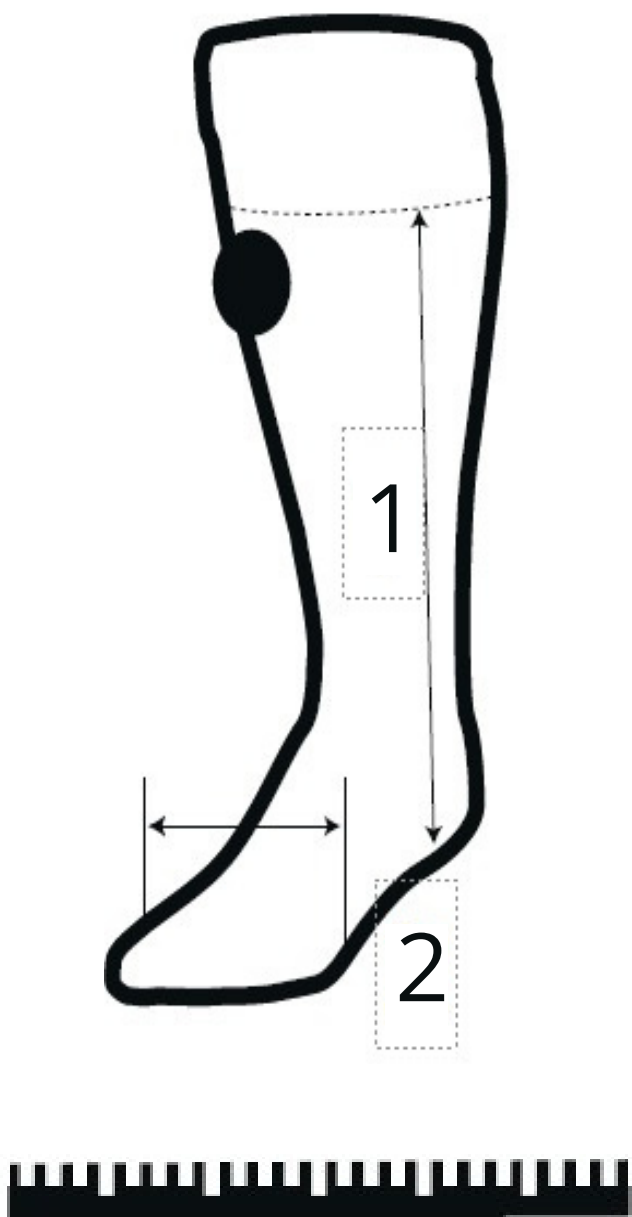
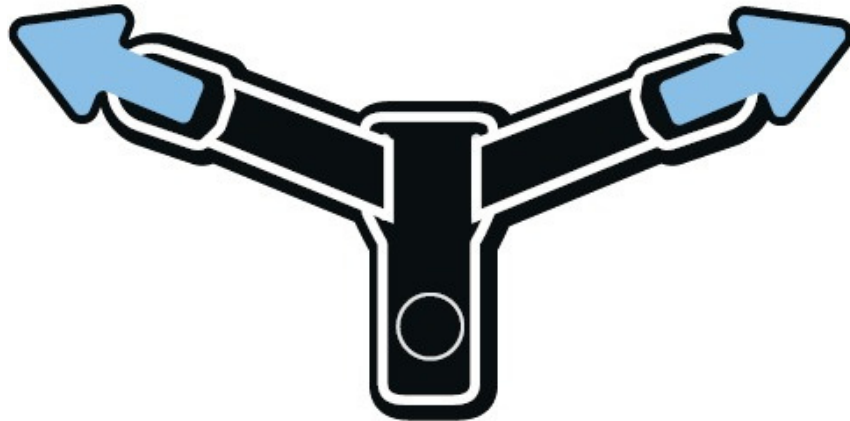
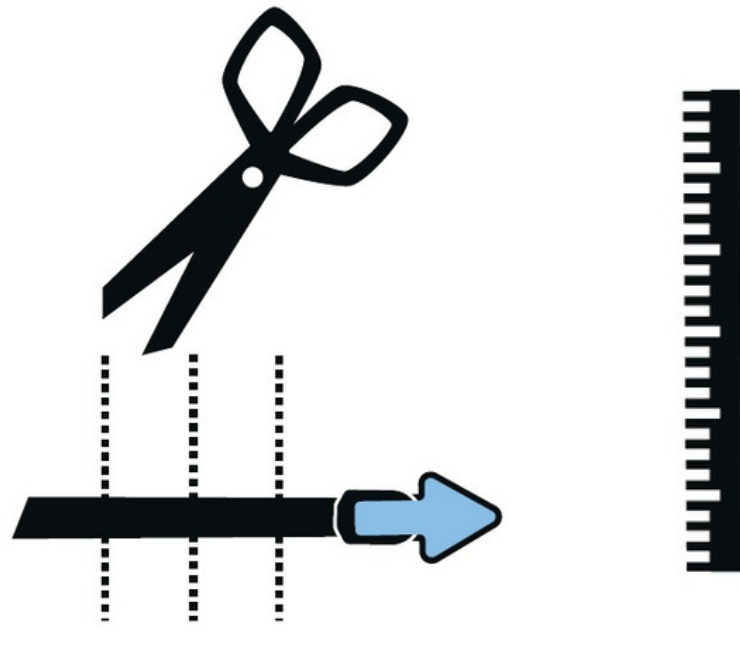
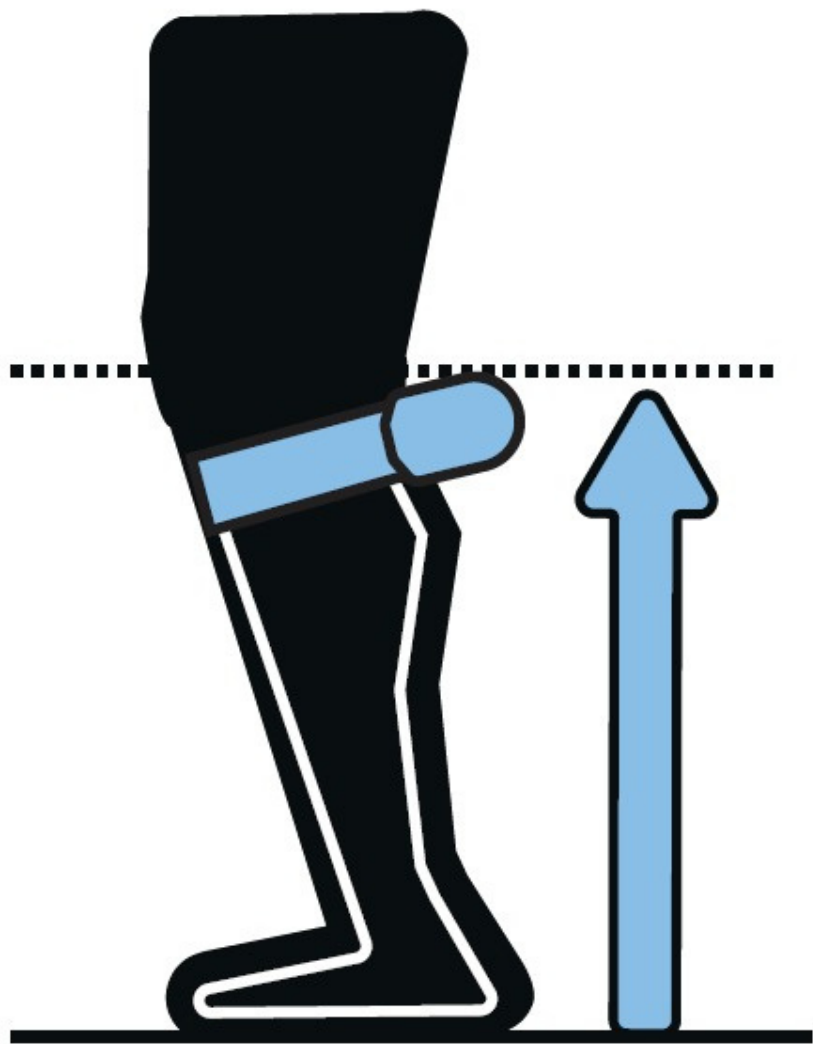
Contents

	Sizes table	2
EN	Instructions for use	6
SE	Bruksanvisning	7
NO	Bruksanvisning	8
DK	Brugsanvisning	9
FI	Käyttöohje	11

Size table

	US M	US W	1	2
SA	2.5 - 4	4 - 5.5	10" - 12.5"	3" - 3.5"
S	4 - 6	5.5 - 7.5	10" - 12"	3.25" - 4"
MA	5 - 8.5	6.5 - 10	11.5" - 14"	3" - 3.75"
M	6 - 9.5	7.5 - 11	11" - 13.5"	3.5" - 4.25"
LA	8 - 11.5	9.5 - 13	13" - 16"	3.25" - 4"
L	9 - 12	10.5 - 13.5	12.5" - 15.5"	3.75" - 4.5"
XL	11.5 +	13 +	14.5" +	4.25" +

Product Overview - English

Intended use

The Sstep is to be used exclusively for orthotic fittings of the foot and ankle and is exclusively suitable for contact with healthy and intact skin.

Indications and Effects

The Sstep supports the foot in case of drop foot with no more than mild spasticity, e.g. after stroke, traumatic brain injury, multiple sclerosis, neural muscle atrophy, peroneal palsy etc.

The Sstep is best suited for patients with:

- No or mild impairment of motor knee control
- No or mild impairment of plantar flexors
- Ambulatory patients, walking indoors as well as outdoors

In case of foot deformity, the product is indicated when the foot deformity can be corrected through use of an additional insole and/or a varus/valgus support strap in combination with a sturdy shoe. The product provides the user with a more natural gait pattern, a faster and more stable walk. The toes and foot are lifted up during the swing phase and foot slap and heel strike is prevented.

Contraindications

- Contractures
- Moderate to severe spasticity in the lower leg
- Any ulcers of the lower limb
- Severe edema
- Moderate to severe foot deformities

In case of the following indications, consultation with a physician is required: Skin diseases / injuries, inflammatory symptoms, prominent scars, lymphatic disorders, sensory and circulatory disorders of the lower extremities.

Safety

- The patient must be instructed in the proper use/care of the product by reading IFUs carefully
- The daily duration of use and period of application are dependent on the medical indication and comfort level of the user
- The product is intended for use on one patient only
- If the orthosis is applied too tight, it can cause local pressure and, in some cases even restrict blood flow or nerves. Do not apply it too tight.
- Consult a physician immediately if you experience unusual changes, such as increase in pain
- The ability to drive a vehicle when wearing the product should be determined on a case-by-case basis
- This product is not flame-resistant. Keep the product away from flames or other heat sources
- The product should not come into contact with grease or acidic agents, unguents and lotion. This may reduce the product's useful life
- This product is not made with natural rubber latex
- Use of this product is not a guarantee against injury

Warranty

60-Day Limited Warranty

We stand behind the quality of our products. Our products are warranted to be free from manufacturing defects in materials and workmanship for 60 days from the date of purchase. This warranty is valid only for the original purchaser and is not transferable.

Warranty Registration

To qualify for the 60-Day Limited Warranty, you must register your product within 30 days of purchase on: www.dropfoot.com/warranty.

Warranty Claims

If you experience a manufacturing defect within the warranty period, please contact our customer service team at hello@elevatemovement.com with your registered information and a description of the issue. We will provide instructions on how to return the defective product for inspection and replacement or refund.

Exclusions

This warranty does not cover:

- Normal wear and tear
- Damage caused by improper use, misuse, or neglect
- Products that have been altered or modified
- Products not purchased from our website or authorized retailers

Limitations

Our liability is limited to the replacement or refund of the defective product. We are not responsible for any incidental or consequential damages resulting from the use of our products.

Legal Rights

This warranty gives you specific legal rights, and you may also have other rights which vary from state to state or country to country.

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