eLock

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EN - A correctly fitted eLock fits comfortably while locking the arm and shoulder in place. The shoulder padding is evenly distributed across the shoulder area. The cuff and overarm clip are easily opened, thus allowing mobility exercises for the arm and shoulder. The shoulder and torso straps need only be unfastened when removing the brace.

If you have any questions with regard to this product, please contact the party having prescribed or dispensed the product to you, or get in touch with Embreis AB.

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1. Place the body strap directly under the chest with the Velcro fastening (A) turned towards the healthy side. Cut the body strap so that it overlaps by approximately 20 cm. Place the overarm strap (B) on the injured side and turn the opening forward. Cut the overarm strap so that it overlaps 10 cm.

2. Place the wrist cuff against the body area at wrist level. The buckle should be facing up and the cuff should be fastened using the Velcro strap.

3. Fasten the shoulder strap at the back of the torso strap and place the shoulder padding so that the pressure is spread evenly across the shoulders.

4. Fasten the shoulder strap with the buckle attached to the wrist cuff.

5. Adjust the length of the shoulder strap by pulling on the end of the strap. Cut the shoulder strap (A) to the desired length and place the Velcro in the end directly over the buckle.

6. Adjust the length and angle of the thumb grip by moving it to a suitable position on the shoulder strap. The thumb grip is designed to provide comfort and relief for the wrist but can be easily removed if so desired.

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Universal

Material/Materialer/ Materialer/Tarvikkeet/Materials

Foam/Skum/Skum/ Vaahto/Foam
Bomull/Bomull/ Bomuld/Puuvilla/Cotton
Nylon/Nylon/Nylon/Nylon/Nylon

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