eLux

PATELLA WA/ PATELLA MINI

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EN - The eLux Patella WA/ Mini is a stabilising kneecap brace designed as a wraparound model for easy application. A lateral pad and strap stabilise the kneecap. The wraparound design provides for individual adjustment to the shape of the leg. The brace is manufactured in a material that provides good compression and elasticity for both optimal fit and comfort.

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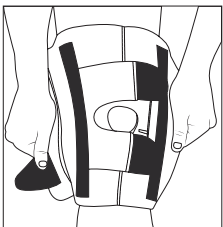
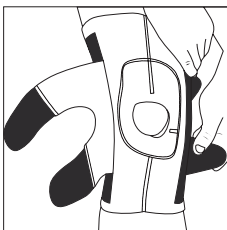
1. Open the Velcro straps and place the brace over the knee.

If the opening around the kneecap is too small and this causes unwanted pressure, simply make the hole larger using scissors.

Material/ Materialer/ Materialer/ Materiaalit/ Materials
Neopren/ Neopren/ Neopren/ Neopreeni/ Neoprene
Lycra/ Lycra/ Lycra/ Lycra/ Lycra
Nylon/ Nylon/ Nylon/ Nylon/ Nylon

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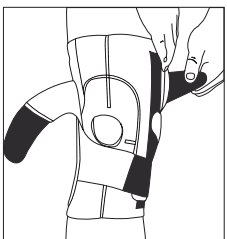
2. Place the pad on the outside/laterally close to the kneecap/patella. Attach the Velcro straps to the inside of the brace. Begin with the lower strap, then the upper and finally the middle strap. Once fastened, adjust all of the straps to achieve a good fit, compression and comfort.

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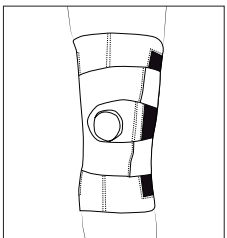
3. Tighten the straps on the outside of the brace and attach to the inside. Begin with the lower strap, then the upper strap. The pressure on the kneecap/patella should feel stable and secure.

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4. A correctly applied eLux Patella WA/ Mini will sit comfortably and stably with the opening located over the kneecap and the pad on the outside of the kneecap.

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Always go through this process when applying the brace in order to achieve the best fit and functionality. Never use a damaged brace.

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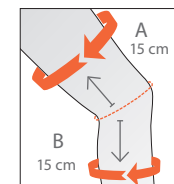
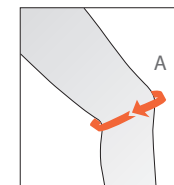
If you have any questions about the product, please contact the responsible prescriber, fitter or Embreis AB.

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Om frågor kring produkten uppstår kontakta ansvarig ordinator, utprovare eller Embreis AB.
Ved spørsmål om produktet, utprøver eller Embreis AB.
Tuotteeseen liittyvissä kysymyksissä ota yhteyttä lähetteen antajaan, sovitajaan tai Embreis AB:hen.