eSkin

PROSTHETIC GLOVES

E41, E42, E43, E44, E45,
E46, E47, E48

ENGLISH

Important notes

This medical device is made for single patient use only. If it is used for treating more than one patient, the manufacturer's product liability will become invalid.

Intended use

The eSkin Glove is a cosmetic cover designed to give prosthetic hand and arm a more natural appearance.

Features

- 18 colors. See our color guide for reference.
- Standard or Natural
- Natural with multilayer construction with fibers for a lifelike appearance.
- Pre-treated with talcum powder for easier donning.
- Durable and stain resistant easily cleaned with soap and water.
- Water resistant.
- Low friction against fabric.
- Recyclable - Free from all heavy metals and phthalates.

eSkin Color Guide

eSkin Gloves comes in 18 different skin tones covering most skin types. For your convenience we made the Color Guide to help determine which color to choose for your patient.

- Art.no - E4N100 - Natural
- Art.no - E4S100 - Standard

Instructions

The material you need

- Heatgun
- Scissor
- Vaseline

How to apply eSkin Gloves

- Cut the prosthetic glove to the desired length.
- eSkin Cosmetic gloves are pre-treated with talcum powder for easier donning. If needed use Vaseline on the prosthetic hand before donning the glove
- Preheat the glove lightly with a heatgun before donning, to make the process even easier.
- Position the prosthetic glove on the prosthetic hand for donning.
- Hold the prosthetic glove at the sleeve end and pull it over the prosthetic hand in one movement.
- If needed, adjust the prosthetic glove to fit the prosthetic hand properly.

CAUTION: Overstretching will cause tension on the eSkin and a greater risk of splitting.

CAUTION: Be careful not to overheat the skin.

Care and Maintenance

eSkin Gloves features the latest innovation in high-quality synthetic plastic polymer. It is water and stain resistant and easily cleaned with soap and water. Do not store your eSkin in direct sunlight.

CAUTION: Do not use acetone or any other solvents.

Warranty

eSkin Gloves is warranted for 6 months from the date of shipment. Items under warranty will be replaced at no charge. The warranty will be void if the skin has been exposed to high heat, corrosive environments, or if the item has been used in excessively abusive activities.

Waste Management

The product is classified as recyclable PVC.

Complaints

In the event of complaints related to the Product, please contact your healthcare product retailer. Only serious incidents that may result in a significant deterioration of health or death should be reported to the manufacturer or the competent authority of the member state. Serious incidents are defined in Article 2, number 65 of Directive (EU) 2017/745 (MDR).

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Embreis AB
Embreis AB | Tumstocksvägen 11B | S-187 66 Täby | Sweden