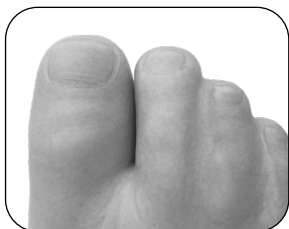
eSkin

LEG COVER

Standard Toe (S)

Split Toe (T)

1.

2.

3.

4.

5.

6.

EN - Instruction for use	p 5
SE - Bruksanvisning	p 7
DK - Brugsanvisning	p 9
NO - Instruksjoner for bruk	p 11
FI - Käyttöohje	p 13
DE - Handbuch	p 15
ES - Instrucciones de uso	p 18
CS - Návod k použití	p 21

EN - Important notes

This medical device is made for single patient use only . If it is used for treating more than one patient, the manufacturer's product liability will become invalid.

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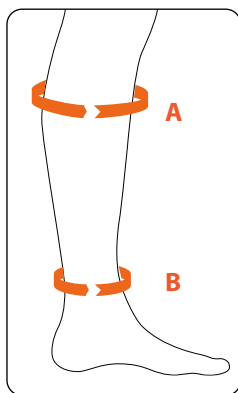
ENGLISH

Intended use

The eSkin Leg Cover is a cosmetic cover designed to give prosthetic legs a more natural appearance.

Features

- 18 colors. See our color guide (art.no E4N100) for reference.
- Multilayer construction with fibers for a lifelike appearance.
- Standard or Split toe.
- Durable and stain resistant easily cleaned with soap and water.
- Flexible and pre expanded each size fits two-foot sizes.
- Water resistant.
- Low friction against fabric.
- Recyclable - Free from all heavy metals and phthalates.

eSkin Color Guide

eSkin Leg Cover comes in 18 different skin tones covering most skin types. For your convenience we made the Color Guide to help determine which color to choose for your patient. Art.no - E4N100

Instructions

All eSkin Leg Cover are expanded for easier donning and are extra expanded over the ankle.

The material you need

- Heatgun
- Scissor
- Knife
- Glue (no acetone)
- Sandpaper
- Vaseline
- Alcohol
- A stick

How to apply eSkin Leg Cover

- Turn the eSkin Leg Cover inside out, except the fore foot and the toes.
- Apply vaseline on the prosthesis foot. Make sure to apply on the heel, but not on the prosthesis leg.
- Apply the Leg Cover, starting with the toes. Be sure toes and heel are orientated correctly. Grab the toe and firmly pull the eSkin over the foot and the heel part. Avoid air pockets and wrinkles on the plantar surface. (1.)
- Before stretching the eSkin all the way up on the leg, place a stick ex. a ruler between the eSkin and the prosthesis leg. This makes a gap allowing air bubbles out. Smooth out wrinkles and remove air. Skin should be stretched just enough to remove slack and wrinkles.

CAUTION: Overstretching will cause tension on the skin and a greater risk of splitting.

- Heat lightly with an heat gun to shrink and conform skin to contours. Keep the heat gun in motion so that the heat is evenly distributed. (3.)

CAUTION: Be careful not to overheat the skin.

- Prepare skin and prosthesis to be glued by sanding the socket's surface with sandpaper until it is rough, then clean with alcohol before gluing. It is not necessary to glue skin in any area other than the proximal brim. Make sure to seal edges well. (4.)

CAUTION: Do not use acetone based glue.

- Use a scissor to cut off excess material above the proximal brim following the curve. Then trim the edges using a sharp scalpel. (5.)
- Make the final touch with sandpaper on the edge. (6.)

Care and Maintenance

eSkin Leg Cover features the latest innovation in high-quality synthetic plastic polymer. It is water and stain resistant and easily cleaned with soap and water. Do not store your eSkin in direct sunlight.

CAUTION: Do not use acetone or any other solvents.

Warranty

eSkin Leg Cover is warranted for 6 months from the date of shipment. Items under warranty will be replaced at no charge. The warranty will be void if the skin has been exposed to high heat, corrosive environments, or if the item has been used in excessively abusive activities.

Waste Management

The product is classified as recyclable PVC.

Complaints

In the event of complaints related to the Product, please contact your healthcare product retailer. Only serious incidents that may result in a significant deterioration of health or death should be reported to the manufacturer or the competent authority of the member state. Serious incidents are defined in Article 2, number 65 of Directive (EU) 2017/745 (MDR).

Instruction Video

Scan the QR Code with your mobile device to see instruction video.

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Embreis AB
Wellkang Ltd
Embreis AB | Tumstocksvägen 11B | S-187 66 Täby | Sweden