Comfit AFO[®]

 Användarguide	2
 Brugervejledning	6
 Brukerveiledning	10
 Käyttöopas	14
 User Guide	18
 Benutzerleitfaden	22
 Guide de l'utilisateur	26
 Guía del usuario	30

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Tel. + 46 8 410 621 00
info@embreis.com
www.embreis.com

IMPORTANT ADVICE AND TIPS

In this brochure you will learn more about your new assistive device.

The purpose of Comfit AFO is to maintain your foot in a neutral position and prevent foot drop. This helps you walk more naturally while reducing the risk of tripping.

The flexibility and design help to provide a more stable and secure gait.

With its function, discreet design, and low weight, it is our sincere hope that you will be satisfied with Comfit AFO.

High load
Movements involving high load may damage the material.

Standing up
Place your feet firmly on the floor and use the armrests for support.

Walking stairs
Always place the entire surface of the foot on the stair tread.

Stability
Walking poles contribute to increased stability and safety.

PUTTING ON AND TAKING OFF

PUTTING ON COMFIT AFO

- To put on the orthosis, place the footplate in a suitable shoe, under the shoe's insole or an insert.
- Ensure that the orthosis is pushed all the way back against the heel counter of the shoe.
- Open the shoe properly and insert the foot. The calf shell should rest comfortably against the calf.
- Tighten the padded calf strap.
- Tie the shoes securely. Check that the orthosis is correctly positioned and does not cause chafing.

REMEMBER

- Stable shoes are best!
- Long knee-high cotton socks.
- The footplate must always be placed under an insert or the shoe's insole.
- Check your AFO daily to detect wear in time.
- Contact your healthcare provider if anything causes chafing or feels incorrect.

REMOVING COMFIT AFO

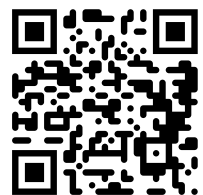
- Open the hook-and-loop strap.
- Untie the shoe and remove the foot.
- You may leave your AFO in the shoe.

WE WOULD LIKE TO KNOW WHAT YOU THINK ABOUT COMFIT AFO

By participating in our survey, you provide important information that helps us improve Comfit AFO. We donate SEK 20 to the Neuro Association (Neuroförbundet) for each completed survey.

Complete the survey directly on your mobile phone (requires a smartphone) by pointing the camera at the QR code below. All responses are compiled into statistics and are completely anonymous.

If you prefer to complete the survey on a computer, enter the following address in your web browser:
www.embreis.com/surveys
You may only participate once.

INSTRUCTIONS FOR USE – YOUR COMFIT AFO

FOR YOU ONLY

This assistive device has been fitted and adapted specifically to your needs and may only be used by you.

DO NOT ADJUST ON YOUR OWN

All adaptations and adjustments must be performed by healthcare professionals. You must not bend, heat, or adjust the assistive device on your own.

PROPER FOOTWEAR MAKES A DIFFERENCE

Comfit AFO is manufactured from a thin and flexible material and therefore fits in most standard shoes. For optimal function, you need to use a shoe with good stability and a heel height of approximately 1–2 cm. If you are unsure, consult your healthcare provider.

Do not use slippers or sandals, as they do not provide sufficient support for the orthosis.

Check the shoe before use. Sand and gravel may damage the orthosis.

AVOID HIGH LOAD

Comfit AFO is designed to enable an active daily life for individuals with

foot drop syndrome, but it is not suitable for jumping, running, or other physically strenuous activities.

CHOOSE THE RIGHT SOCKS

For optimal comfort, we recommend knee-length cotton socks. They reduce the risk of chafing and protect the leg.

PROTECT THE FOOT AND LOWER LEG

The footplate of Comfit AFO must always be placed in the shoe, under an insert or the insole. If you have difficulty removing the insole, contact your healthcare provider.

The calf shell where the hook-and-loop strap is attached has a softly padded inner surface that protects the leg from pressure and chafing. If it becomes worn, it can be replaced with a new one. Your healthcare provider can assist with this.

CARING FOR YOUR AFO

Clean the brace, padding, and straps with mild soap and water. Then allow it to air dry. Also avoid drying in direct sunlight.

DRIVING

We do not recommend driving a car while wearing your orthosis, as an AFO restricts foot mobility and thereby your reaction capability. Always consult your healthcare provider.

IF SOMETHING FEELS WRONG

If anything feels uncomfortable, for example if the brace causes chafing or if you experience pain, numbness, or swelling, you should contact your healthcare provider as soon as possible. This also applies if you notice redness, wounds, or skin irritation.

DAMAGE AND WEAR

Check your orthosis daily. If you discover cracks or other damage, the orthosis must be replaced; contact your healthcare provider.

IN CASE OF AN ACCIDENT

If you suffer a serious injury in connection with using the assistive device, you must immediately contact your healthcare provider. You may also contact the manufacturer if you have questions or wish to report an incident.

SERVICE LIFE OF THE ASSISTIVE DEVICE

If you follow the advice in this manual, the expected service life is 1–2 years with daily use.

DISPOSAL

Comfit AFO must be disposed of in accordance with local requirements and guidelines in the country where the product is used.

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Embreis AB